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
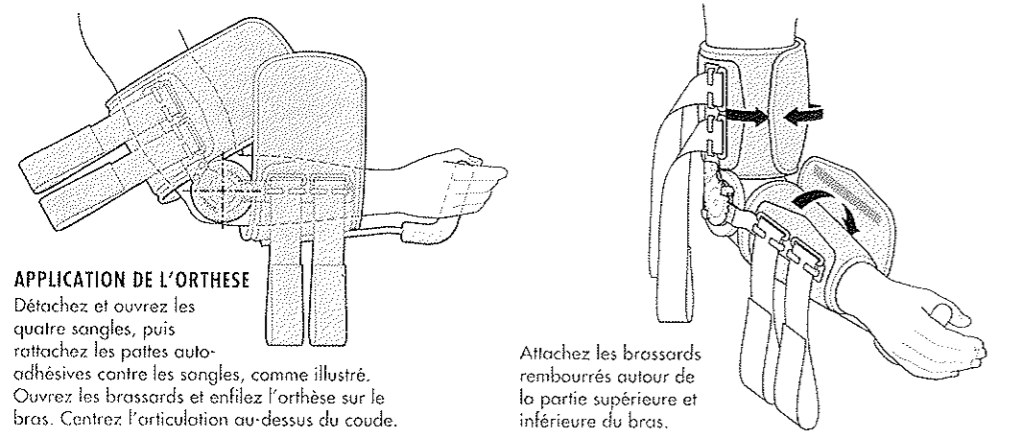
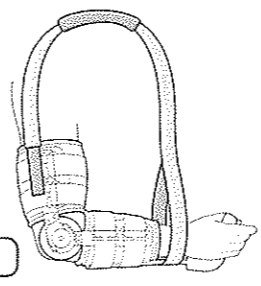
**CE**  
 EC responsible person  
 MDSS  
 Burschardstrasse 1  
 D-30163 Hannover, Germany

BREG Inc.  
 Vista, CA 92081 U.S.A.  
 Telephone (800) 321-0607  
 (760) 599-3000  
 Telefax (760) 598-6193  
 PN-1.01300 REV C


**CAUTION:** FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.  
**CAUTION:** FOR SINGLE PATIENT USE ONLY.  
**CAUTION:** IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.  
**WARNING:** CAREFULLY READ USE INSTRUCTIONS AND WARNINGS PRIOR TO OPERATION.  
**WARNING:** THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM

**PostOp Elbow Brace**

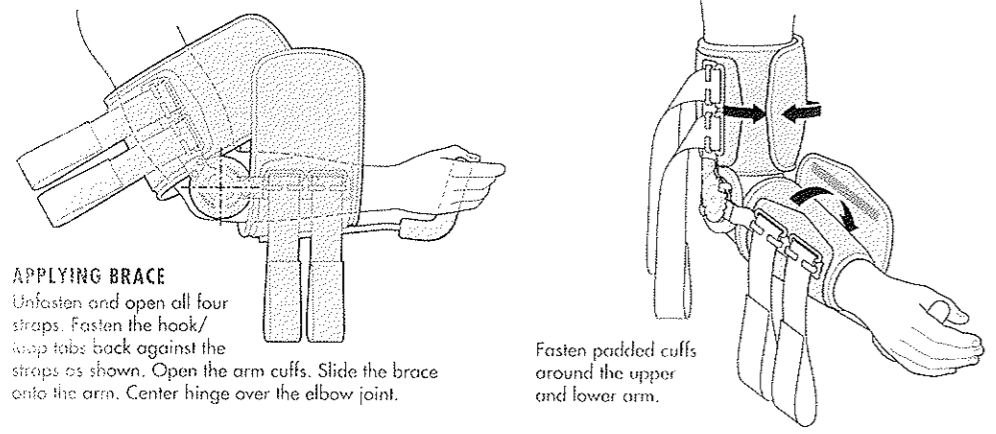**APPLYING BRACE**  
 Unfasten and open all four straps. Fasten the hook/loop tabs back against the straps as shown. Open the arm cuffs. Slide the brace onto the arm. Center hinge over the elbow joint.

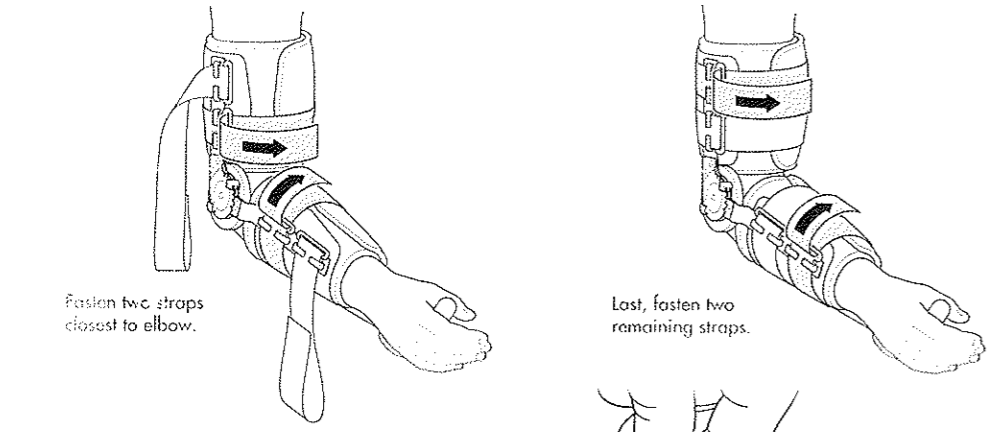
Fasten padded cuffs around the upper and lower arm.

Fasten two straps closest to elbow.

Last, fasten two remaining straps.

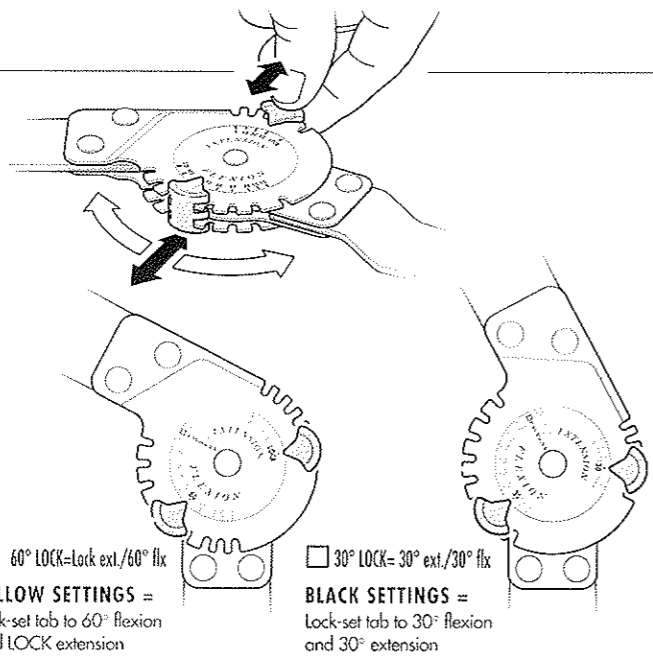**HINGE ADJUSTMENT**  
 To change settings, pull the lock-set tabs out as shown, and move to the desired position. Set hinge to the appropriate extension or flexion stop desired by following the key protocols illustrated below:


**ORANGE SETTINGS** = Lock-set tabs to 10° flexion and 105° flexion

**YELLOW SETTINGS** = Lock-set tabs to 60° flexion and LOCK extension


**BLACK SETTINGS** = Lock-set tabs to 30° flexion and 30° extension

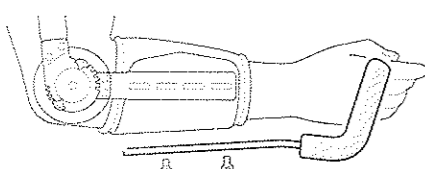**HAND PIECE ADJUSTMENT OR REMOVAL**  
 To adjust length of hand piece, remove screws and move the bar to the next set of corresponding holes, then replace the screws. Remove hand piece by removing screws.

**HAND WASH/DRIP DRY**

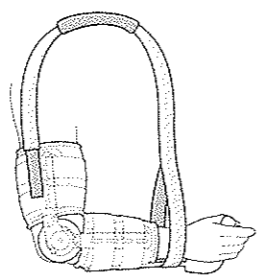**SHOULDER STRAP**  
 To use optional shoulder strap, attach the hook/loop material to upper arm cuff as shown, then slide the hand through the loop end.

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