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Telephone (800) 321-0607
(760) 599-3000
Telefax (760) 598-6193

AW 1.04010 REV D

CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.

CAUTION: FOR SINGLE PATIENT USE ONLY.

CAUTION: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.

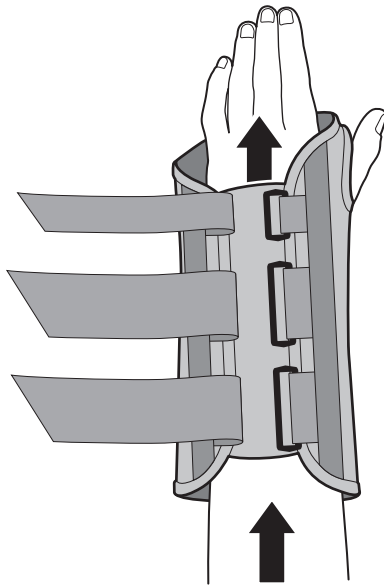
WARNING: CAREFULLY READ USE INSTRUCTIONS AND WARNINGS PRIOR TO OPERATION.

WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM


WRIST BRACE

APPLICATION INSTRUCTIONS

- 1 Slide brace on affected hand. The two layers of stockinette should be on the back of the hand. Do not insert hand between the stockinette layers.

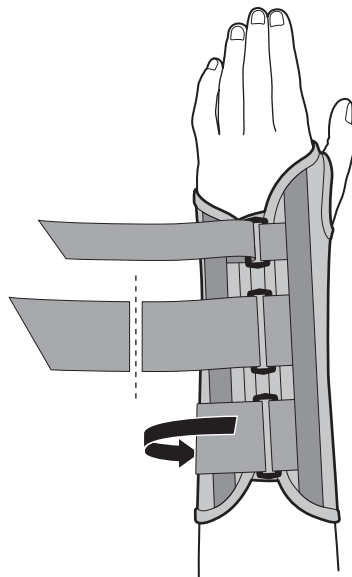- 2 Thread hook and loop through D-rings securing the hook and loop closures. The metal stay should fit into the palm of the hand and extend down the forearm.

- 3 Trim the straps leaving approximately 1" - 2" of overlap.

WASHING INSTRUCTIONS

Remove metal stay. Fasten hook and loop. Hand wash in warm water using a mild detergent. Air dry. Blot excess water with a towel, do not wring out.

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