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APPLICATION DE L'ORTHESE PIN CAM

APLICACION DEL PIN CAM

EC responsible person
 93/42/EEC Annex 1 Essential Requirements
 MDSS
 Hauptstrasse 39
 D-30974 Wennigsen, Germany

BREG Inc.
 Vista, CA 92081 U.S.A.

Telephone (800) 321-0607
 (760) 599-3000
 Telefax (760) 598-6193

PN-1.07860 REV B

 **WARNING: CAREFULLY READ USE/CARE INSTRUCTIONS AND WARNINGS PRIOR TO USE.**
WARNING: DO NOT REMOVE WALKER BOOT UNLESS INSTRUCTED BY YOUR MEDICAL TREATMENT PROFESSIONAL.
 **WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM.**
WARNING: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.
WARNING: TAKE APPROPRIATE PRECAUTIONS WHEN WALKING ON SLIPPERY SURFACES, UNEVEN TERRAIN, OR ANY OTHER PROPERTY CONDITION WHERE THE EXERCISE OF CAUTION APPEARS NECESSARY.
WARNING: DO NOT OPERATE A MOTOR VEHICLE OR HUMAN POWERED VEHICLE WHILE WEARING THIS WALKER BOOT.
CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.
CAUTION: FOR SINGLE PATIENT USE ONLY. OUTER SOLE MAY DETERIORATE WITH MULTIPLE USES. 90 DAY PRODUCT WARRANTY.
CAUTION: DEVICE CONTAINS LATEX—NON SKIN CONTACTING COMPONENTS.

Pin Cam Application

- Place foot and leg inside soft liner with heel firmly back; wearing a sock is recommended.
- Pads may be inserted to accommodate extra space at either side of ankle or at the heel. The larger tongue pad should be positioned over the top of the foot before securing the liner.
- Close the liner snugly securing the Velcro® closures.
- Position the heel of the foot against the back of the rocker boot. Brace uprights should be positioned midline over the ankle.
- Press the uprights securely against the liner. If there is a slight overlap, position the upright under the overlap. (This will allow later removal of the brace without changing the position of the upright.)
- Position the tibial straps in alternating fashion and secure all straps.
- Determine the required angle of dorsiflexion and/or plantar flexion and adjust according to the instructions below. Note: Adjustments should be managed by the clinician directly involved with the patient.
- Uprights may be contoured to the individual patient if required. The edge of a table or chair may be used to aid in contouring.

FIXED ANKLE

The Pin Cam Walker is shipped from the factory in the neutral position (0°). If a fixed position other than neutral is desired, observe the following:

- Remove the lower cap screw and rotate the cover 180°, exposing the upper cap screw. Loosen but do not remove the upper cap screw.
- Rotate the cover to allow removal of the two pin stops. The pins may be easily pushed out from behind.
- Note the angular markings of the disc part of the ankle joint and replace the pins in the appropriate holes, e.g. 15-15 for 15° plantar flexion.
- Retighten the upper cap screw.
- Rotate the plastic cover until the opening faces down. Replace the lower cap screw.

FREE OR LIMITED MOTION

- Remove the upper hexsocket screw to rotate the hinge cover.
- Rotate the cover to allow removal of the two pin stops. The pins may be easily pushed out from behind.
- Select the appropriate holes for limited motion stops and replace the pins. For free motion, the pins are not replaced.
- Do not retighten the upper cap screw for either free or limited motion settings.
- Rotate the plastic cover until the opening faces up. Replace the hexsocket screw.

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