VENTURE WALKER Short and Tall

Fitting, Use and Care Instructions


Anleitung zu Anpassung, Gebrauch und Pflege
Istruzioni per l'adattamento, l'uso e la manutenzione
Mode d'emploi, mise en place et entretien
Instrucciones de ajuste, uso y cuidado

BREG Inc.
2611 Commerce Way
Vista, CA 92083 U.S.A.
Telephone (800) 321-0607
(760) 599-3000
Telefax (760) 598-6193
PN-1.81040 REV D 8/02

E/Ü authorized representative
MDSS
Burckhardtstrasse 1
D-30163 Hannover
Germany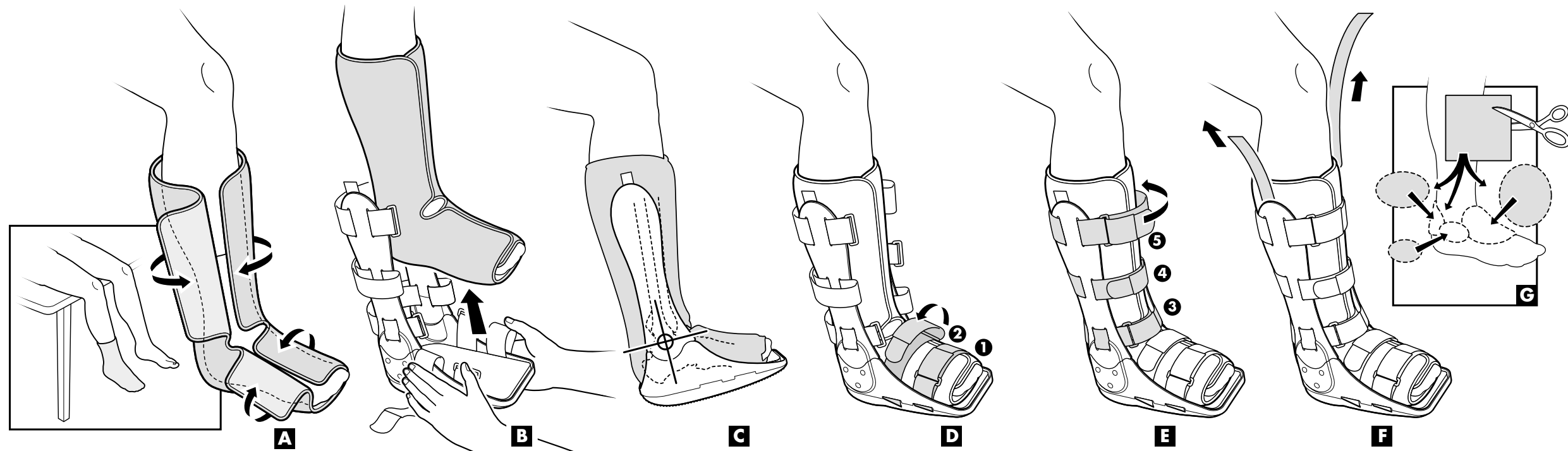

WARNING

WARNING: CAREFULLY READ USE/CARE INSTRUCTIONS AND WARNINGS PRIOR TO USE.

WARNING: DO NOT REMOVE WALKER BOOT UNLESS INSTRUCTED BY YOUR MEDICAL TREATMENT PROFESSIONAL.

WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM.

WARNING: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.

WARNING: TAKE APPROPRIATE PRECAUTIONS WHEN WALKING ON SLIPPERY SURFACES, UNEVEN TERRAIN, OR ANY OTHER PROPERTY CONDITION WHERE THE EXERCISE OF CAUTION APPEARS NECESSARY.

WARNING: DO NOT OPERATE A MOTOR VEHICLE OR HUMAN POWERED VEHICLE WHILE WEARING THIS WALKER BOOT.

CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.

CAUTION: FOR SINGLE PATIENT USE ONLY. OUTER SOLE MAY DETERIORATE WITH MULTIPLE USES. 90 DAY PRODUCT WARRANTY.

CAUTION: DEVICE CONTAINS LATEX — NON SKIN CONTACTING COMPONENTS.

INITIAL APPLICATION BY A MEDICAL PROFESSIONAL ONLY!

A REMOVE FOAM LINER AND APPLY TO LEG

Patient seated with legs bent, feet above floor, close liner snugly. Wearing sock is recommended.

B SLIDE WALKER ONTO LEG FROM BELOW

C POSITION WALKER WITH UPRIGHTS CENTERED OVER THE MALLEOLI (ANKLE BONE)

Do not remove cardboard strips yet!

D FASTEN STRAPS

Start with straps 1 and 2 closest to toes.

E Fasten remaining straps 3 through 4.

F REMOVE CARDBOARD STRIPS LOCATED INSIDE UPRIGHTS

G OPTIONAL PAD SET (Accessory Order Only) LOCATIONS

Optional pads may be positioned to accommodate excess space on either side of ankle or to provide padding over the foot top or around outside of Heel/Achilles area.

USE AND CARE OF YOUR VENTURE:

- After initial application, the VENTURE may be removed and reapplied without removing foam liner from boot.
- The foam liner may be trimmed with scissors as needed.
- Hand wash the foam liner and straps with mild soap, air dry. Do not place liner or straps into a mechanical dryer.
- Extra foam liners, with strap sets available from Customer Service: (800) 321-0607 or (760) 599-3000.

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