# VENTURE WALKER 61 & 61s

## Fitting, Use and Care Instructions


Anleitung zu Anpassung, Gebrauch und Pflege  
Istruzioni per l'adattamento, l'uso e la manutenzione  
Mode d'emploi, mise en place et entretien  
Instrucciones de ajuste, uso y cuidado

**BREG Inc.**  
2611 Commerce Way  
Vista, CA 92083 U.S.A.  
Telephone (800) 321-0607  
(760) 599-3000  
Telefax (760) 598-6193  
PN-1.81210 REV A 8/02

E/U authorized representative  
MDSS  
Burckhardtstrasse 1  
D-30163 Hannover  
Germany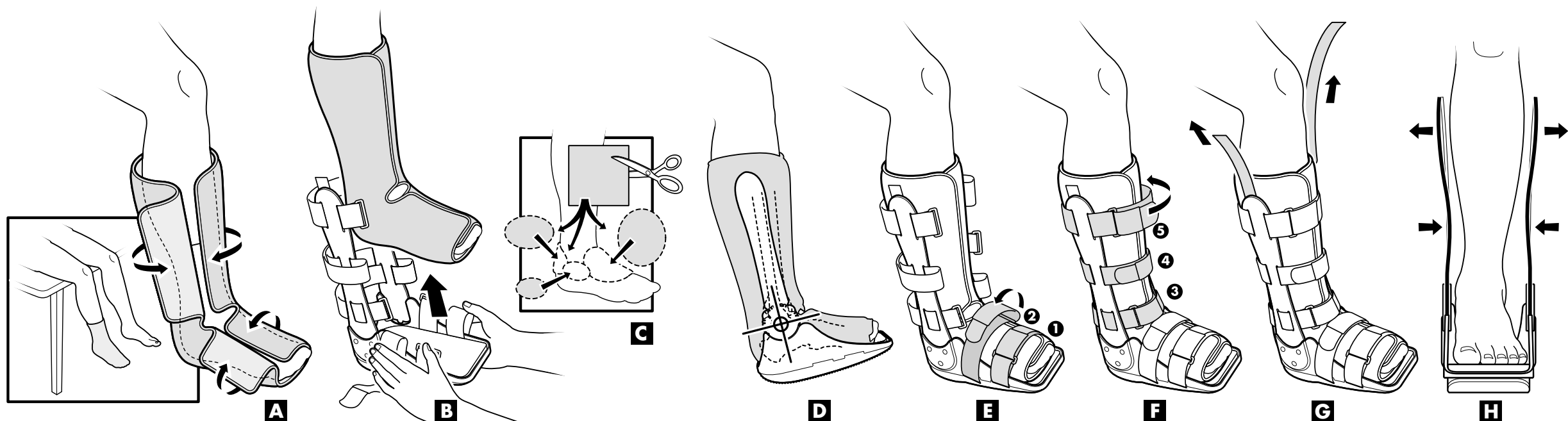

**! WARNING**

**WARNING:** CAREFULLY READ USE/CARE INSTRUCTIONS AND WARNINGS PRIOR TO USE.

**WARNING:** DO NOT REMOVE WALKER BOOT UNLESS INSTRUCTED BY YOUR MEDICAL TREATMENT PROFESSIONAL. DO NOT CONTOUR SIDE ALUMINIUM STRUTS—ONLY TREATING MEDICAL PROFESSIONAL MAY CONTOUR STRUTS.

**WARNING:** THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM.

**WARNING:** IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.

**WARNING:** TAKE APPROPRIATE PRECAUTIONS WHEN WALKING ON SLIPPERY SURFACES, UNEVEN TERRAIN, OR ANY OTHER PROPERTY CONDITION WHERE THE EXERCISE OF CAUTION APPEARS NECESSARY.

**WARNING:** DO NOT OPERATE A MOTOR VEHICLE OR HUMAN POWERED VEHICLE WHILE WEARING THIS WALKER BOOT.

**CAUTION:** FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.

**CAUTION:** FOR SINGLE PATIENT USE ONLY. OUTER SOLE MAY DETERIORATE WITH MULTIPLE USES. 90 DAY PRODUCT WARRANTY.

**CAUTION:** DEVICE CONTAINS LATEX — NON SKIN CONTACTING COMPONENTS.

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### INITIAL APPLICATION BY A MEDICAL PROFESSIONAL ONLY!

- A REMOVE FOAM LINER AND APPLY TO LEG**  
Patient seated with legs bent, feet above floor, close liner snugly. Wearing sock is recommended.
- B SLIDE WALKER ONTO LEG FROM BELOW**
- C OPTIONAL PAD SET LOCATIONS**  
Optional pads may be positioned to accommodate excess space on either side of ankle or to provide padding over the foot top or around outside of heel/Achilles area.
- D POSITION WALKER WITH UPRIGHTS CENTERED OVER THE MALLEOLI (ANKLE BONE)**  
Do not remove cardboard strips yet.
- E FASTEN STRAPS**  
Start with straps 1 and 2 closest to toes.
- F Fasten remaining straps 3 through 4.**
- G REMOVE CARDBOARD STRIPS LOCATED INSIDE UPRIGHTS**
- H CONTOURING UPRIGHTS**  
If necessary, aluminum uprights may be contoured to fit each individual patient. Contouring is to be performed by a medical professional only. Bending irons or edge of a solid bench or table may be used to contour uprights.

### USE AND CARE OF YOUR VENTURE:

- After initial application, the VENTURE may be removed and reapplied without removing foam liner from boot.
- The foam liner may be trimmed with scissors as needed.
- Hand wash the foam liner and straps with mild soap, air dry. Do not place liner or straps into a mechanical dryer.
- Extra foam liners with strap sets available from Customer Service: (800) 321-0607 or (760) 599-3000

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