VENTURE AIR WALKER

VENTURE AIR WALKER

STIVALETTO AD ARIA "AIR WALKER" VENTURE

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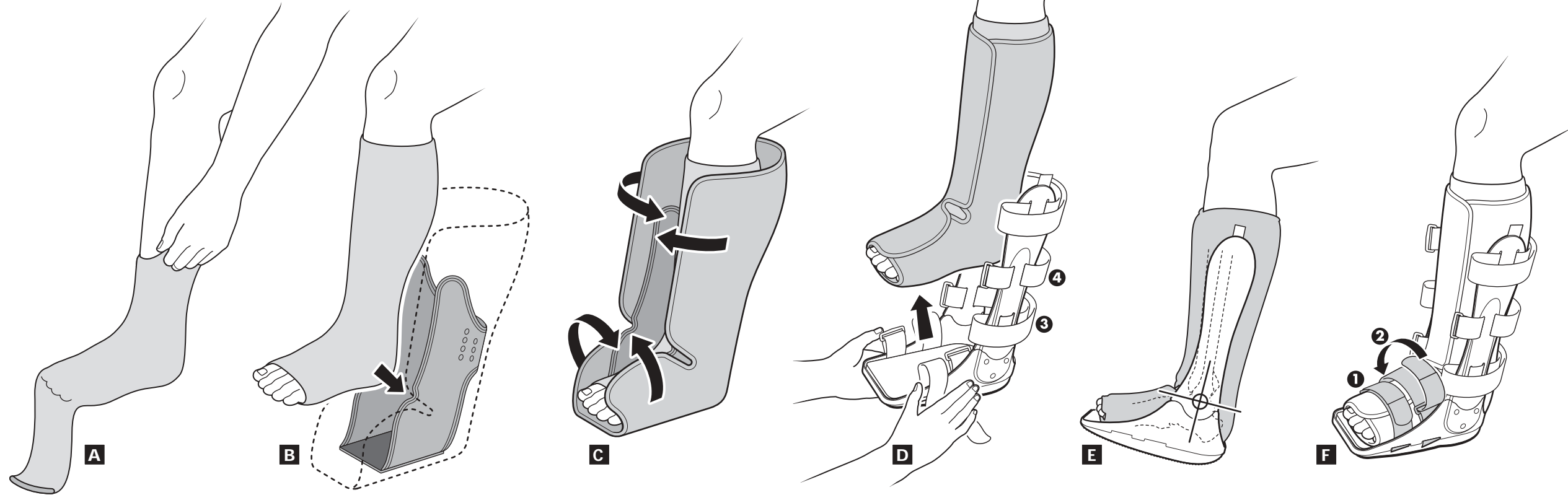
BOTA VENTURE AIR WALKER

E/U authorized representative
MDSS
Burckhardtstrasse 1
D-30163 Hanover
Germany

BREG Inc.
Vista, CA 92081 U.S.A.
Telephone (800) 321-0607
(760) 599-3000
Telefax (760) 598-6193
PN-1.81381 REV A

! W A R N I N G

WARNING: CAREFULLY READ WARNINGS AND USE/CARE INSTRUCTIONS PRIOR TO USE.

WARNING: INITIAL APPLICATION TO BE PERFORMED BY A LICENSED HEALTH CARE PRACTITIONER ONLY. PATIENT REMOVAL AND REAPPLICATION SHOULD BE PURSUANT TO SPECIFIC ORDER AND INSTRUCTION OF THE LICENSED HEALTH CARE PRACTITIONER.

WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM.

WARNING: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR LICENSED HEALTH CARE PRACTITIONER.

WARNING: TAKE APPROPRIATE PRECAUTIONS WHEN WALKING ON SLIPPERY SURFACES, UNEVEN TERRAIN, OR ANY OTHER PROPERTY CONDITION WHERE THE EXERCISE OF CAUTION APPEARS NECESSARY.

WARNING: DO NOT OPERATE A MOTOR VEHICLE OR HUMAN POWERED VEHICLE WHILE WEARING THIS WALKER BOOT.

CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.

CAUTION: FOR SINGLE PATIENT USE ONLY. OUTER SOLE MAY DETERIORATE WITH MULTIPLE USES. 90 DAY PRODUCT WARRANTY.

CAUTION: DEVICE CONTAINS LATEX — NON SKIN CONTACTING COMPONENTS.

CAUTION: ALWAYS WEAR COTTON COMPRESSION SLEEVE OR OTHER BARRIER OVER THE SKIN. DO NOT ALLOW DIRECT CONTACT BETWEEN THE SKIN AND THE INNER AIR BLADDER.

CAUTION: IF AIR BLADDER DOES NOT HOLD PRESSURE OR LEAKS, CONSULT YOUR LICENSED HEALTH CARE PRACTITIONER IMMEDIATELY.

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Initial application by a LICENSED HEALTH CARE PRACTITIONER only!

- A** APPLY COTTON COMPRESSION SLEEVE
- B** APPLY FOAM LINER AND INNER AIR BLADDER TO LEG
Patient seated with legs bent, feet above floor. Place the patient's foot into the foam liner so that the heel is positioned in the opening at the back of the air bladder.
- C** CLOSE BOTH SIDES OF THE FOAM LINER SNUGLY
- D** SLIDE WALKER ONTO LEG FROM BELOW
Note that the air fill valve is accessible at the back of the Air Walker, between straps ③ & ④.
- E** POSITION WALKER WITH UPRIGHTS CENTERED OVER THE MALLEOLI (ANKLE BONE)
Do not remove cardboard strips yet!
- F** FASTEN STRAPS
Start with straps ① and ② closest to toes.

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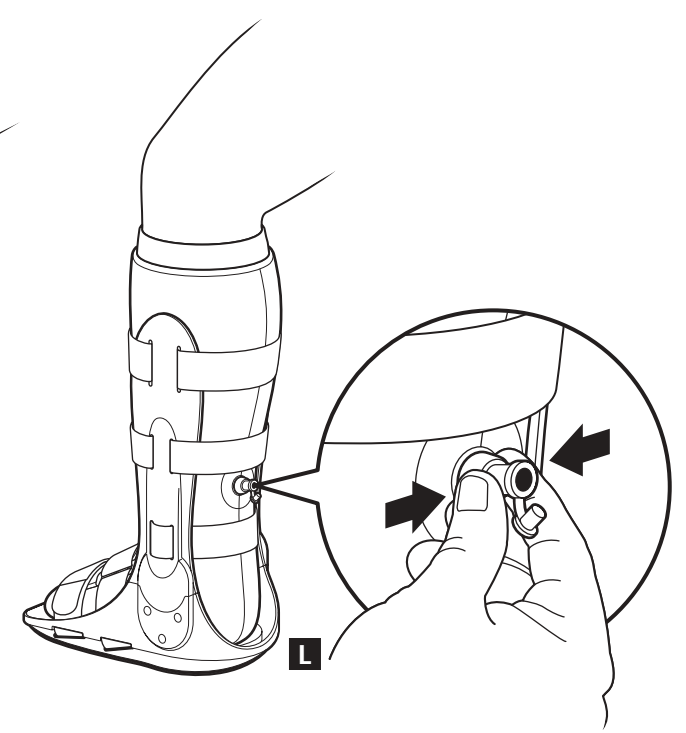
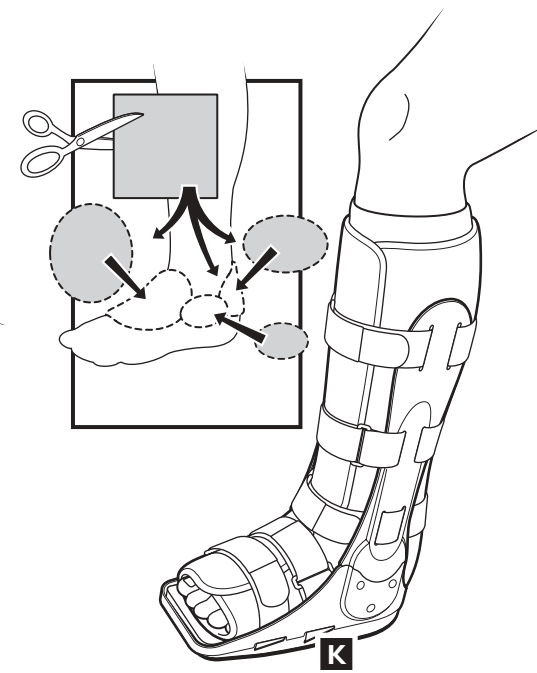
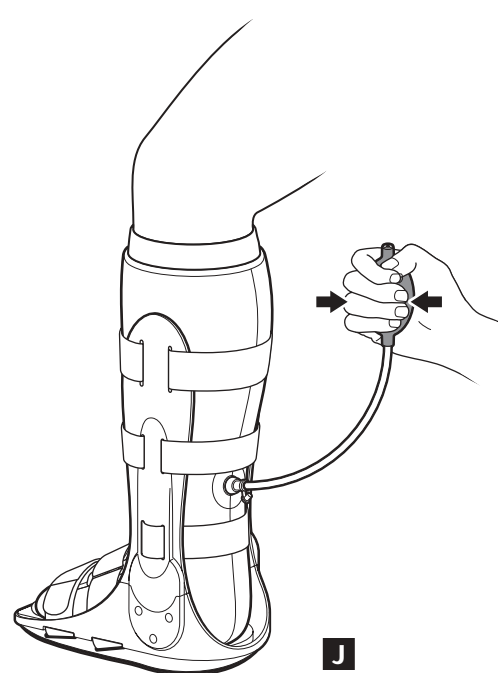
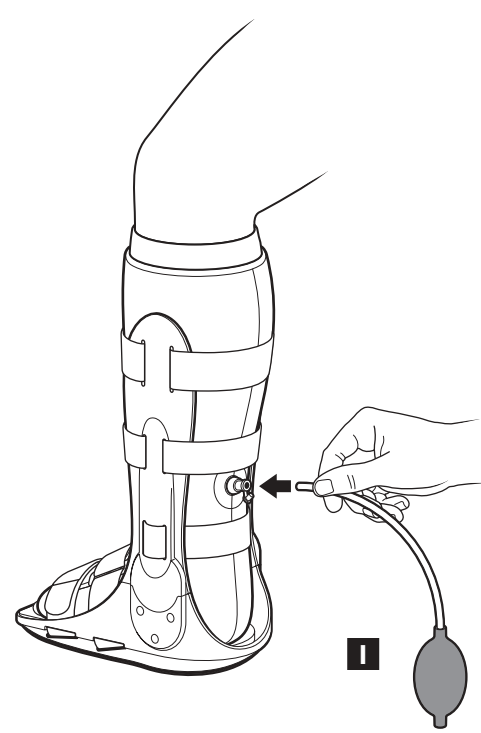
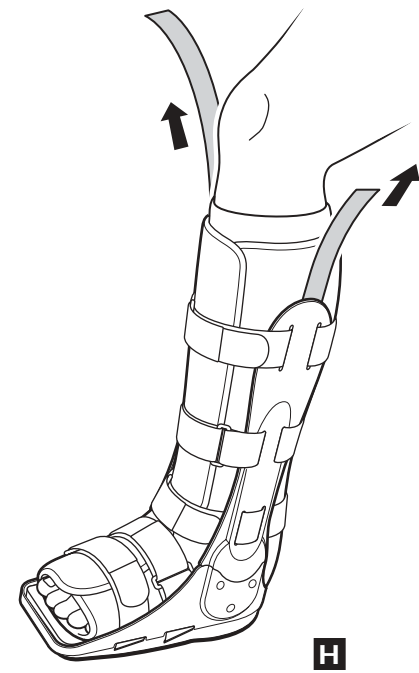
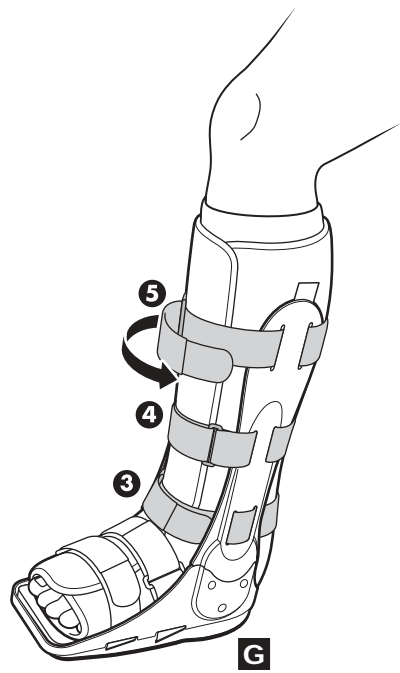
G FASTEN REMAINING STRAPS
Straps **3**, **4** and **5**.

H REMOVE CARDBOARD STRIPS LOCATED INSIDE UPRIGHTS

I FILL THE INNER AIR BLADDER WITH AIR

Access the air fill valve at the back of the foam liner between straps 3 & 4. Insert the plastic tube of the inflation device into the air fill valve.

J Pump the hand bulb on the inflation device approximately 5 times to attain proper fit, comfort and compression. Your Licensed Health Care Practitioner should instruct on how to attain proper fit, comfort and compression during reapplication. Cap the air fill valve with the attached plug.

K OPTIONAL PAD SET LOCATIONS (ACCESSORY ORDER ONLY)

Optional pads may be positioned to accommodate excess space on either side of ankle or to provide padding over the foot top or around outside of Heel/Achilles area.

L REMOVAL AND REAPPLICATION

Upon specific order and instruction of the Licensed Health Care Practitioner, the Walker Boot may be removed and reapplied without removing the foam liner or deflated air bladder from the boot. The air bladder is deflated by squeezing the air fill valve. It is not necessary to remove all the air in the bladder.

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