# VECTRA Lite Fixed Walker VECTRA Premium Fixed Walker

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AW-1.90271 REV A

**W A R N I N G**

**WARNING:** CAREFULLY READ USE/CARE INSTRUCTIONS AND WARNINGS PRIOR TO USE.  
**WARNING:** DO NOT REMOVE WALKER BOOT UNLESS INSTRUCTED BY YOUR MEDICAL TREATMENT PROFESSIONAL.  
**WARNING:** THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM.  
**WARNING:** IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.  
**WARNING:** TAKE APPROPRIATE PRECAUTIONS WHEN WALKING ON SLIPPERY SURFACES, UNEVEN TERRAIN, OR ANY OTHER PROPERTY CONDITION WHERE THE EXERCISE OF CAUTION APPEARS NECESSARY.  
**WARNING:** DO NOT OPERATE A MOTOR VEHICLE OR HUMAN POWERED VEHICLE WHILE WEARING THIS WALKER BOOT.  
**CAUTION:** FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.  
**CAUTION:** FOR SINGLE PATIENT USE ONLY. OUTER SOLE MAY DETERIORATE WITH MULTIPLE USES. 90 DAY PRODUCT WARRANTY.

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## Vectra Lite Fixed Walker

- Apply liner to leg.** With patient seated, leg(s) bent and feet off floor, position tongue pad as needed for maximum comfort, close liner snugly.
- Slide walker onto leg over the liner straight up directly under foot.**
- Fasten straps.** Start with straps 1 and 2 closest to the toes (Straps may be trimmed to desired length).
- Position walker with uprights centered over the malleoli (ankle bones).**
- Remove plastic bags from the uprights and discard.** Attach uprights to the liner. Press firmly.
- Fasten straps.** Affix straps 3 through 5 to the outside of the uprights by alternating the buckle placement. Then fasten straps 3 through 5.

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
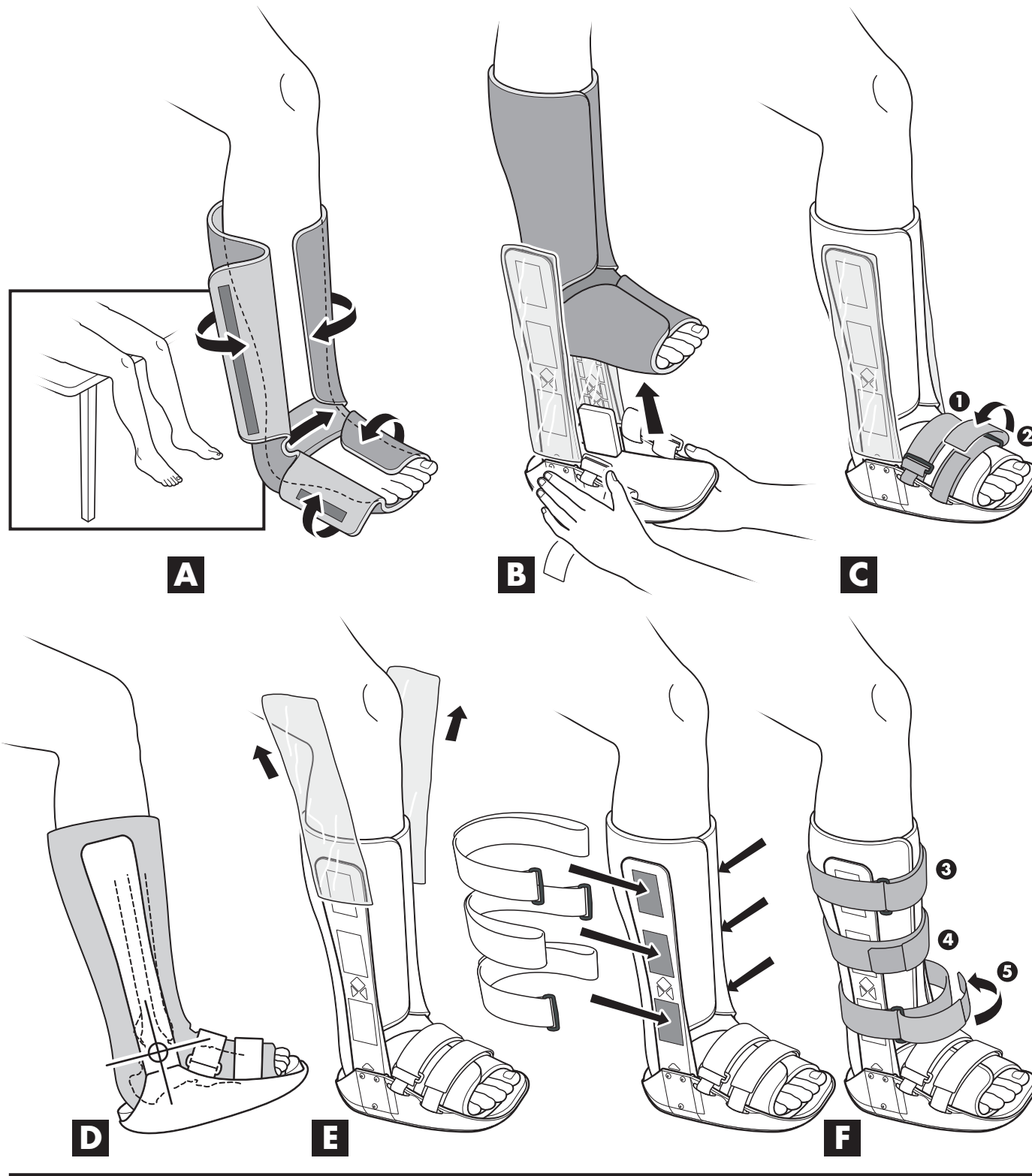
**Optional:** For maximum liner fixation and patient comfort, attach the 1 x 1/2" hook pieces to the liner at malleoli foam pads (or on the foot bed of the walker).

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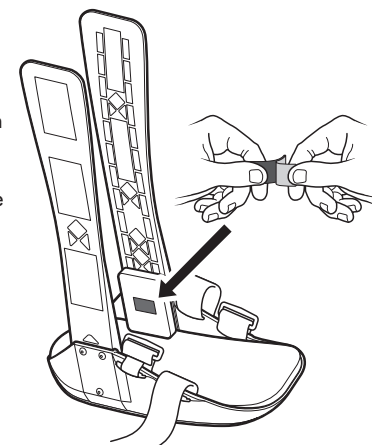## Vectra Premium Fixed Walker

- A Apply liner to leg.** With patient seated, leg(s) bent and feet off floor, position tongue pad as needed for maximum comfort, close liner snugly.
- B Slide walker onto leg over the liner straight up directly under foot.**
- C Fasten straps.** Start with straps 1 and 2 closest to the toes (Straps may be trimmed to desired length by removing tab at end of straps, trim, and reattach tab).
- D Position walker with uprights centered over the malleoli (ankle bones).**
- E Remove cardstock quick-pulls from the uprights and discard.** Attach uprights to the liner. Press firmly.
- F Fasten straps.** Fasten remaining straps 3 through 5.

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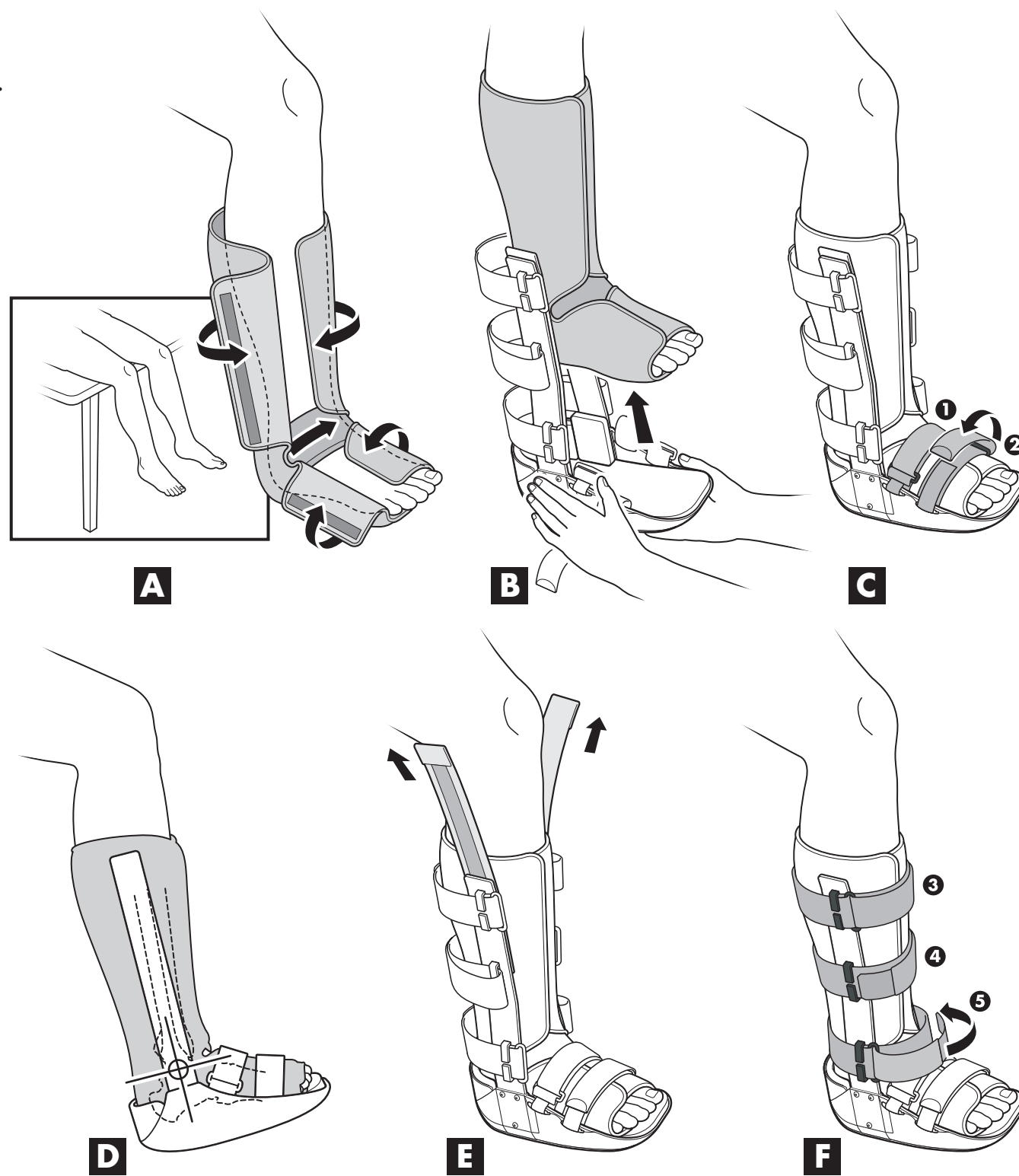
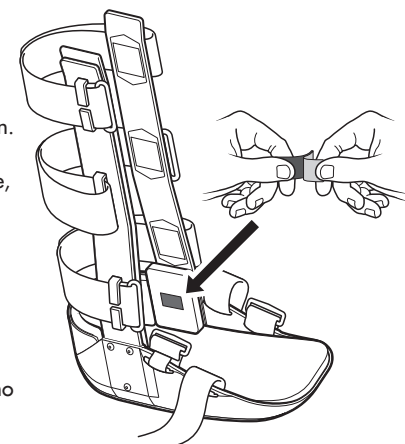
**Optional heel clip removal** Pull up on the heel clip.

**Optional heel clip replacement** Lift up foot bed padding and foot plate. Slide heel clip into place using the dove-tail slot guides. Release the foot plate and padding and push down into proper position.

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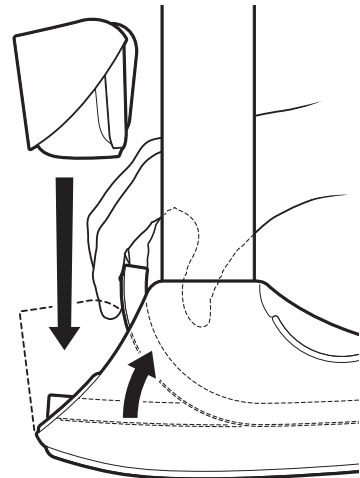
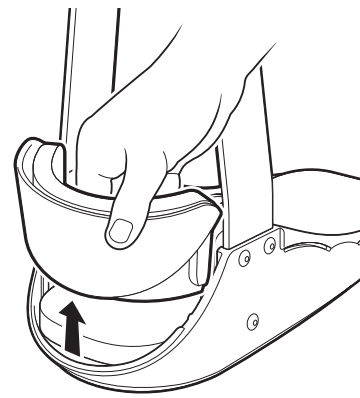
## USE AND CARE OF YOUR VECTRA:

- After initial application, the VECTRA may be removed and reapplied without removing liner from boot.
- Hand wash the liner and straps with mild soap, air dry. Do not place liner or straps into a mechanical dryer.
- Extra liners with strap sets are available from Customer Relations: (800) 321-0607 or (442) 599-3000.

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