FUSION™ XT OA

ADJUSTABLE HINGE INSTRUCTIONS

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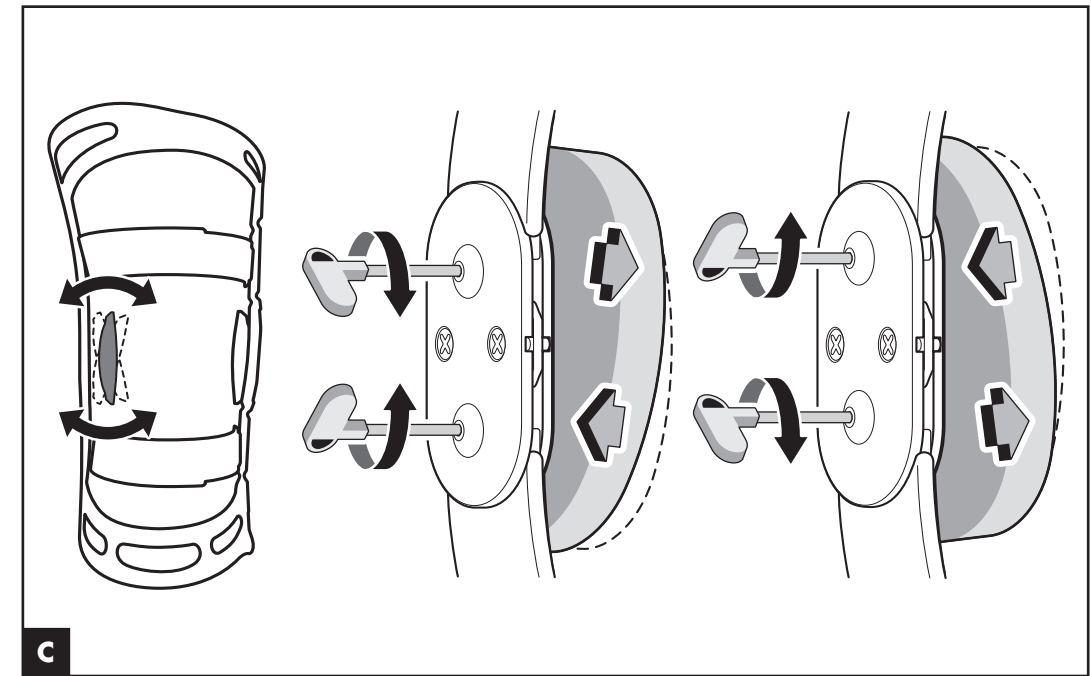
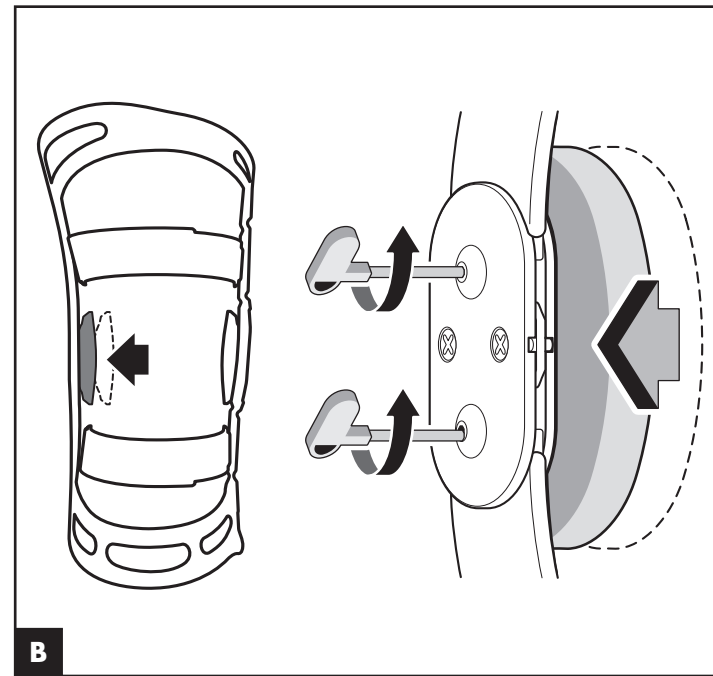
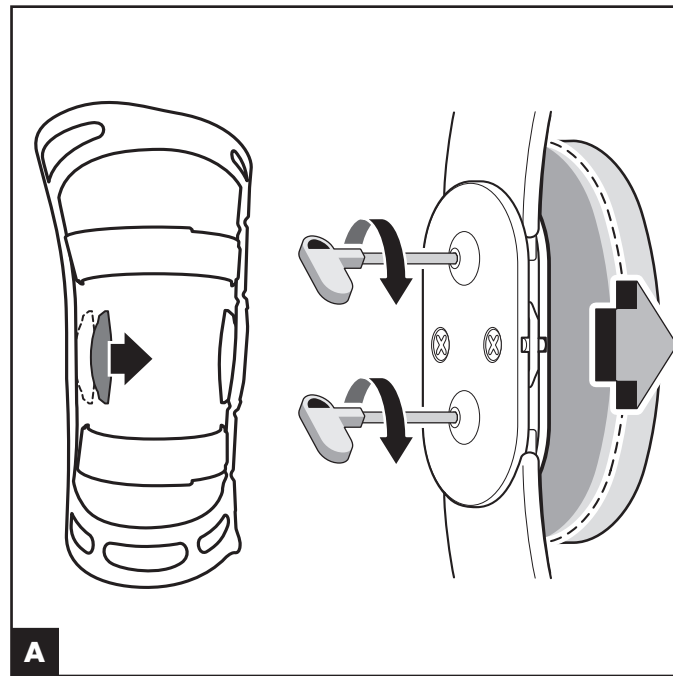
WARNINGS

- CAREFULLY READ FITTING INSTRUCTIONS AND WARNINGS PRIOR TO USE.
- FOR SINGLE PATIENT USE ONLY.
- IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.
- THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO ESSENTIAL PARTS OF A SAFE TREATMENT PROGRAM. CONSULT YOUR LICENSED HEALTH CARE PROFESSIONAL SHOULD YOU HAVE ANY QUESTIONS REGARDING THE SAFE APPLICATION OF THIS PRODUCT RELATED TO YOUR MEDICAL HISTORY.

CAUTION: WHEN USING BRACES BILATERALLY, MODIFY ACTIVITY AS NECESSARY TO KEEP MEDIAL HINGES FROM CONTACTING EACH OTHER.

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Adjustable Hinge Instructions

ADJUSTMENT FOR LOOSE FITTING PADS

Use the tool provided to turn the upper and lower adjustment screws on the lateral hinge clockwise. This will move the condyle pad inward, tighter against knee. Adjust for a comfortable, consistent fit.

ADJUSTMENT FOR TIGHT FITTING PADS

Turn the upper and lower adjustment screws on the lateral hinge counterclockwise. This will move the condyle pad outward, away from the knee.

ADJUSTMENT IF TOO LOOSE OR TIGHT AT UPPER OR LOWER EDGE OF PADS

The condyle pad may be tilted to fit the contour of the knee. To tilt the condyle pad, adjust either the top or bottom screws. To do so, turn the screws counterclockwise to move the condyle pad edge outward, away from the knee. Turn the screws clockwise to move the condyle pad edge inward, tighter against knee.

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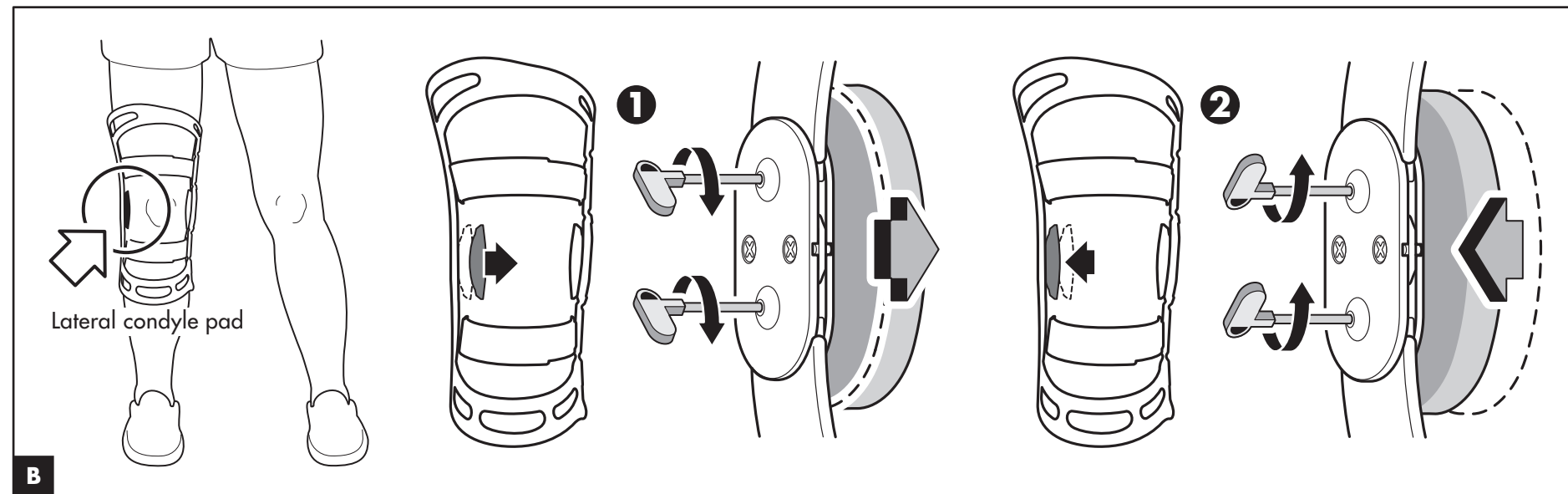
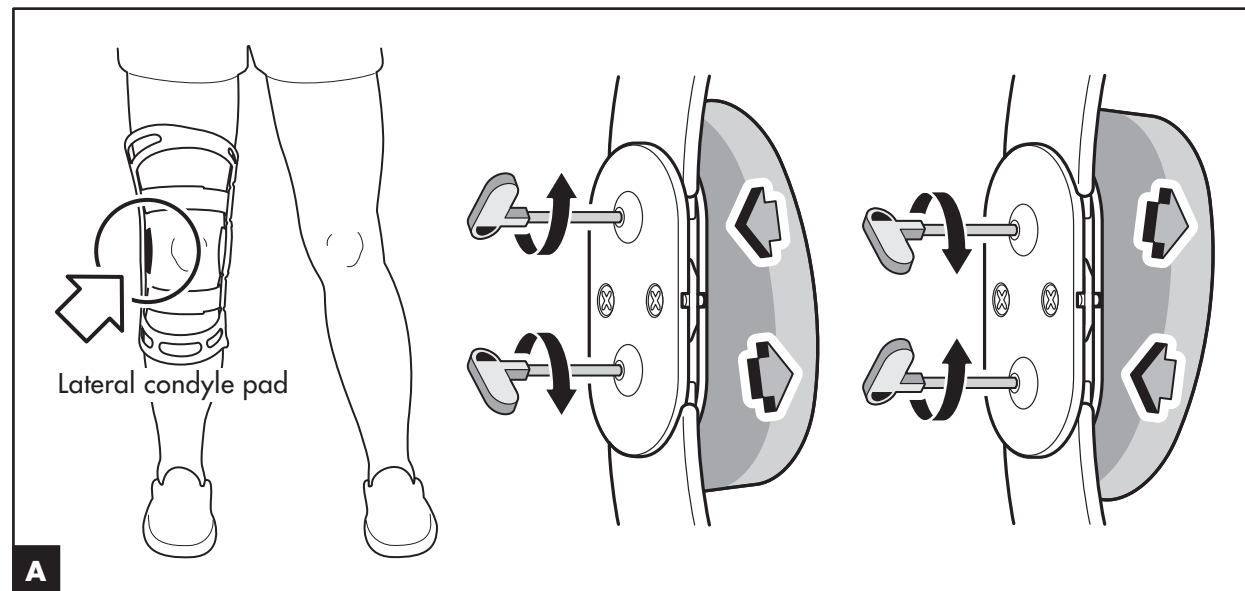
Fusion XT OA Unicompartmental Osteoarthritis Instructions

A ADJUST THE LATERAL (OUTSIDE) CONDYLE PAD

With brace on, walk 5 minutes to position brace properly and ensure that lateral condyle pad is applying lateral force to knee. If more lateral force is required to relieve symptoms turn upper and lower adjustment screws on side of hinge clockwise. This will move condyle pad inward (toward knee), applying more lateral force to knee.

- Upper and lower adjustment screws can be adjusted differently to accommodate various knee contours.
- For more comfort, different adjustments of upper and lower screws will distribute lateral force more evenly.
- Make sure large gel condyle pad is attached to the lateral (outside) condyle basket.

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Fitting Instructions for loss or gain in leg girth:

B ADJUSTING THE LATERAL CONDYLE PAD INWARD OR OUTWARD

If there is a loss or gain in leg girth (swelling) the pressure against the knee from the condyle pads may be too great or insufficient. The lateral condyle pad may be adjusted to accommodate a change in leg girth.

- 1 ADJUSTMENT FOR LOSS IN LEG GIRTH:**
Use the tool provided to turn the upper and lower hinge adjustment screws clockwise. This will move the condyle pad inward, tighter against knee.
- 2 ADJUSTMENT FOR GAIN IN LEG GIRTH (SWELLING):**
Use the tool provided to turn the upper and lower hinge adjustment screws counterclockwise. This will move the condyle pad outward, away from knee.

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