EPO Knee Brace

EPO KNIESCHIENE
SUPPORTO EPO DEL GINOCCHIO
ORTHESE DE GENOU EPO
RODILLERA EPO

EPO Lite Knee Brace

LEICHTE EPO KNIESCHIENE
SUPPORTO LEGGERO EPO DEL GINOCCHIO
ORTHESE DE GENOU EPO LEGERE
RODILLERA LIGERA EPO

EPO QuickFit Knee Brace

EPO QUICKFIT-KNIESCHIENE
SUPPORTO DEL GINOCCHIO EPO QUICKFIT
ORTHESE EPO QUICKFIT
RODILLERA EPO QUICKFIT

CE

EC REP

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AW-1.90033 REV A

WARNING

CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.
CAUTION: FOR SINGLE PATIENT USE ONLY.
CAUTION: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.
WARNING: CAREFULLY READ USE INSTRUCTIONS AND WARNINGS PRIOR TO OPERATION.
WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM

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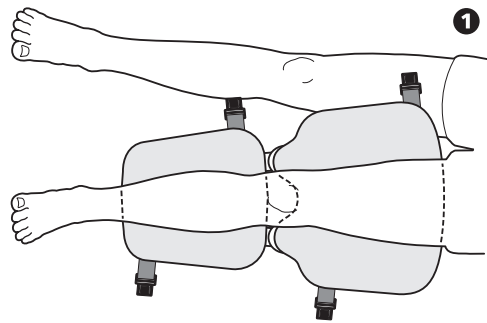
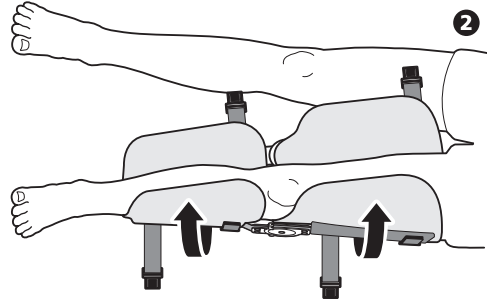
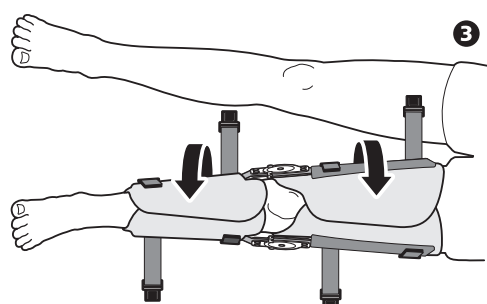
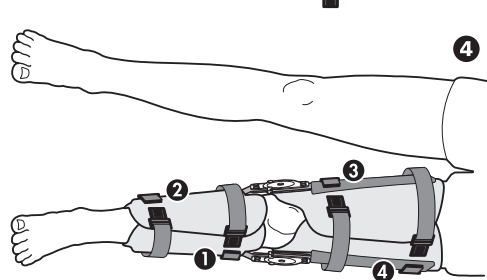
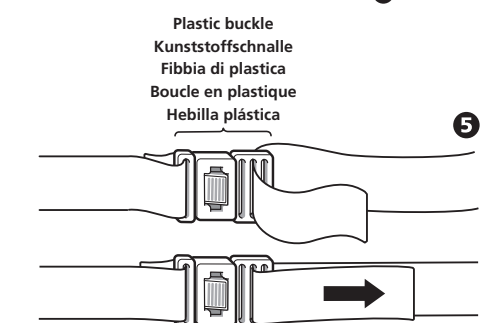
- 1 Open the EPO QuickFit brace and place it under the fully extended leg. The hinges should be at mid-patella level and medial/lateral level of the leg.
- 2 Fold foam wrap over from the lateral or outside of the leg.
- 3 Fold foam wrap over from the medial or inside of the leg so that the wrap fits snugly. Close the wrap at the front of the leg with the hoop and loop closure.
- 4 Buckle the straps, starting with the upper calf (1), the lower calf (2), the lower thigh (3), and last the upper thigh (4).
- 5 Tighten the straps by pulling the strap tight through the buckle and fastening back against itself by using the hook and loop closure.

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ROM HINGE ADJUSTMENTS:

To adjust the hinge, simply pull the stop out of its slot and move it to the appropriate slot that corresponds to the degree of flexion or extension desired. The slot identified as "LOCK" is used for locking the brace in full extension.

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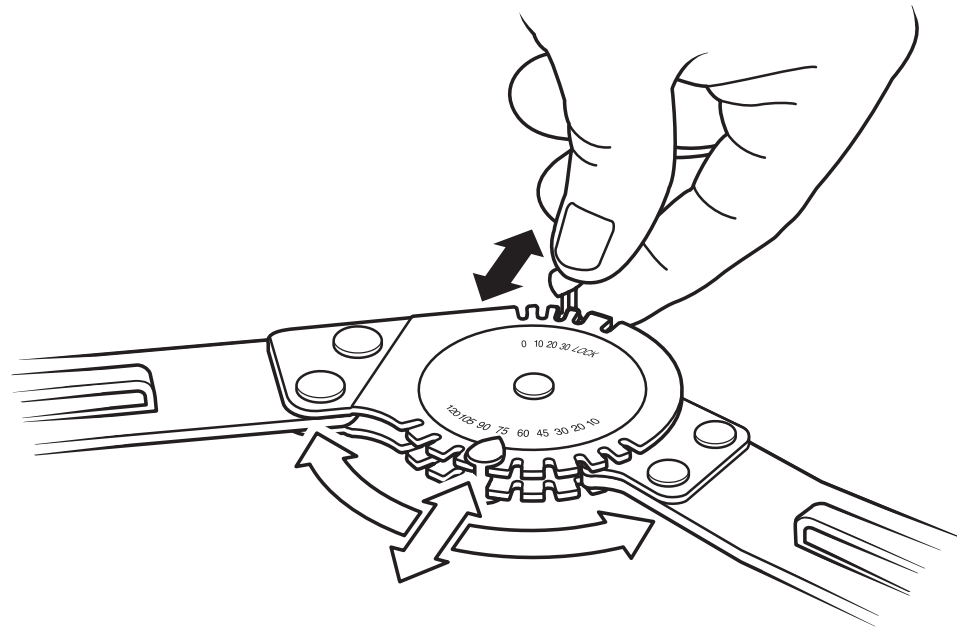
TIPS ON USE AND CARE OF YOUR POST-OP KNEE BRACE:

- Close the foam wraps in the front to allow easy removal of the brace later.
- The foam wraps may be trimmed with scissors as needed.
- The side bars may be bent as illustrated at RIGHT to accommodate individual legs. Bend in equal amounts above and below the hinge and on each side.
- Wash foam wraps and straps by hand only and air dry. Extra wraps are available from BREG.
- To tighten straps securely it is easiest to use two hands—one hand to pull the end of the strap after it passes through the ring and the other to hold the ring in place next to the side bar.

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