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
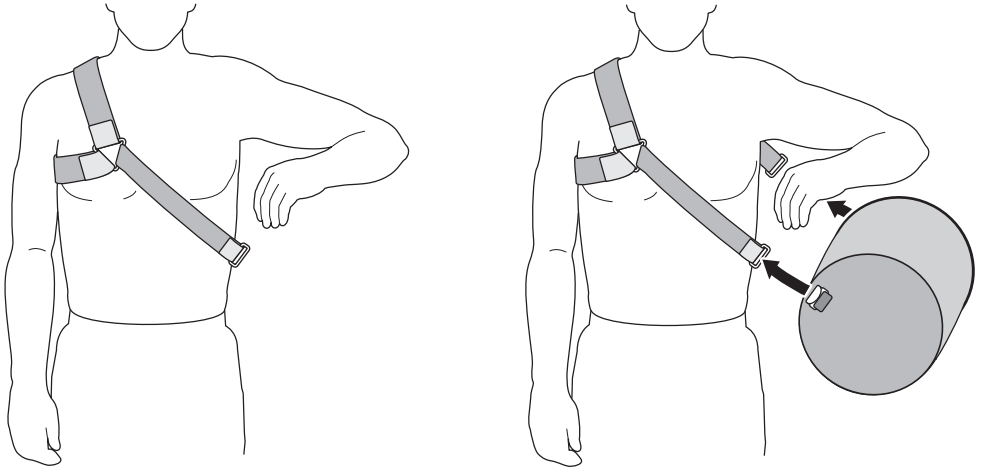
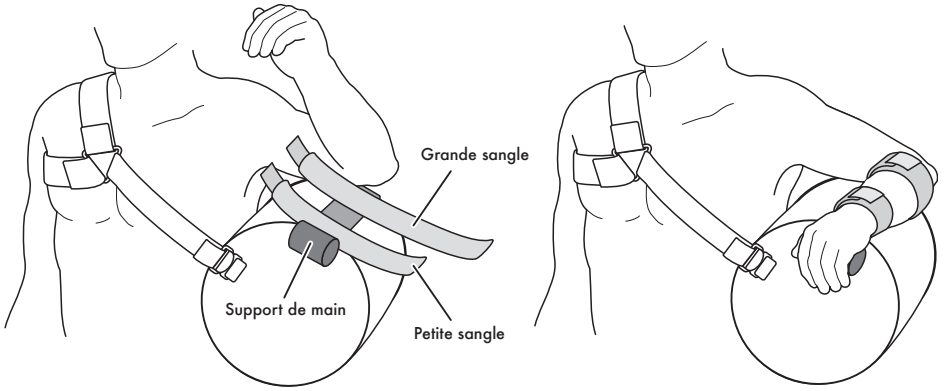
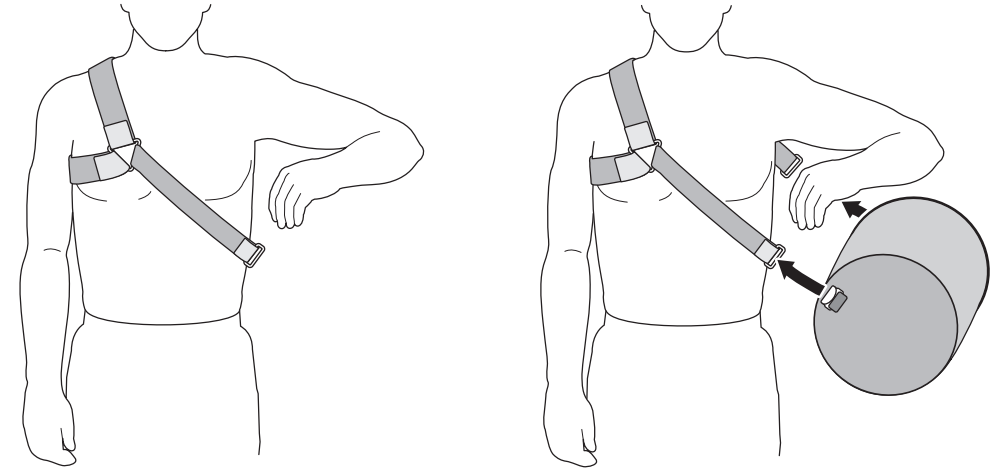
CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.
CAUTION: FOR SINGLE PATIENT USE ONLY.
CAUTION: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.
WARNING: CAREFULLY READ USE INSTRUCTIONS AND WARNINGS PRIOR TO OPERATION.
WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM

SHOULDER ABDUCTION PILLOW


The BREG Shoulder Abduction Pillow is universally sized and used on either a right or left involved shoulder. Begin by inflating the pillow, then follow illustrated steps 1 through 5.

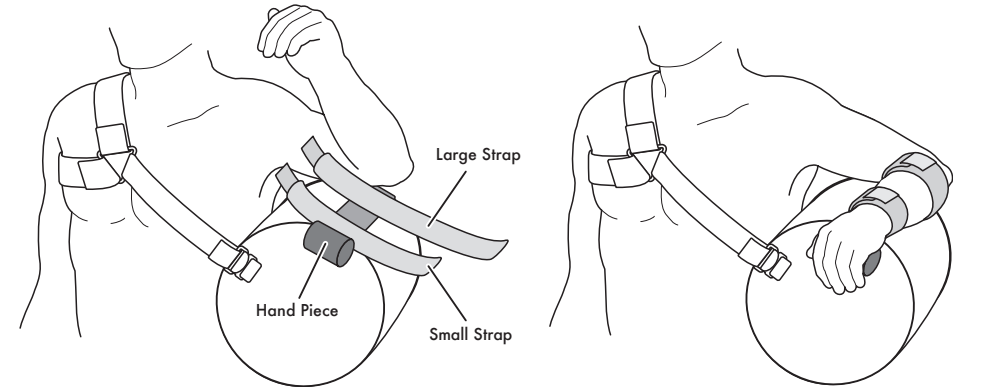1 Position the strap harness on the non-involved shoulder as shown.
Note: Make sure the velcro ends on the straps face outwards for easy adjustment.
2 Place the inflated pillow under the involved arm and attach to the strap harness.

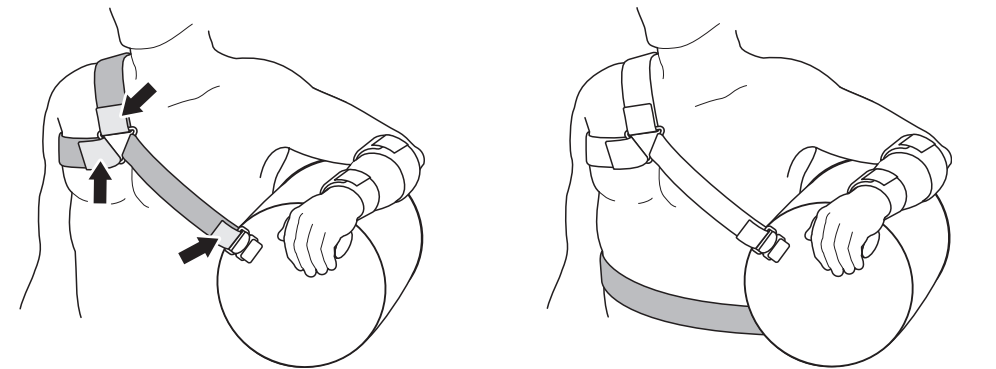3 After noting the desired arm position on the pillow, attach the hand piece and straps as shown.

4 Adjust the 4 straps on the strap harness as necessary for appropriate and comfortable fit.
5 A completed application with swath strap which may be used if desired. Note that the pillow may be deflated and strap lengths changed to achieve different arm positions.

CLEANING INSTRUCTIONS
 After removing the inflatable pillow, hand wash using mild soap and warm water or use "delicate" setting on an automatic washing machine. Air dry or use low heat in a clothes dryer.

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