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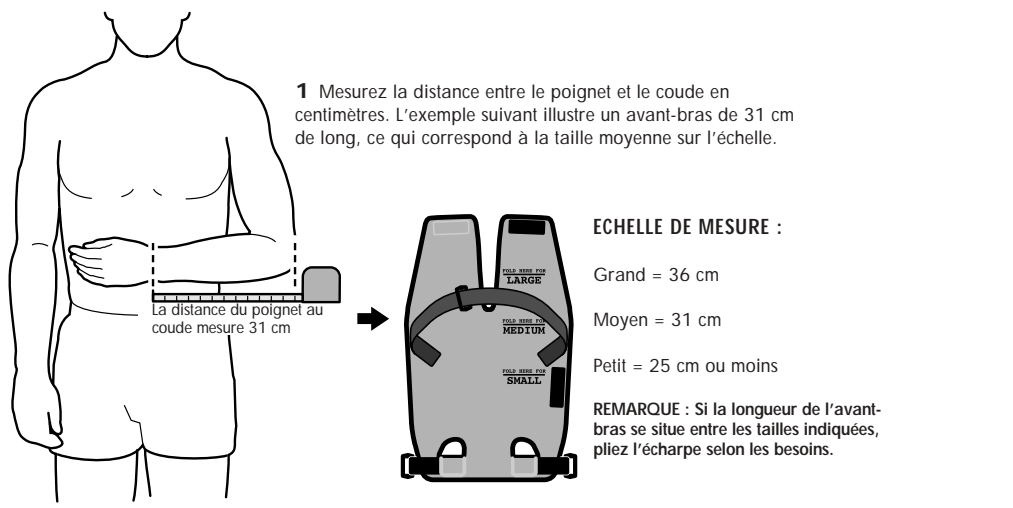
CE
 E/U authorized representative
 MDSS
 Burckhardtstrasse 1
 D-30163 Hanover
 Germany

BREG Inc.
 Vista, CA 92081 U.S.A.
 Telephone (800) 321-0607
 (760) 599-3000
 Telefax (760) 598-6193
 PN-1.07200 REV E

CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.
CAUTION: FOR SINGLE PATIENT USE ONLY.
CAUTION: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.
WARNING: CAREFULLY READ USE INSTRUCTIONS AND WARNINGS PRIOR TO OPERATION.
WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM

THE SLINGSHOT™

1 Measure the distance between the wrist and the elbow in inches. This example illustrates a forearm 12" long, which equals the Medium size on the scale.

Wrist to elbow measures 12"

SIZING SCALE:
 Large= 14"
 Medium= 12"
 Small= 10" or less

NOTE: If the forearm length falls between marked sizes, fold appropriately.

2 After obtaining the patient size, fold the sling as illustrated. Fold the panel edge down to meet the line for the indicated size. Next, fold the sling in half as shown. Last, fold over the second panel to close the end of the sling.

Medium 12"

3 Slip the sling onto the forearm with the patient's thumb through the thumb hole. Attach the shoulder strap to the sling, A. Pass the strap diagonally over the unaffected shoulder, B. Pass the end of the strap through the plastic loops, and adjust as necessary, C.

4 Place the pillow at the waistline. Attach the strap and adjust to fit. Attach the sling to the pillow, lining up the velcro on the pillow with the sling. The length of the waist strap can be adjusted at the clip on the back of the pillow.

5 The properly applied sling.

NOTE: The pillow is designed to keep the injured arm in 15° of abduction. The position of the pillow can be adjusted for internal or external rotation by sliding the pillow forward or backward along the waistline.

SHORTENING THE SHOULDER STRAP:

1 Fold shoulder strap to the desired length, fasten hook velcro strap end back onto original strap.

2 Attach 'alligator strap tab' to the folded end of shoulder strap.

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