

# BOA S.I. BELT

Before using this device, please read the following instructions completely and carefully. Correct application is vital to the proper functioning of the device.

**Indications:** Sacroiliac Pain and Dysfunction

## Application Information:

1. Position and fasten pull-handle as close as possible to the compression panels and fully extend the cord.
2. Referencing the attached label for correct brace orientation, position the brace approximately 1" above the greater trochanter (widest part of the hips) and fasten. The pull-handle will be on the left side. The soft lining will be against the body.
3. For proper fit and compression, center the two side panels and position symmetrically on the back.
4. To apply compression, grasp the pull handle with the thumb and forefinger. After the desired level of compression is achieved, fasten the pull-handle to the contact strip on the side panel.

**Cleaning Instructions:** Hand wash in cold water using mild soap. Air dry. If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

**Warning:** If you experience any pain, swelling, sensation changes, or any unusual reaction while using this product, consult your medical professional. If any fraying or deterioration of the cording is noticed, discontinue use.

**Warranty:** Breg, Inc. will replace all or part of this unit and its accessories for material and workmanship defects for a period of 3 months from the date of sale. The proprietary lacer components are guaranteed for the lifetime of the brace.

Single Patient Use Only  
Patent-Pending

Part Number	Description		
60072	BOA S.I. BELT, SMALL	25" - 35"	
60073	BOA S.I. BELT, MEDIUM	35" - 45"	
60074	BOA S.I. BELT, LARGE	45" - 55"	

Measurement: Pelvic circumference, 1" above superior aspect of the greater trochanter.

Distributed by:



EC Rep  
MDSS GmbH  
Schiffgraben 41  
D-30175 Hanover, Germany



AW-1.90111 Rev. B 0309