T Scope® Elbow

FITTING INSTRUCTIONS

ANLEITUNG ZUM ANLEGEN
ISTRUZIONI PER L'ADATTAMENTO
MISE EN PLACE
INSTRUCCIONES DE AJUSTE

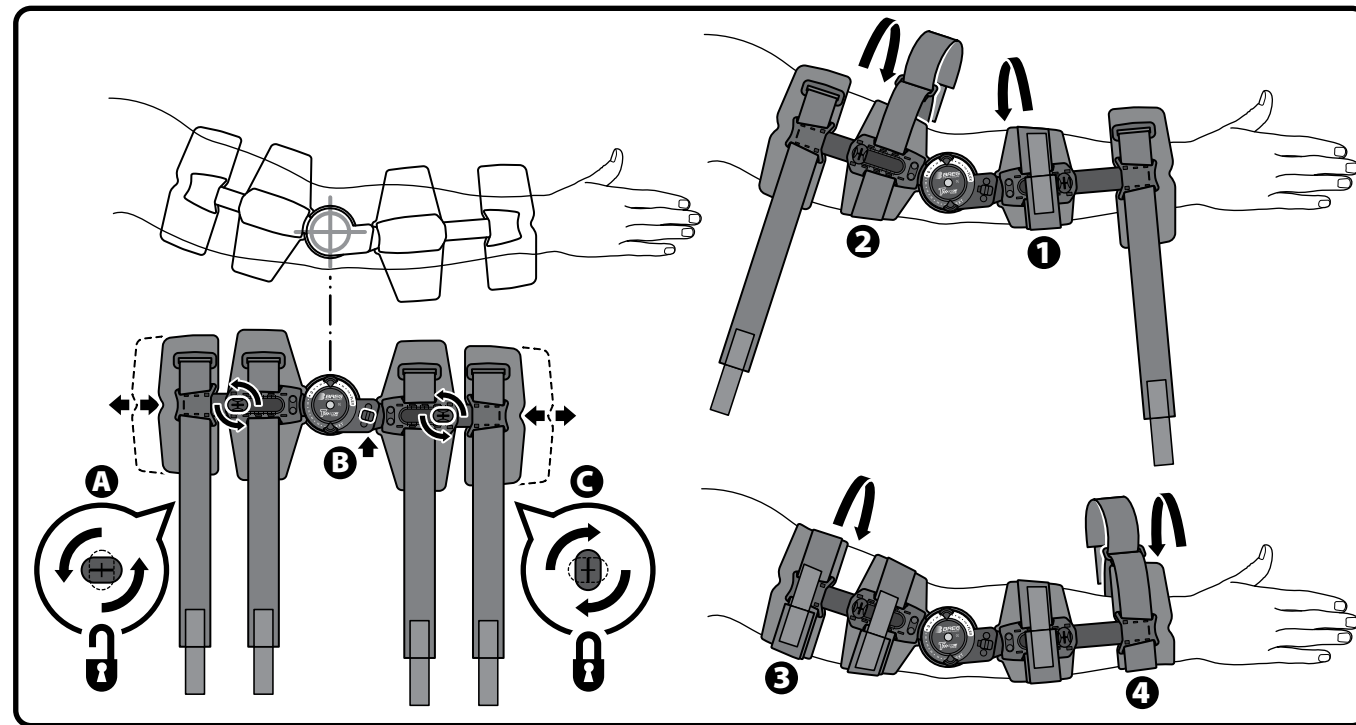
Breg, Inc.
2611 Commerce Way
Vista, CA 92081 U.S.A.
P: 800-321-0607
F: 800-329-2734
www.breg.com
Breg is an Orthofix company

AW-1.00038 Rev C 0411

E/U authorized representative
MDSS GmbH
Schiffgraben 41
D-30175 Hannover
Germany

⚠️ WARNINGS ⚠️

- WARNING:** CAREFULLY READ USE/CARE INSTRUCTIONS AND WARNINGS PRIOR TO USE.
- WARNING:** DO NOT REMOVE T-SCOPE ELBOW BRACE UNLESS INSTRUCTED BY YOUR MEDICAL TREATMENT PROFESSIONAL. DO NOT CHANGE RANGE OF MOTION HINGE SETTINGS WITHOUT SUPERVISION BY A MEDICAL PROFESSIONAL.
- WARNING:** THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM. CONSULT YOUR MEDICAL TREATMENT PROFESSIONAL REGARDING SAFE AND APPROPRIATE ACTIVITY LEVEL WHILE WEARING THIS DEVICE.
- WARNING:** IF YOU EXPERIENCE INCREASED PAIN, SWELLING, SKIN IRRITATION, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.
- CAUTION:** FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.
- CAUTION:** FOR SINGLE PATIENT USE ONLY.

Fitting the T-Scope Elbow:

Unlock the **telescope locks (A)** on the forearm and humeral bars by rotating as shown. Hold the brace up to the arm (example shows right arm) with the hinge centered over the elbow joint. The **hinge lock (B)** on the hinge must be facing the patient's hand. Slide telescoping bars until they match the length of the upper arm and forearm, lock the **telescope locks (C)** by rotating as shown.

While supporting the injured arm, fasten the forearm strap closest to the hinge **(1)**, then the the upper arm strap closest to the hinge **(2)**.

Next, fasten the upper arm strap furthest from the hinge **(3)**, lastly, fasten the strap closest to the hand **(4)**.

Controlling Range Of Motion:

The **spring loaded tabs** may be adjusted to limit range of motion. Extension settings between 10° hyperextension and 120° and flexion settings between 10° and 120° may be selected by pulling the **spring loaded tab** out and sliding it to the desired position.

To prevent range of motion lock the T-Scope Elbow brace by sliding **hinge lock** toward the center of the hinge after positioning the arm at the appropriate angle. For additional security, position the spring loaded tabs at the appropriate lock-out angle.

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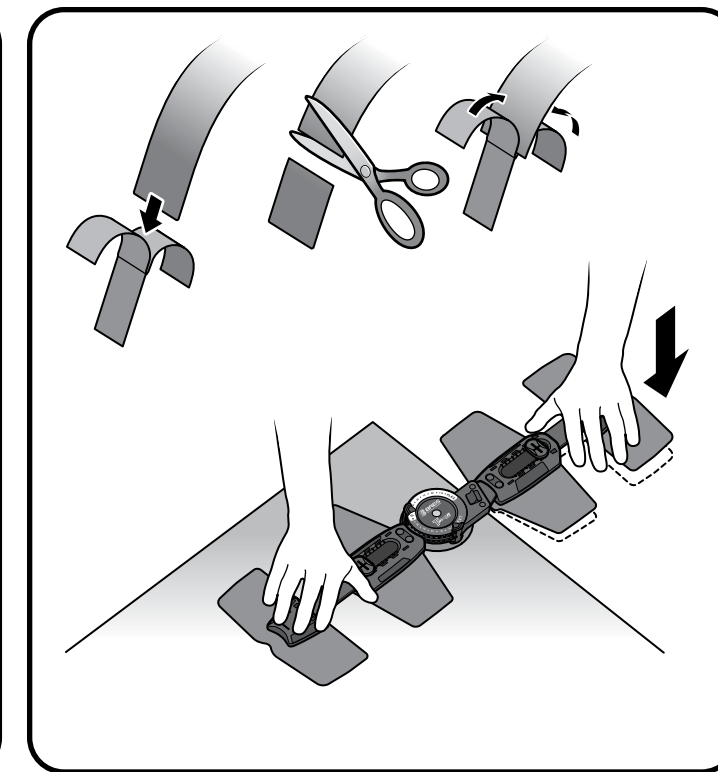
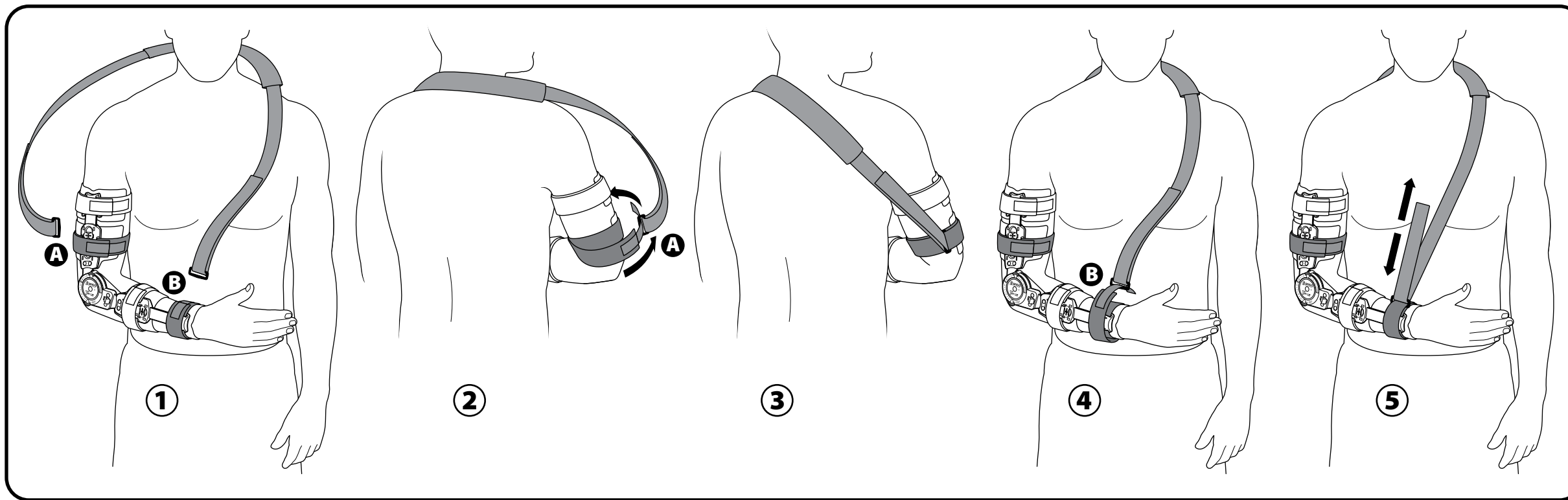
Applying the Shoulder Strap:

- 1) Position the shoulder strap over the patient's shoulder and head with the strap pad located behind the neck. The D-rings on each end of the strap will attach to the upper arm strap closest to the hinge (A) and the forearm strap closest to the hand (B).
- 2) Attach the upper arm strap (A) by sliding the alligator tab at the end of the strap through the D-ring on the end of the shoulder strap. Pull the strap completely through the D-ring and fasten firmly.
- 3) The D-ring on the shoulder strap should be positioned behind the arm as shown.
- 4) Next, attach the forearm strap (B) by sliding the alligator tab at the end of the strap through the D-ring on the end of the shoulder strap. Pull the strap completely through the D-ring and fasten firmly.
- 5) To adjust the length of the shoulder strap, loosen the strap, pull or release the strap through the D-ring until the forearm is held at the proper elevation. Reattach the shoulder strap back to itself.

Customizing the T-Scope Elbow:

The straps may be shortened by trimming off the excess strap with scissors and reattaching the **alligator strap tab**. The brace struts may be bent to achieve a better fit. Place the brace on a table top with one strut protruding off the edge as shown. Apply downward pressure to bend the strut.

Washing Instructions:

Straps and foam pads may be removed by sliding the straps out of the strut housings. Hand wash the foam pads and straps with mild soap, air dry. Do not place the straps or foam pads into a mechanical dryer. Extra straps and foam pads are available from Customer Care: (800) 321-0607 or (760) 599-3000.

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