# KoolAir Ankle

## Fitting, Use and Care Instructions

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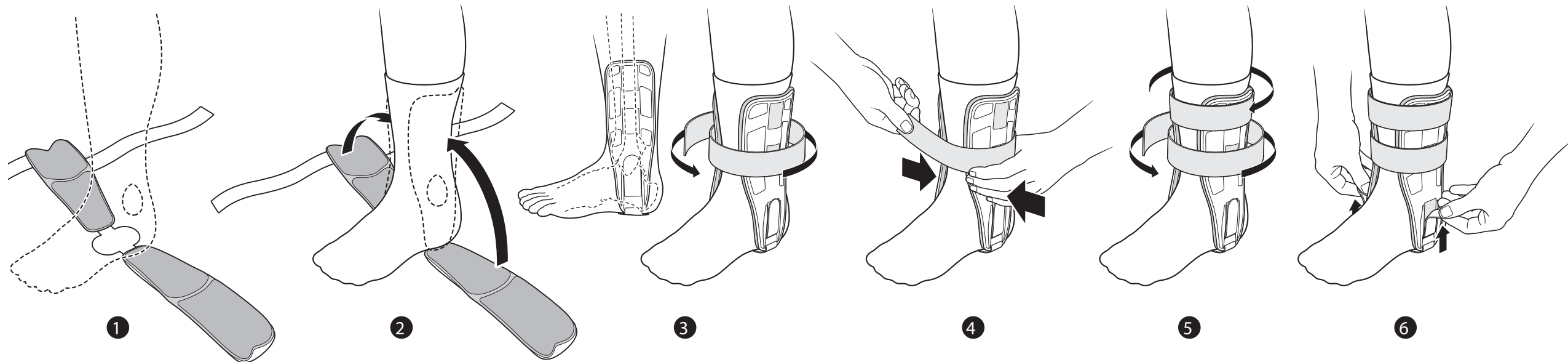
AW 1.00297 Rev A 1210

**WARNING**

WARNING: CAREFULLY READ USE/CARE INSTRUCTIONS AND WARNINGS PRIOR TO USE.

WARNING: THIS DEVICE SHOULD BE WORN OVER A PIECE OF STOCKINETTE OR THIN COTTON SOCK. CONSULT YOUR LICENSED HEALTH CARE PRACTITIONER FOR FURTHER INSTRUCTIONS ON APPLICATION AND TREATMENT.

WARNING: ALWAYS WEAR AN ABSORBENT SOCK BEFORE COLD THERAPY IS APPLIED. LIMIT COLD TREATMENT TO 30 MINUTES. REMOVE. DO NOT REAPPLY UNTIL APPLICATION AREA RETURNS TO NORMAL TEMPERATURE.

WARNING: PRIOR TO COLD APPLICATION, CONSULT YOUR LICENSED HEALTH CARE PRACTITIONER REGARDING POTENTIAL ADVERSE REACTIONS AND CONTRAINDICATIONS. PROVIDE COMPLETE MEDICAL HISTORY AND ANY HISTORY OF PREVIOUS REACTIONS TO COLD APPLICATION. CONTRAINDICATIONS INCLUDE, BUT ARE NOT LIMITED TO, USERS WITH SENSITIVITY TO COLD, PRIOR ADVERSE REACTIONS TO COLD APPLICATION, COMPROMISED CIRCULATION, INCLUDING LOCALIZED COMPROMISE DUE TO MULTIPLE SURGICAL PROCEDURES, COLD APPLICATION AREA DESENSITIZATION, DIABETES, CIRCULATORY SYNDROMES, COGNITIVE DISABILITIES OR COMMUNICATION BARRIERS. WHILE IN USE, INSPECT SKIN RECEIVING COLD TREATMENT REGULARLY. SHOULD CHANGES IN SKIN APPEARANCE OCCUR, SUCH AS: BLISTERS, INCREASED REDNESS, DISCOLORATION, WELTS, OR ANY OTHER ADVERSE REACTION IDENTIFIED BY YOUR PRACTITIONER, DISCONTINUE USE AND CONSULT YOUR PRACTITIONER IMMEDIATELY. DISCONTINUE AND CONSULT YOUR PRACTITIONER IMMEDIATELY IF BURNING, ITCHING, INCREASED PAIN OR SWELLING OCCUR.

CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.

CAUTION: FOR SINGLE PATIENT USE ONLY.

CAUTION: IF YOU ARE WEIGHT BEARING ON THE AFFECTED ANKLE ALWAYS WEAR AN APPROPRIATE SHOE OVER THE ANKLE BRACE.

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- 1 OPEN ANKLE STIRRUP. POSITION HEEL EVENLY ON THE CENTER PAD.**  
Be sure that the gel and closed cell foam are facing upward. Wearing sock is recommended.
- 2 CENTER THE SIDE SHELLS ON MEDIAL & LATERAL MALLEOLUS.**
- 3 WRAP THE LOWER HOOK AND LOOP CLOSURE STRAP AROUND THE ANKLE FOR ALIGNMENT.**
- 4 APPLY PRESSURE TO BOTH SIDE SHELLS.**  
Pressure will form the therapeutic gel bags to the ankle contours.
- 5 RETIGHTEN LOWER STRAP. TIGHTEN UPPER STRAP.**
- 6 ADJUST SIDE HEEL STRAPS.**  
Pull upward until heel center pad feels snug but comfortable. Secure side straps to loop closure pad.

- USE AND CARE:**
- Outer shell may be wiped clean with mild soap and water.
  - If applying cold therapy remove gel bags and place in freezer. Reminder: Always wear an absorbent sock when ankle product is applied for 30 minute cold therapy treatment.

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