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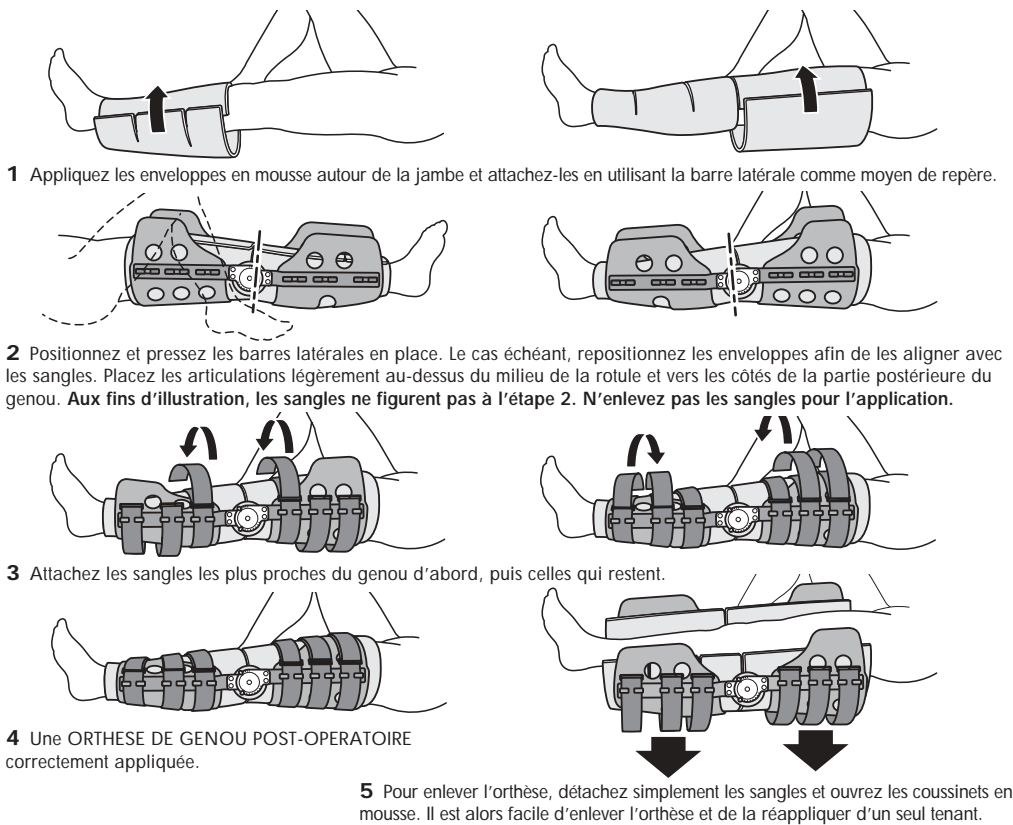
EC responsible person  
 93/42/EEC Annex 1 Essential Requirements  
 MDSS  
 Hauptstrasse 39  
 D-30974 Wennigsen, Germany

BREG Inc.  
 Vista, CA 92081 U.S.A.  
 Telephone (800) 321-0607  
 (760) 599-3000  
 Telefax (760) 598-6193

PN-1.07320 REV D

**CAUTION:** FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.  
**CAUTION:** FOR SINGLE PATIENT USE ONLY.  
**CAUTION:** IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.  
**WARNING:** CAREFULLY READ USE INSTRUCTIONS AND WARNINGS PRIOR TO OPERATION.  
**WARNING:** THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM

**PostOp Knee Brace with shells**

**1** Wrap and fasten the foam wraps around the leg using the sidebar as a gauge for positioning.

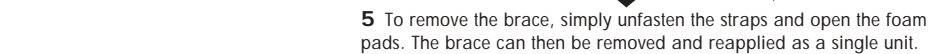
**2** Position and press into place the side bars. Reposition the wraps if necessary to align with the straps. The hinges should be placed slightly above the middle of the knee cap and toward the back of the knee on the sides. For illustration purposes the straps are not shown in step 2. Do not remove the straps for application.

**3** Secure and fasten the straps closest to the knee first, then the remaining straps.

**4** A properly applied POST-OP KNEE BRACE.


**5** To remove the brace, simply unfasten the straps and open the foam pads. The brace can then be removed and reapplied as a single unit.

**TIPS ON USE AND CARE OF YOUR POST-OP KNEE BRACE:**

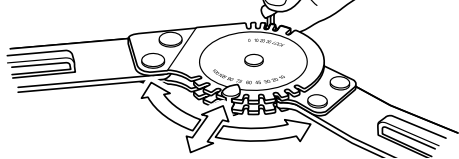
- Close the foam wraps in the front to allow easy removal of the brace later.
- The foam wraps may be trimmed with scissors as needed.
- The side bars may be bent as illustrated at RIGHT to accommodate individual legs. Bend in equal amounts above and below the hinge and on each side.
- Wash foam wraps and straps by hand only and air dry. Extra wraps are available from BREG.
- To tighten straps securely it is easiest to use two hands—one to pull the end after it passes through the ring and the other to push the strap on the leg toward the ring.

**QUICK-SET™ HINGE ADJUSTMENT INSTRUCTIONS:**


To adjust the hinge, simply pull the stop out of its slot and move it to the appropriate slot that corresponds to the degree of flexion or extension desired. The slot identified as "LOCK" is used for locking the brace in full extension.


**TO BREAK THE POST-OP BRACE DOWN:**

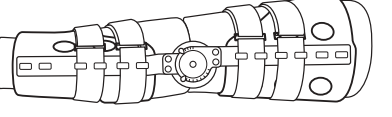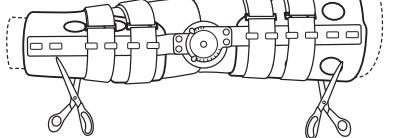
**1** To shorten the brace, first remove the top thigh strap and bottom calf strap from the side bars.

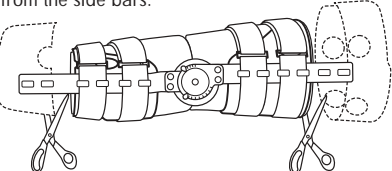**2** Cut the foam wraps down in length, leave the foam at least 1/4" to 1/2" longer than the indentations in the plastic shells. This will allow the foam wraps to extend beyond the length of the cut down plastic shells.

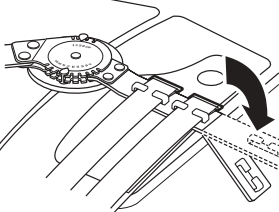**3** After shortening the foam wraps, cut across the plastic shell at the indentations, repeat on all the shells.

**4** Last, shorten the side bars by placing one of the side bars on a table, extend the bar off the edge of the table. Bend the side bar at the molded crease, bend back and forth until the bar end breaks off. Repeat procedure with the other bar ends.

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