

# Arm Sling, Shoulder Immobilizer, Pediatric Sling

## FITTING INSTRUCTIONS



**BREG Inc.**  
 Vista, CA 92081 U.S.A.  
 WWW.BREG.COM

Telephone (800) 321-0607  
 (760) 599-3000  
 Telefax (760) 598-6193

AW-1.90210 REV A

**⚠ W A R N I N G S ⚠**

**CAUTION:** FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.  
**CAUTION:** FOR SINGLE PATIENT USE ONLY.  
**CAUTION:** IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.  
**WARNING:** CAREFULLY READ FITTING INSTRUCTIONS AND WARNINGS PRIOR TO USE.  
**WARNING:** THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM.

### Arm Sling

- 1** Slide arm into sling and place thumb in thumb loop.
- 2** Wrap shoulder strap around neck and shoulders and position shoulder pad on neck. Lace strap end through plastic D-ring and attach to shoulder strap.
- 3** Properly applied sling with arm comfortably supported and thumb in the thumb loop.

### Shoulder Immobilizer

After fitting the sling as indicated above, wrap waist strap around waist and feed through plastic ring. Attach hook and loop closure.

### Pediatric Sling

Wrap shoulder strap around neck and shoulders and position shoulder pad on neck.

Lace strap end through the 2 steel D-rings and attach to shoulder strap.

**Washing Instructions:** Fasten hook and loop closures. Hand wash in warm water using a mild detergent. Air dry.