# Vectra™ Basic Air Walker Vectra™ Premium Air Walker

Short and Tall  
Klein und Groß  
Corto e lungo  
Courte et haute  
Corta y alta

Fitting, Use, and Care Instructions  
Anleitung zu Anpassung, Gebrauch und Pflege  
Istruzioni per l'adattamento, l'uso e la manutenzione  
Mode d'emploi, mise en place et consignes d'entretien  
Instrucciones de ajuste, uso y cuidado

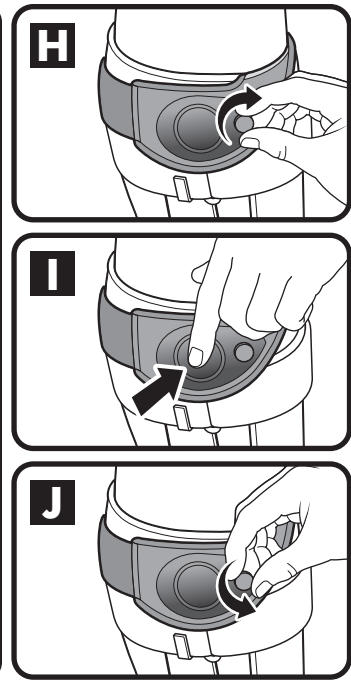
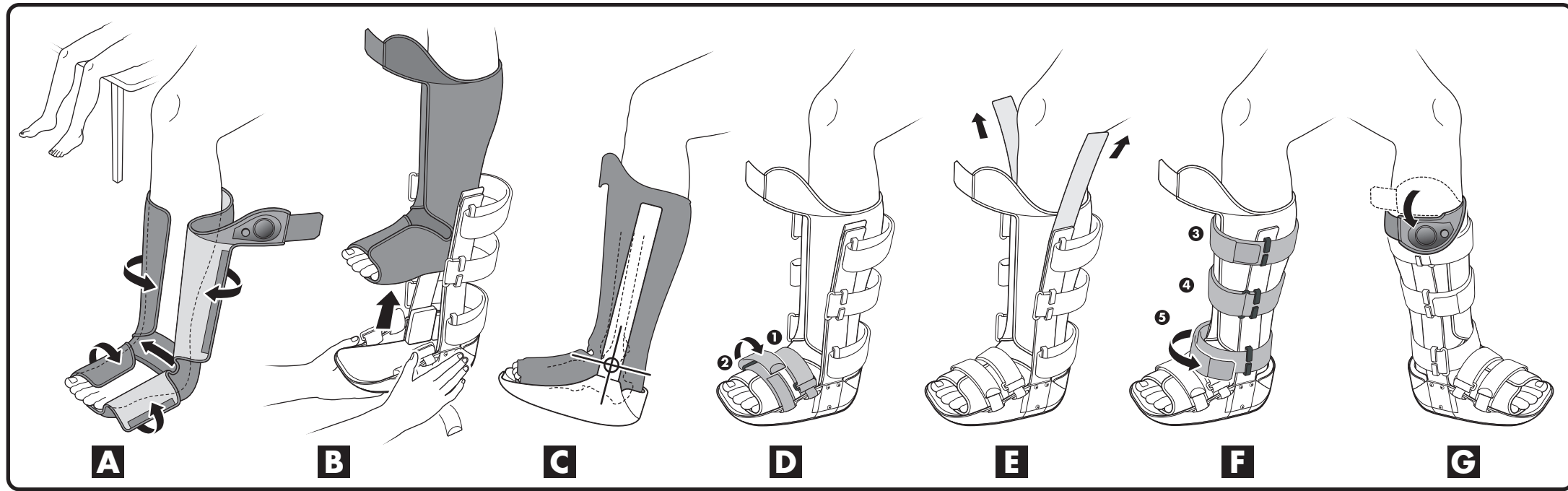
BREG, Inc. 2611 Commerce Way | Vista, CA 92081  
P: 800.321.0607 | F: 800.FAX.BREG  
www.breg.com

AW-1.90333 REV C 1010

E/U authorized representative  
MDSS GmbH  
Schiffgraben 41  
D-30175 Hannover  
Germany

**⚠ WARNINGS ⚠**

**WARNING:** CAREFULLY READ USE/CARE INSTRUCTIONS AND WARNINGS PRIOR TO USE.

**WARNING:** DO NOT REMOVE WALKER BOOT UNLESS INSTRUCTED BY YOUR MEDICAL TREATMENT PROFESSIONAL.

**WARNING:** THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM.

**WARNING:** IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.

**WARNING:** IF YOU EXPERIENCE ANY SKIN IRRITATION OR ADVERSE SKIN REACTION WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR HEALTH CARE PROFESSIONAL.

**WARNING:** TAKE APPROPRIATE PRECAUTIONS WHEN WALKING ON SLIPPERY SURFACES, UNEVEN TERRAIN, OR ANY OTHER PROPERTY CONDITION WHERE THE EXERCISE OF CAUTION APPEARS NECESSARY.

**WARNING:** DO NOT OPERATE A MOTOR VEHICLE OR HUMAN POWERED VEHICLE WHILE WEARING THIS WALKER BOOT.

**WARNING:** DO NOT OVER INFLATE THIS PRODUCT. SIGNS OF OVER-INFLATION INCLUDE, BUT ARE NOT LIMITED TO: DECREASED CIRCULATION (NUMBNESS AND TINGLING), SKIN SENSATION CHANGES, SKIN COLOR CHANGES, AND AN INCREASE IN PAIN.

**WARNING:** PLEASE CONSULT YOUR HEALTH CARE PROFESSIONAL BEFORE USING THIS PRODUCT ABOARD AN AIRPLANE AS THE INCREASE IN AIR PRESSURE CAN ADVERSELY AFFECT ITS NORMAL FUNCTION AND CAUSE FURTHER INJURY.

**WARNING:** DEVICE MAY NEED TO BE RE-INFLATED PERIODICALLY.

**CAUTION:** FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH PRACTITIONER.

**CAUTION:** FOR SINGLE PATIENT USE ONLY. OUTER SOLE MAY DETERIORATE WITH MULTIPLE USES. 180 DAY PRODUCT WARRANTY.

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## Initial application by a medical professional only!

- A Apply liner to leg.** With patient seated, leg(s) bent, and feet off floor, position tongue pad as needed for maximum comfort, close liner snugly.
- B Slide walker onto leg over the liner straight up directly under foot.**
- C Position walker with uprights centered over the malleoli (ankle bones).** Ensure heel is positioned at the rear of the Walker.
- D Fasten straps.** Start with straps 1 and 2 closest to the toes (Straps may be trimmed to desired length by removing tab at end of straps, trim, and reattach tab).
- E Remove quick-pulls from the uprights and discard. Attach uprights to the liner. Press firmly.**
- F Fasten straps.** Fasten remaining straps 3 through 5.
- G Fold down** inflation bulb and affix in preferred location (bulb position is variable and dependent on calf circumference).
- H Close release valve clockwise (to the right).**
- I Inflate air bladders by pressing on inflation bulb** to desired pressure for maximum comfort and stabilization.
- J To release the air, twist the release valve counter-clockwise (to the left).**  
  
**To re-inflate the air bladders, twist the release valve clock-wise and press the inflation bulb to desired pressure.**

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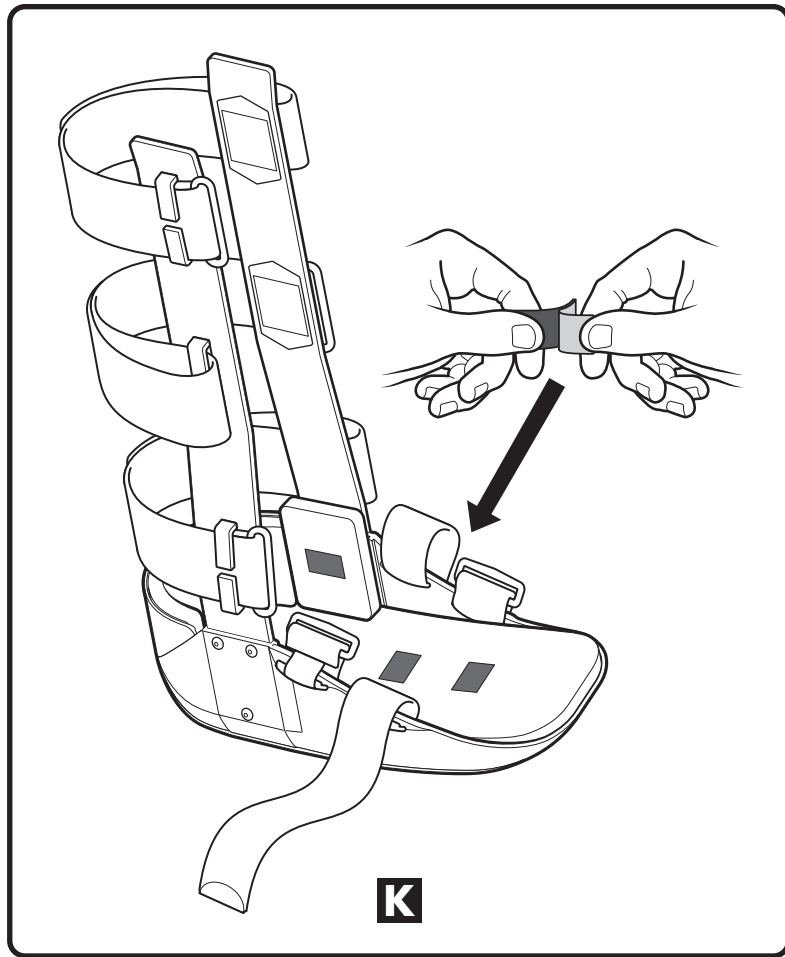
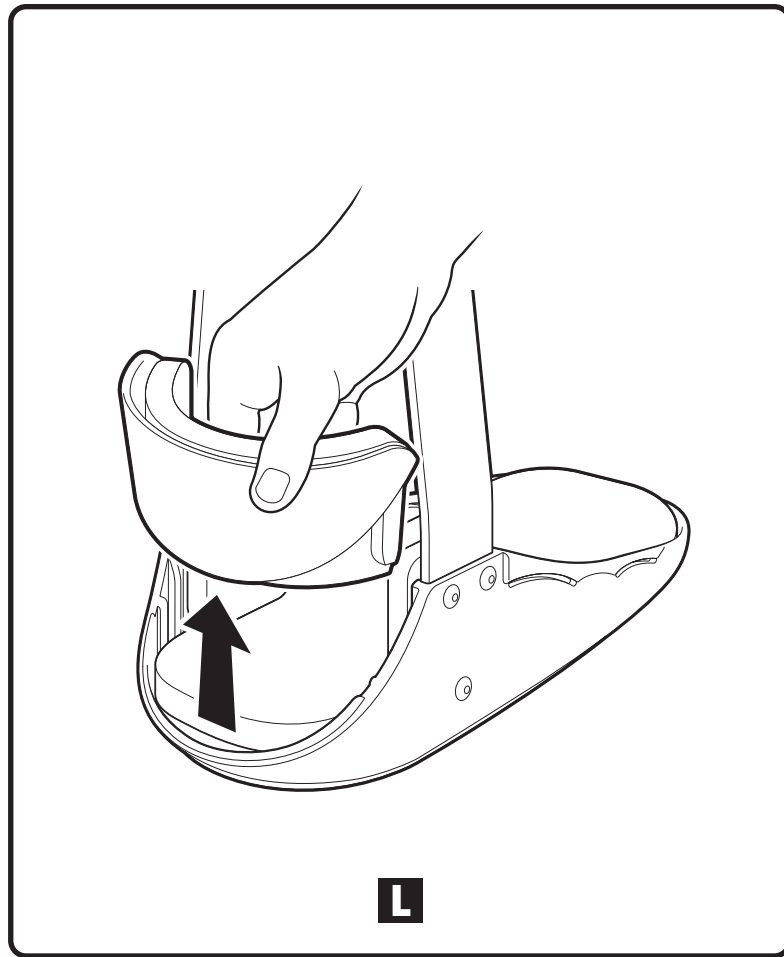
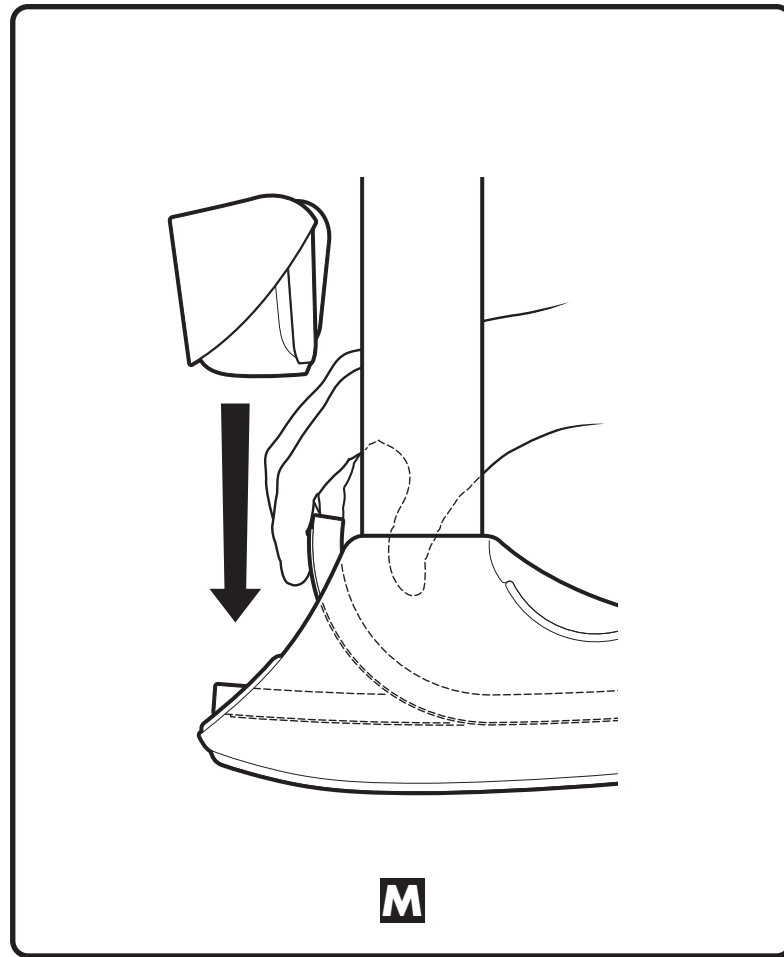
**M**

### Optional application instructions

- K Optional:**  
To enhance liner fixation, attach the 1" x 1/2" hook pieces to the liner at malleoli foam pads (or on the foot bed of the walker).
- L Optional:**  
Heel clip removal; Pull up on the heel clip.
- M Optional:**  
Heel clip replacement; Lift up foot bed padding and foot plate. Slide heel clip into place using the dove-tail slot guides. Release and push the foot plate and padding down into proper position

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### Care instructions

After initial application, the Vectra walker boot may be removed and reapplied without removing liner from boot.

Hand wash the liner and straps with mild soap, rinse thoroughly and air dry. Do not wash or dry in any mechanical device.

Extra liners with strap sets are available from Customer Relations: (800) 321-0607

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