### Break-In Period

Like a new pair of shoes, ankle braces require a break-in period. New shoes combined with a new ankle brace require an even longer break-in period (usually 6-8 hours). Loosen the shoelaces to reduce friction until the brace and the shoe break-in. Wear an additional full tall sock to help reduce friction.

### Wear a Full Tall Sock

Blisters can occur if the brace is in direct contact with the skin. Always apply the brace over an absorbent full tall sock.

### Blisters

Blisters are the result of friction. Loosen the shoelaces to reduce friction until the brace and the shoe break-in. Wear an additional full tall sock to help reduce friction. You may require a different size brace.

### Custom Orthotic or Arch Support

Place the bottom of the brace beneath the orthotic or arch support.

### Reinforce & Recover

[www.UltraAnkle.com](http://www.UltraAnkle.com)

**WARNING:** While the Ultra High-5™ brace may be used for injury prevention, no ankle brace can prevent all ankle injuries and no guarantee is made either express or implied that any injury will be prevented by this brace.

Ultra High-5™ is a trademark of Ultra Athlete LLC  
US Patents 6,053,884 - 6,524,266 - 6,749,578  
6,858,017 European Patent 1,152,720  
Canadian Patent 2,362,779

AW-193010 REV A 1011

SIZE CHART	MEN	WOMEN
Small	5-9	6-10
Medium	9½-12	10½-13
Large	12½ & Up	13½ & Up

**Reinforce & Recover**

## FITTING INSTRUCTIONS

### TREAT ALL ANKLE INJURIES

Introducing the first brace to treat all ankle injuries, including high ankle injuries. The Ultra High-5™ brace with superior Hinged Cuff Technology is designed for the athlete who values exceptional ankle protection when playing sports.

### HINGED CUFF TECHNOLOGY

Helps restrict excessive rotational (twisting) and inversion (turning) forces that cause ankle injuries.

### STEP #1

If your shoe has a removable insole, place the Ultra High-5™ beneath it.

### STEP #2

Place your foot inside the shoe. Then slide the bottom of the brace toward the back of the shoe as far as it will go.

### STEP #3


Secure the strap firmly and tie shoe laces.

**Breg, Inc.**  
2885 Loker Ave. East  
Carlsbad, CA 92010 U.S.A.  
P: 800-321-0607  
F: 800-329-2734  
www.breg.com  
Breg is an Orthofix company  
AW-1.93010 REV A 1011

©2011 Breg, Inc. Breg is an Orthofix company.  
All rights reserved.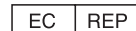

E/U authorized representative  
MOSS GmbH  
Schiffgraben 41  
D-30175 Hannover  
Germany

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