Break-In Period

Like a new pair of shoes, ankle braces require a break-in period. New shoes combined with a new ankle brace require an even longer break-in period (usually 6-8 hours). Loosen the shoelaces to reduce friction until the brace and the shoe break-in. Wear an additional full tall sock to help reduce friction.

Wear a Full Tall Sock

Blisters can occur if the brace is in direct contact with the skin. Always apply the brace over an absorbent full tall sock.

Blisters

Blisters are the result of friction. Loosen the shoelaces to reduce friction until the brace and the shoe break-in. Wear an additional full tall sock to help reduce friction. You may require a different size brace.

Prevent & Perform

www.UltraAnkle.com

WARNING: While the Ultra Zoom™ brace may be used for injury prevention, no ankle brace can prevent all ankle injuries and no guarantee is made either express or implied that any injury will be prevented by this brace.

Ultra Zoom™ is a trademark of Ultra Athlete LLC
US Patents 6,749,578 - 7,615,026 - 7,713,224 -
D611,157 - D611,158

AW-1.93011 REV A 1011

SIZE CHART	MEN	WOMEN
Small/Medium	6-10	7-11
Large/X-Large	10 & Up	11 & Up

FITTING INSTRUCTIONS

STEP #1

- Wear a FULL TALL sock
- Secure the brace to the ankle FIRST, then insert foot in the shoe.
- 4-6 hour break-in period required.

Prevent & Perform

STEP #2

Squeeze the brace tightly around the leg while securing the TOP strap. Securing the TOP STRAP FIRST ensures a proper fit.

The TOP strap length can be adjusted by repositioning it through the buckle.

STEP #3

Secure both ends of the Inversion Strap to the center pad section (see correct application on opposite page). Do not overlap the straps.

Secure straps adjacent to one another. Do not overlap the straps. Two slots are located on each side of the foot portion for maximum adjustability.

Breg, Inc.
2885 Loker Ave. East
Carlsbad, CA 92010 U.S.A.
P: 800-321-0607
F: 800-329-2734
www.breg.com
Breg is an Orthofix company

AW-193011 REV A 1011

©2011 Breg, Inc. Breg is an Orthofix company.
All rights reserved.

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