# Orthopedic Support

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SUPPORTS ORTHOPEDIQUES

SOPORTES ORTOPÉDICOS

Breg, Inc.  
2611 Commerce Way  
Vista, CA 92081 U.S.A.  
P: 800-321-0607  
F: 800-329-2734

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CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.

CAUTION: FOR SINGLE PATIENT USE ONLY.

CAUTION: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.

WARNING: CAREFULLY READ USE INSTRUCTIONS AND WARNINGS PRIOR TO OPERATION.

WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM

## ORTHOPEDIC SUPPORTS

- Hand wash and thoroughly rinse support. Air dry. To prolong product life and reduce skin problems, wash regularly.
- Certain products have hinge bars that may be removed for cleaning. Normally, hinge lubrication is not required, however a dry lubricant, such as teflon spray may be used.
- After exposure to salt water, thoroughly rinse support in fresh water and air dry.
- Periodically inspect support for wear. Do not use if worn excessively.
- Do not wear support directly over open wounds.
- If you experience dermatitis or adverse skin reactions while using this product, consult your health care professional.

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