


## Cervical Collar with Open Trachea Application Instructions



1. Please select the correct size as the collar comes in different heights and circumferences. Apply collar piece "Back" to the back of patient's neck. Height - measure bottom of chin to top of sternum.

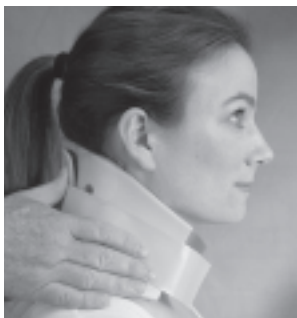

2. Apply front half of collar with chin secured in recess. Center collar to ensure neutral alignment. Front half overlaps back half.



3. With the hook and loop fasteners tighten collar with bi-lateral adjustments. Center collar with arrow pointing up.



4. Adjust if necessary.



Manufactured For:

Breg, Inc.  
2611 Commerce Way  
Vista, CA 92081 U.S.A.  
P: 800-321-0607  
F: 800-329-2734


[www.breg.com](http://www.breg.com)  
Breg is an Orthofix company

AW - 1.94000 Rev B 0411

## Cervical Collar with Open Trachea Application Instructions



1. Please select the correct size as the collar comes in different heights and circumferences. Apply collar piece "Back" to the back of patient's neck. Height - measure bottom of chin to top of sternum.

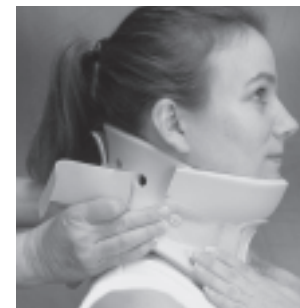

2. Apply front half of collar with chin secured in recess. Center collar to ensure neutral alignment. Front half overlaps back half.



3. With the hook and loop fasteners, tighten collar with bi-lateral adjustments. Center collar with arrow pointing up.



4. Adjust if necessary.



Manufactured For:

Breg, Inc.  
2611 Commerce Way  
Vista, CA 92081 U.S.A.  
P: 800-321-0607  
F: 800-329-2734

[www.breg.com](http://www.breg.com)  
Breg is an Orthofix company

AW - 1.94000 Rev B 0411