



Apollo Wrist Brace - Universal


Application Instructions

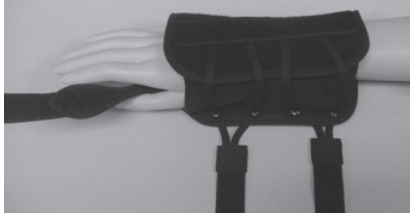


1. Open brace and position hand in wrist brace.

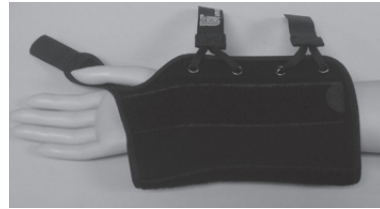

2. The palmer stay should line up below the middle crease of the hand.

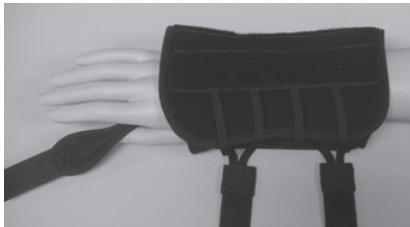

3. Align the lacing dorsal stay with the middle of the knuckles.

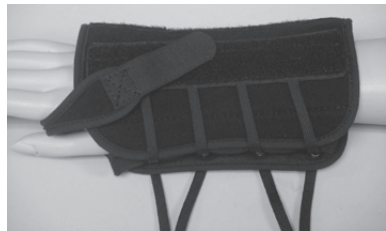

4. Secure the thumb strap to the brace, aligning it between the thumb and index finger.



5. Wrap the lacing straps around wrist and attach to hook and loop. Re-adjust if necessary. The knuckles should be free for bending.



Manufactured For:


Breg, Inc.
2611 Commerce Way
Vista, CA 92081 U.S.A.
P: 800-321-0607
F: 800-329-2734

www.breg.com
Breg is an Orthofix company


AW - 1.94001 Rev C 0411

Apollo Wrist Brace - Universal


Application Instructions

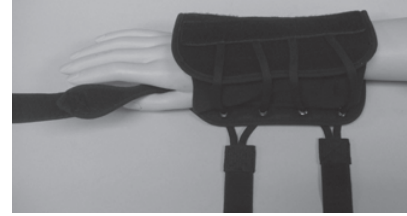


1. Open brace and position hand in wrist brace.

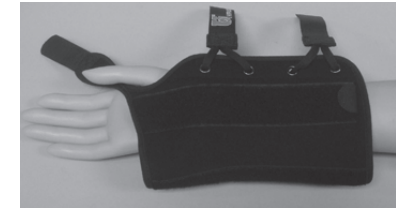

2. The palmer stay should line up below the middle crease of the hand.

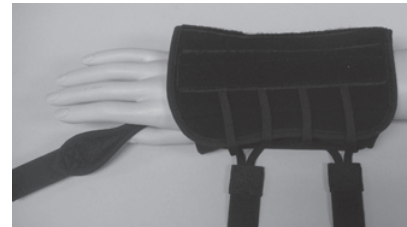

3. Align the lacing dorsal stay with the middle of the knuckles.

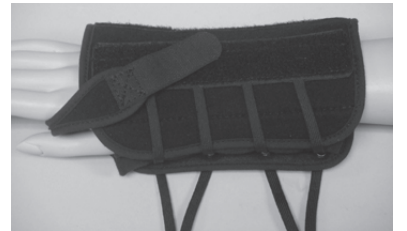

4. Secure the thumb strap to the brace, aligning it between the thumb and index finger.



5. Wrap the lacing straps around wrist and attach to hook and loop. Re-adjust if necessary. The knuckles should be free for bending.



Manufactured For:

Breg, Inc.
2611 Commerce Way
Vista, CA 92081 U.S.A.
P: 800-321-0607
F: 800-329-2734

www.breg.com
Breg is an Orthofix company

AW - 1.94001 Rev C 0411