


Apollo Wrist Brace with Spica

Application Instructions

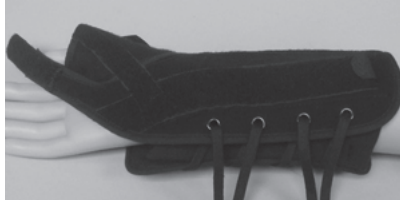

1. Open brace and position hand in thumb brace.



2. The palmer stay should line up below the middle crease of the hand.



3. The thumb stay should follow the length of the thumb and arm.



4. Align the lacing dorsal stay with the middle of the knuckles.



5. Wrap the wide thumb strap around the thumb and secure to hook.



6. Secure the thinner strap between the thumb and index finger and attach to hook.



7. Wrap the lacing straps around wrist and attach to hook and loop. Re-adjust if necessary. The knuckles should be free for bending.


Hand Wash Cold - Air Dry

Apollo Wrist Brace with Spica

Application Instructions



1. Open brace and position hand in thumb brace.

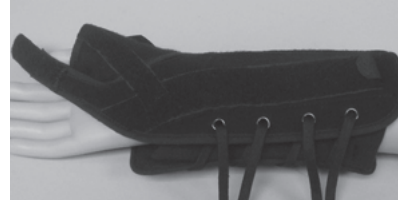

2. The palmer stay should line up below the middle crease of the hand.



3. The thumb stay should follow the length of the thumb and arm.



4. Align the lacing dorsal stay with the middle of the knuckles.



5. Wrap the wide thumb strap around the thumb and secure to hook.



6. Secure the thinner strap between the thumb and index finger and attach to hook.



7. Wrap the lacing straps around wrist and attach to hook and loop. Re-adjust if necessary. The knuckles should be free for bending.

Hand Wash Cold - Air Dry



Manufactured For:

Breg, Inc.
2611 Commerce Way
Vista, CA 92081 U.S.A.
P: 800-321-0607
F: 800-329-2734

www.breg.com
Breg is an Orthofix company

AW - 1.94002 Rev C 0411



Manufactured For:

Breg, Inc.
2611 Commerce Way
Vista, CA 92081 U.S.A.
P: 800-321-0607
F: 800-329-2734

www.breg.com
Breg is an Orthofix company

AW - 1.94002 Rev C 0411