



Wrist Mate 8" & 10"

The special wedge contour design gives a true fit for the hand and wrist while allowing full freedom to the thumb maintaining excellent support. The design includes a unique loop lock closure giving optimal immobilization. It does not extend above the palmar crease. Available in 8" and 10" length.


APPLICATION INSTRUCTIONS

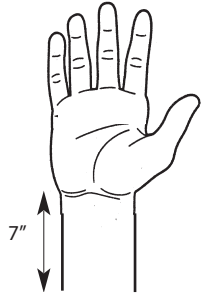


1. Measure circumference of the injured forearm 7" (17.8cm) above the wrist joint to determine the proper size.

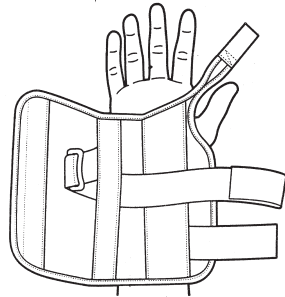

2. Loosen closures. Slip the splint onto the affected hand.

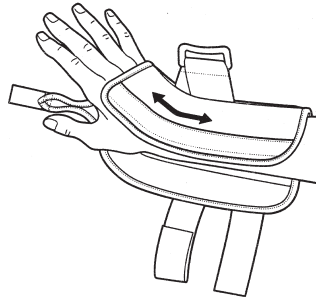

3. Bend the dorsal stay to match the wrist contour.

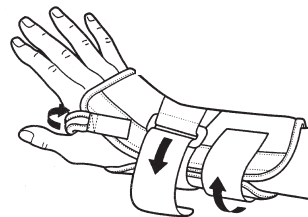

4. Pass circumferential wrist strap through the loop lock closure. Secure the remaining closures and adjust for comfort.

Note : The palmar stay is malleable so it may be adjusted.



Manufactured For:

Breg, Inc.
2611 Commerce Way
Vista, CA 92081 U.S.A.
P: 800-321-0607
F: 800-329-2734


www.breg.com
Breg is an Orthofix company

AW - 1.94008 Rev B 0411


Wrist Mate 8" & 10"

The special wedge contour design gives a true fit for the hand and wrist while allowing full freedom to the thumb maintaining excellent support. The design includes a unique loop lock closure giving optimal immobilization. It does not extend above the palmar crease. Available in 8" and 10" length.


APPLICATION INSTRUCTIONS

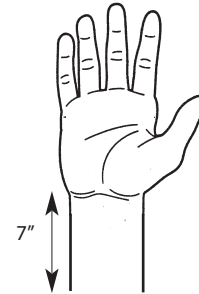


1. Measure circumference of the injured forearm 7" (17.8cm) above the wrist joint to determine the proper size.

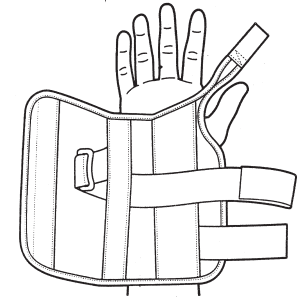

2. Loosen closures. Slip the splint onto the affected hand.

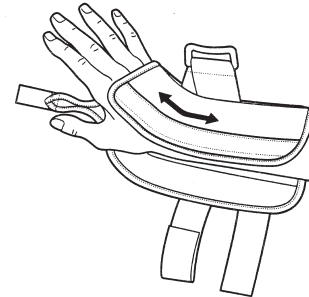

3. Bend the dorsal stay to match the wrist contour.

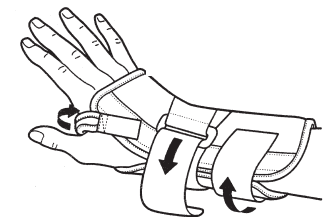

4. Pass circumferential wrist strap through the loop lock closure. Secure the remaining closures and adjust for comfort.

Note : The palmar stay is malleable so it may be adjusted.



Manufactured For:

Breg, Inc.
2611 Commerce Way
Vista, CA 92081 U.S.A.
P: 800-321-0607
F: 800-329-2734

www.breg.com
Breg is an Orthofix company

AW - 1.94008 Rev B 0411