

# Shure Shoulder Immobilizer



- Breathable, cool cotton material
- Removable foam swathe strap adjusts easily
- O-Ring on strap for easy adjustment
- Hook and Loop closure

## Application Instructions

1. Place injured arm in sling and place thumb in thumb holder
2. Place strap around neck and secure through buckle and adjust for proper height and comfort
3. Place waist strap around waist and feed strap through O-ring and attach hook and loop closure



## Manufactured For:

Breg, Inc.  
2611 Commerce Way  
Vista, CA 92081 U.S.A.  
P: 800-321-0607  
F: 800-329-2734

[www.breg.com](http://www.breg.com)  
Breg is an Orthofix company