

PREMIER THUMB SUPPORT with STAYS

APPLICATION INSTRUCTIONS



1. Open thumb orthosis flat, place on affected thumb and wrap around thumb and secure thumb strap.
2. Wrap wrist strap snugly, securing to both hook areas along wrist area.
3. Re-adjust as necessary for comfortable fit.
4. Interchangeable flexible, rigid and malleable stays can be removed if desired.



Manufactured For:

Breg, Inc.
2611 Commerce Way
Vista, CA 92081 U.S.A.
P: 800-321-0607
F: 800-329-2734

www.breg.com
Breg is an Orthofix company

AW - 1.95002 Rev B 0411

PREMIER THUMB SUPPORT with STAYS

APPLICATION INSTRUCTIONS



1. Open thumb orthosis flat, place on affected thumb and wrap around thumb and secure thumb strap.
2. Wrap wrist strap snugly, securing to both hook areas along wrist area.
3. Re-adjust as necessary for comfortable fit.
4. Interchangeable flexible, rigid and malleable stays can be removed if desired.



Manufactured For:

Breg, Inc.
2611 Commerce Way
Vista, CA 92081 U.S.A.
P: 800-321-0607
F: 800-329-2734

www.breg.com
Breg is an Orthofix company

AW - 1.95002 Rev B 0411