

Arm Sling, Shoulder Immobilizer, Pediatric Sling

FITTING INSTRUCTIONS



Breg, Inc.
2885 Loker Ave. East
Carlsbad, CA 92010 U.S.A.
P: 800-321-0607
760-795-5440
F: 800-329-2734
www.breg.com

©2014 Breg, Inc.
AW 1.90210 REV B 1014

⚠ W A R N I N G S

WARNING: CAREFULLY READ FITTING INSTRUCTIONS AND WARNINGS PRIOR TO USE. TO ENSURE PROPER PERFORMANCE OF THE BRACE, FOLLOW ALL INSTRUCTIONS.


WARNING: IF YOU EXPERIENCE, OR HAVE AN INCREASE IN PAIN, SWELLING, SKIN IRRITATION, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.

WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM. CONSULT YOUR LICENSED HEALTH CARE PROFESSIONAL REGARDING SAFE AND APPROPRIATE ACTIVITY LEVEL WHILE WEARING THIS DEVICE.




CAUTION: FOLLOW APPLICATION AND CARE PROCESS DESCRIBED IN INSTRUCTIONS.

CAUTION: FOR SINGLE PATIENT USE ONLY.



Arm Sling

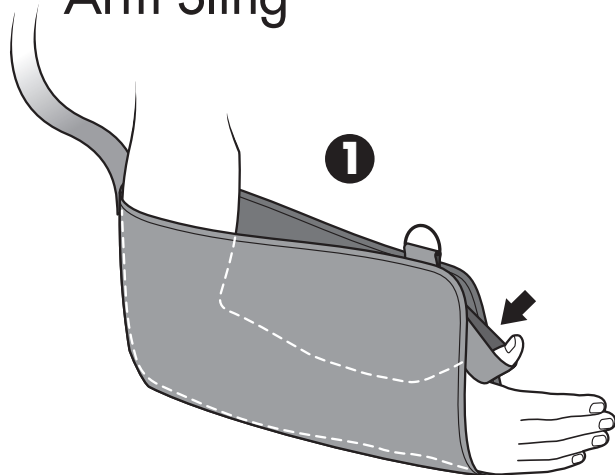

- 1 Slide arm into sling and place thumb in thumb loop.
- 2 Wrap shoulder strap around neck and shoulders and position shoulder pad on neck. Lace strap end through plastic D-ring and attach to shoulder strap.
- 3 Properly applied sling with arm comfortably supported and thumb in the thumb loop.

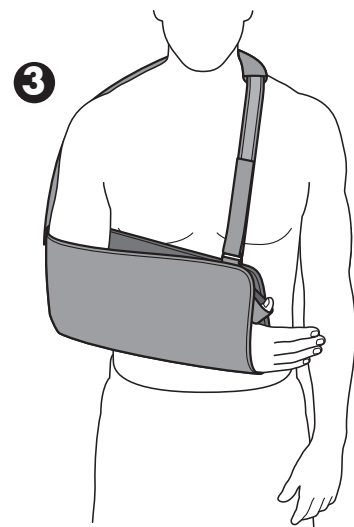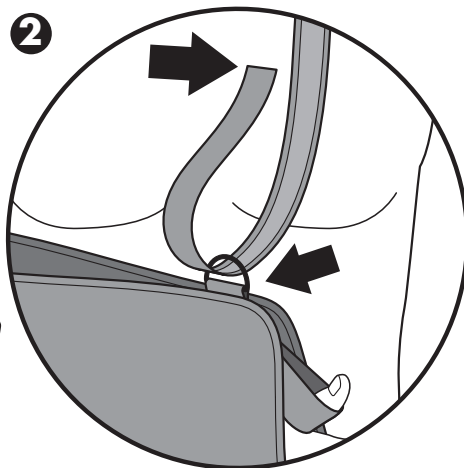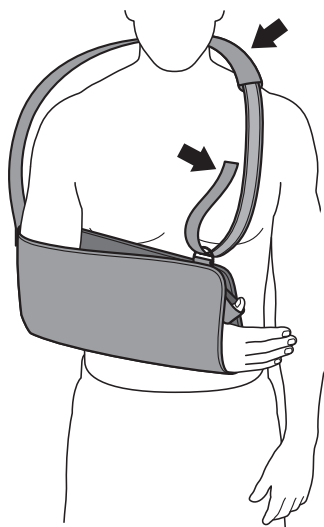


Shoulder Immobilizer

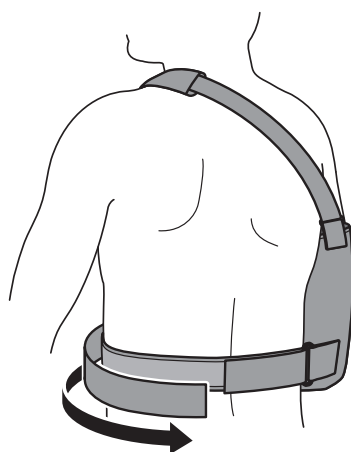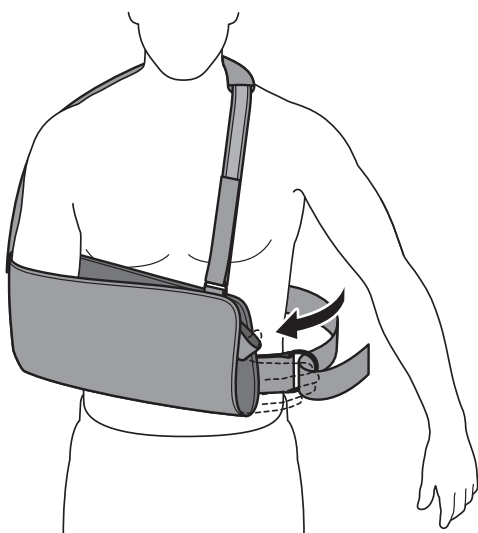

After fitting the sling as indicated above, wrap waist strap around waist and feed through plastic ring. Attach hook and loop closure.

Pediatric Sling

Wrap shoulder strap around neck and shoulders and position shoulder pad on neck.

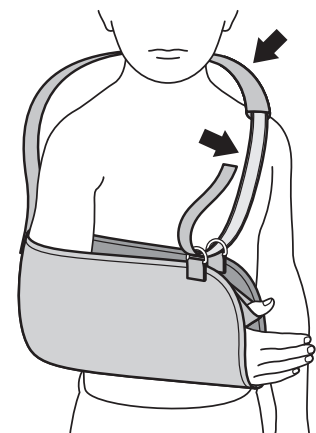

Lace strap end through the 2 steel D-rings and attach to shoulder strap.

Washing Instructions: Fasten hook and loop closures. Hand wash in warm water using a mild detergent. Air dry.