Svenska

1. ÖPPNA SPÄNNENA

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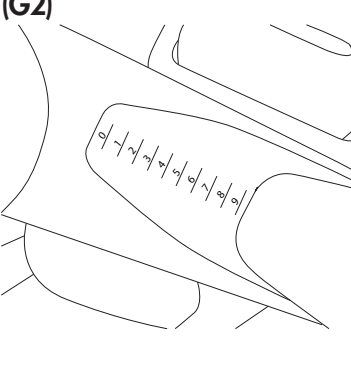
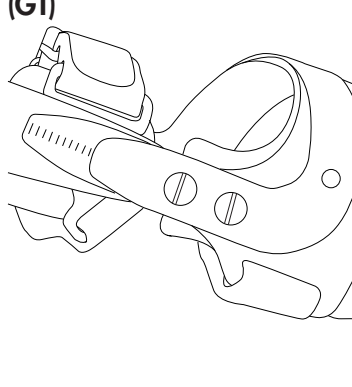
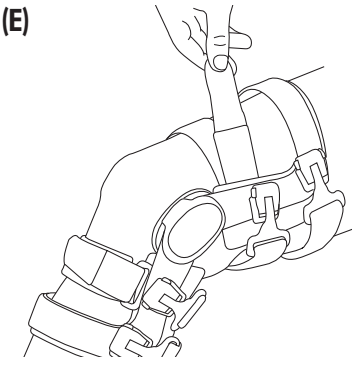
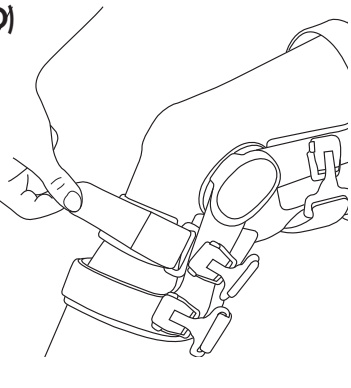
INDICATIONS: The DUO brace can be configured to address medial or lateral osteoarthritis. The brace is designed to decrease joint pain by creating a space between the bones on the affected side of the knee.

CONTRAINDICATIONS: The DUO is contraindicated to treat medial osteoarthritis with a lateral osteoarthritis brace or to treat lateral osteoarthritis with a medial osteoarthritis brace.

CAUTION: This device is not intended for use if such an incident has occurred.

CAUTION: Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device,

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discontinue use and seek medical attention.

CAUTION: This device is not intended to prevent injury, or reduce or eliminate the risk of re-injury.

CAUTION: This device has been developed for the indications listed above, and may be damaged by unusual activities such as sports, falls, or other accidents, or exposure of the hinges to debris or liquids. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

CAUTION: Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150F (65 C.) for any extended period of time.

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© 2015 Breg, Inc. All rights reserved.
MDSS GmbH