Freestyle™ OA Brace

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
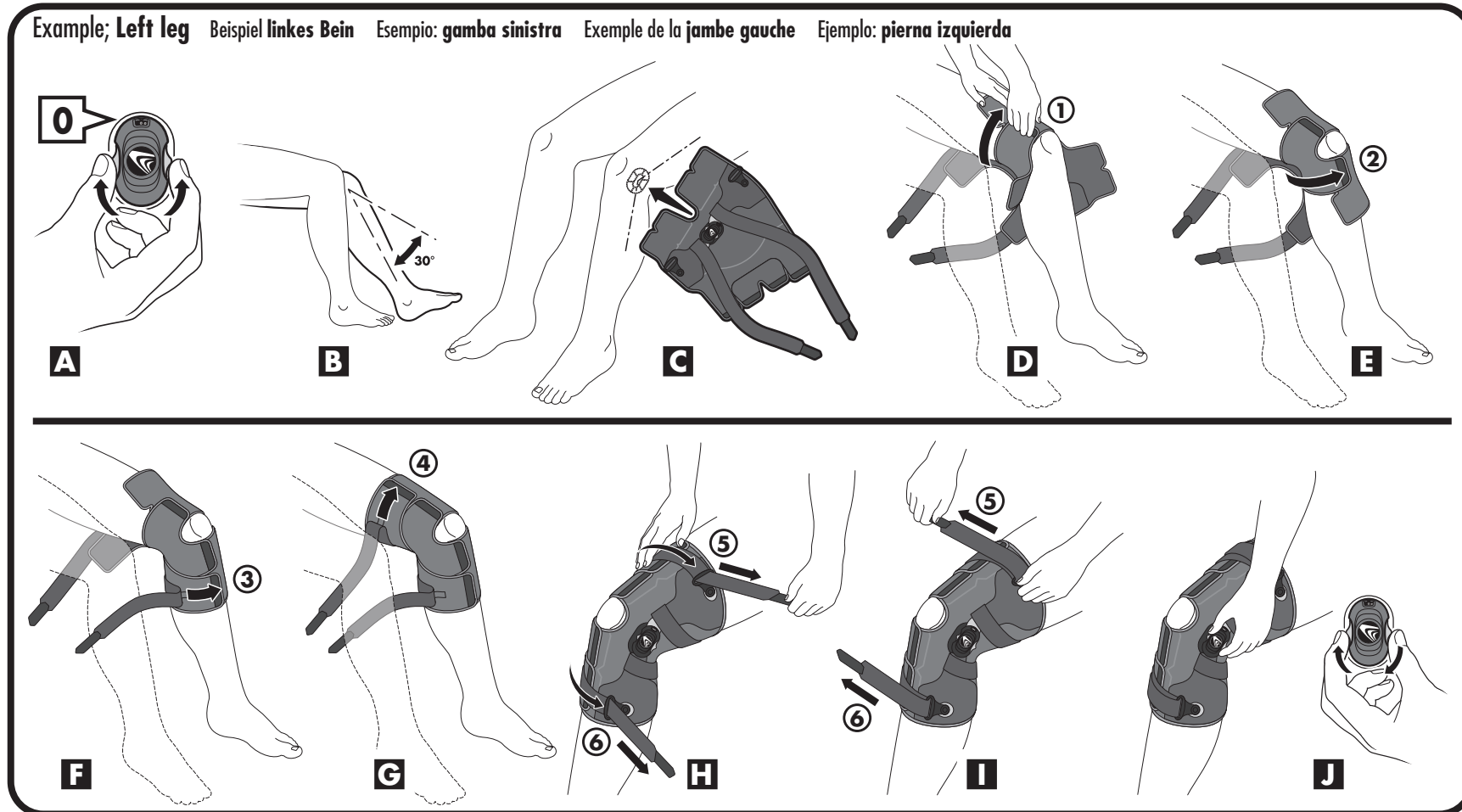
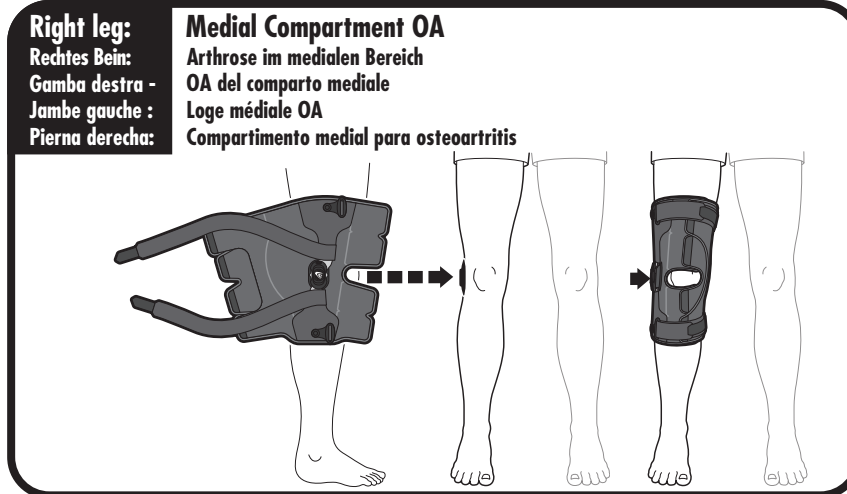
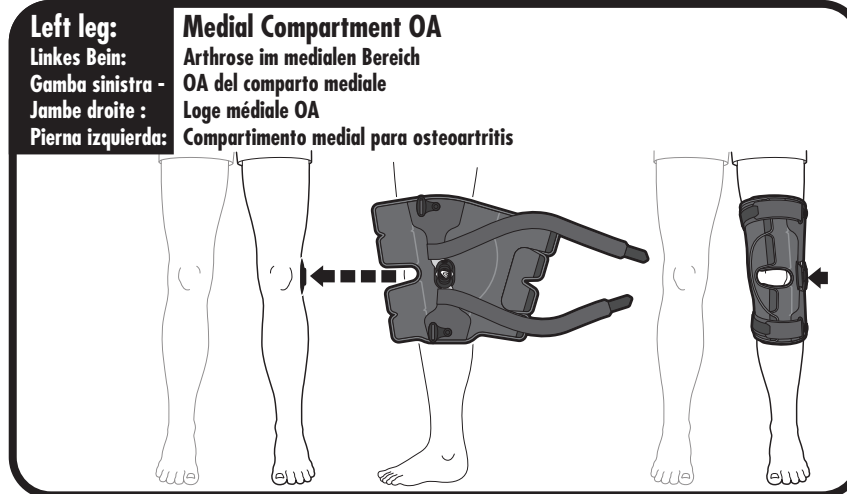
- A** Prior to application, set the hinge to 0.
- B** Sit with the knee bent at 30°
- C** Position the brace by placing the hinge on the outside of the knee.
- D** Maintain the brace position with one hand while using the other hand to pull tab #1 across the thigh above the knee and fasten securely.
- E** Pull tab #2 across the shin below the knee and fasten securely.
- F** Pull tab #3 across the lower shin and fasten securely.
- G** Pull tab #4 across the upper thigh and fasten securely.
- H** Pass strap #5 through the plastic ring on the outside of the thigh.
- I** To avoid rotation of the brace during application, secure the brace with one hand, as shown, while tensioning strap #5. Lay the strap onto itself and fasten securely. Repeat for strap #6.
- J** Once the brace is properly applied, pain relief may be achieved by turning the thumbwheel hinge dial clockwise to increase pressure to the outside of the knee.

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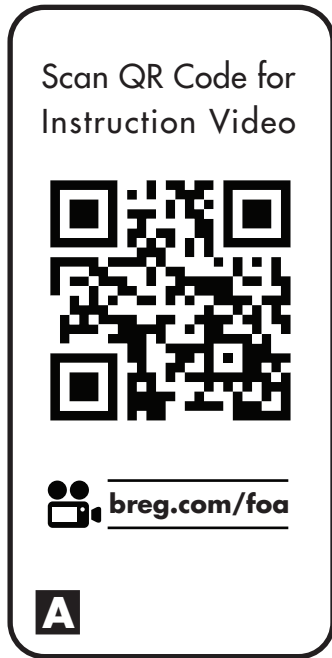A

Instruction Video

To view a video demonstrating how to use the brace, scan the QR code or go to the website:

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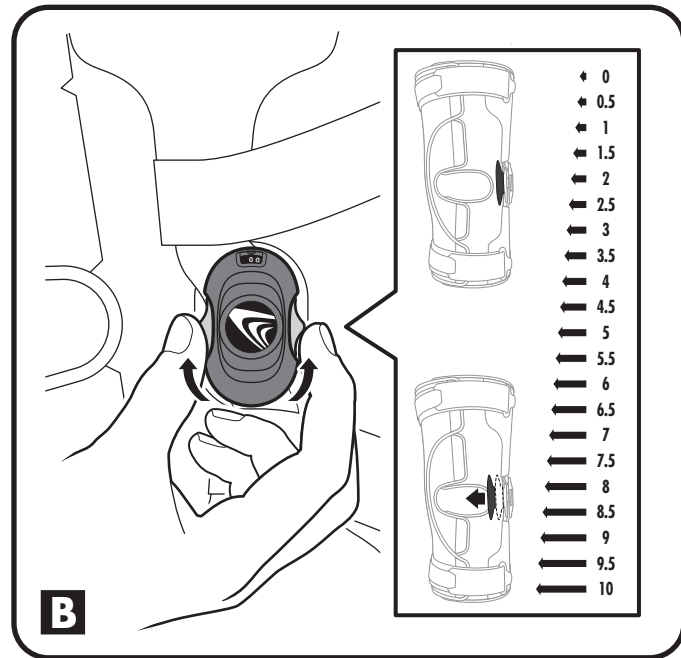
B

Hinge Adjustment

With the brace properly applied, OA pain symptoms can be relieved by adjusting the thumbwheel hinge dial. Turn the thumbwheel dial clockwise to increase pressure and counterclockwise to reduce the pressure on the outside of the knee. The hinge settings window displays settings ranging from 0 to 10mm. The display setting will increase or decrease as you adjust the thumbwheel dial.

Make sure a large gel condyle pad is attached to the hinge.

Always follow your medical professional's recommendation regarding the proper hinge setting.

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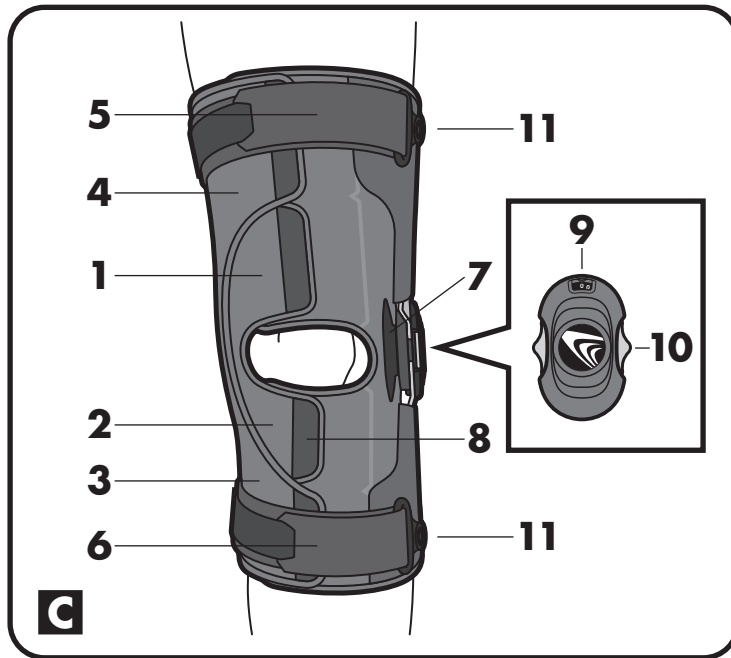
C

Product diagram

- | | |
|--------------------|----------------------------------|
| 1: Tab #1 | 7: Condyle pad |
| 2: Tab #2 | 8: Finger Pockets |
| 3: Tab #3 | 9: Hinge Settings Window |
| 4: Tab #4 | 10: Hinge Thumbwheel Dial |
| 5: Strap #5 | 11: Plastic Rings |
| 6: Strap #6 | |

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Condyle Pad Adjustment

If you require a more snug fit of the brace at the knee, shims can be installed to the condyle pad.

1. Bend the brace at the hinge to access the condyle pad.
2. Remove the condyle pad from the hinge.
3. Attach the desired number of shim(s) to the condyle pad.
4. Re-attach the condyle pad to the brace.
5. Turn the Hinge Thumbwheel Dial clockwise until the desired pressure is reached.

Important: Make sure the condyle pad is properly aligned with the hinge thumbwheel dial before applying the brace to the knee.

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Care and Cleaning

Gently hand wash the material in cold water with a mild detergent, rinse, lay flat and air dry. Apply a dry lubricant such as a Teflon spray to the hinge after washing. DO NOT machine wash or tumble dry.

Fitting Tips

- A properly applied brace should fit snug on all areas of the knee.
 - Use the finger pockets to help fasten tabs #1 – #4.
 - Avoid rotation of the brace during application of straps #5 and #6.
- Hold the brace securely with one hand at the hinge near the plastic ring while tensioning the straps with the other hand.

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Freestyle™ OA Brace

FITTING INSTRUCTIONS for Lateral OA application

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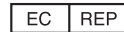
AW 1.00289 REV C 1211

⚠ WARNINGS ⚠

WARNING: CAREFULLY READ FITTING INSTRUCTIONS AND WARNINGS PRIOR TO USE. TO ENSURE PROPER PERFORMANCE OF THE BRACE, FOLLOW ALL INSTRUCTIONS. FAILURE TO PROPERLY POSITION THE BRACE AND FASTEN EACH STRAP WILL COMPROMISE PERFORMANCE AND COMFORT.

WARNING: FOR SINGLE PATIENT USE ONLY.

WARNING: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, SKIN IRRITATION, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.

WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM. CONSULT YOUR LICENSED HEALTH CARE PROFESSIONAL REGARDING SAFE AND APPROPRIATE ACTIVITY LEVEL WHILE WEARING THIS DEVICE.

CAUTION: CARE AND CLEANING OF THIS PRODUCT IS ESSENTIAL TO ITS CONTINUING STRENGTH AND PERFORMANCE. FOLLOW REMOVAL AND CLEANING PROCEDURE DESCRIBED IN INSTRUCTIONS.

CAUTION: WHEN USING BRACES BILATERALLY, MODIFY ACTIVITY AS NECESSARY TO KEEP MEDIAL HINGES FROM CONTACTING EACH OTHER.

CAUTION: WHEN USING THE FREESTYLE OA BRACE IT IS IMPORTANT TO KEEP DIRT AND GRIT OUT OF THE HINGE MECHANISM TO MAINTAIN PROPER HINGE MOBILITY AND BRACE FUNCTION.

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
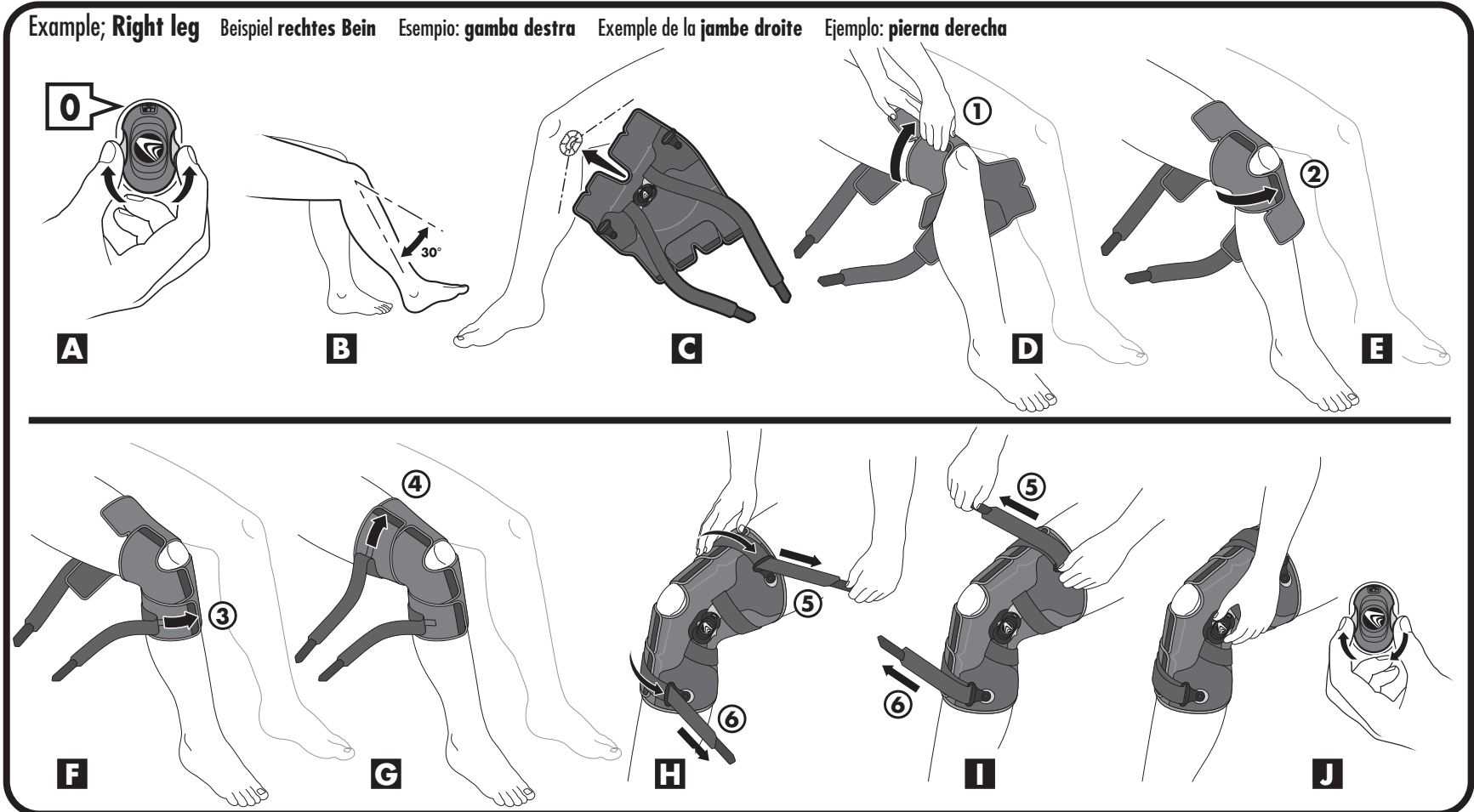
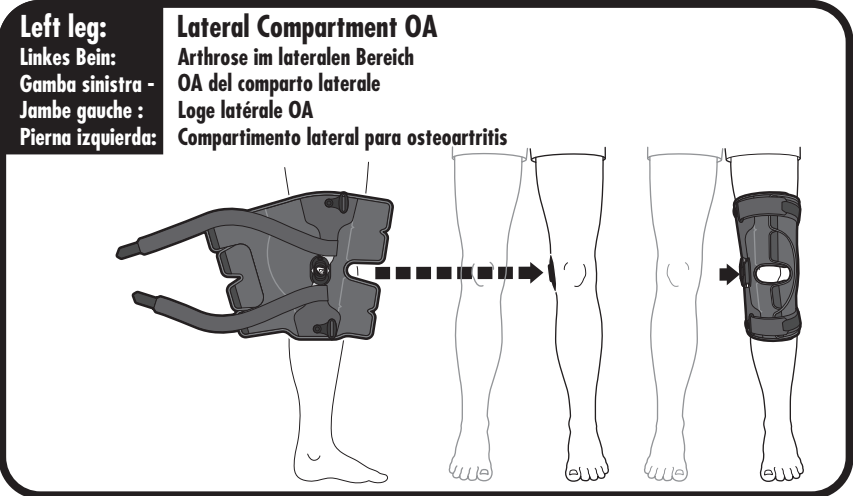
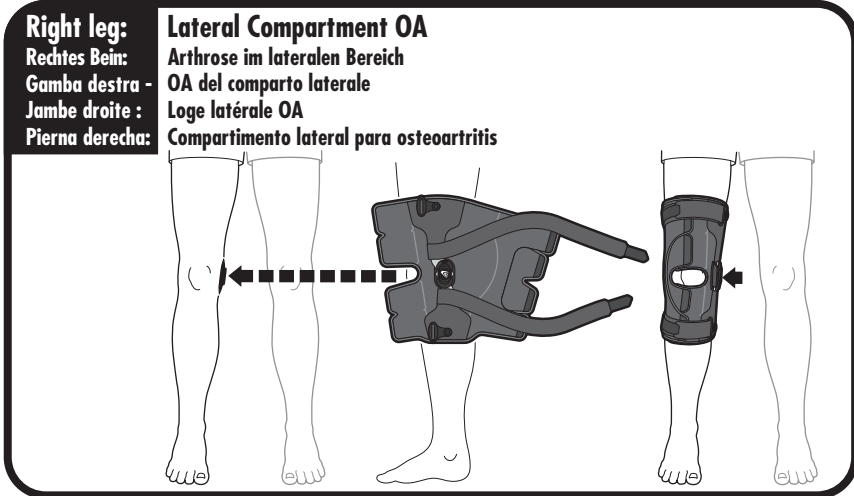
- A** Prior to application, set the hinge to 0.
- B** Sit with the knee bent at 30°
- C** Position the brace by placing the hinge on the inside of the knee.
- D** Maintain the brace position with one hand while using the other hand to pull tab #1 across the thigh above the knee and fasten securely.
- E** Pull tab #2 across the shin below the knee and fasten securely.
- F** Pull tab #3 across the lower shin and fasten securely.
- G** Pull tab #4 across the upper thigh and fasten securely.
- H** Pass strap #5 through the plastic ring on the inside of the thigh.
- I** To avoid rotation of the brace during application, secure the brace with one hand, as shown, while tensioning strap #5. Lay the strap onto itself and fasten securely. Repeat for strap #6.
- J** Once the brace is properly applied, pain relief may be achieved by turning the thumbwheel hinge dial clockwise to increase pressure to the inside of the knee.

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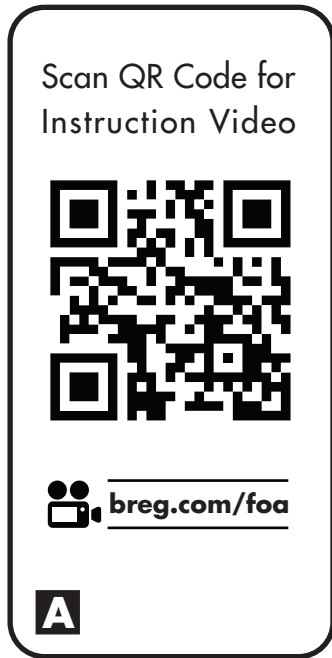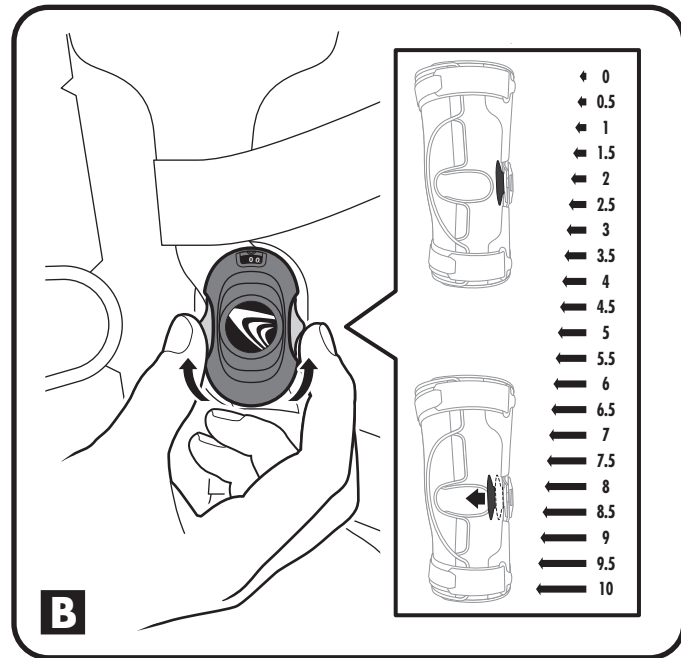
B

Hinge Adjustment

With the brace properly applied, OA pain symptoms can be relieved by adjusting the thumbwheel hinge dial. Turn the thumbwheel dial clockwise to increase pressure and counterclockwise to reduce the pressure on the inside of the knee. The hinge settings window displays settings ranging from 0 to 10mm. The display setting will increase or decrease as you adjust the thumbwheel dial.

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