T-Scope®

Full Foam-Post-Op Brace Fitting Instructions

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AW 1.00057 REV E 1012

WARNING

- WARNING: CAREFULLY READ USE/CARE INSTRUCTIONS AND WARNINGS PRIOR TO USE.
- WARNING: DO NOT REMOVE T-SCOPE BRACE UNLESS INSTRUCTED BY YOUR MEDICAL TREATMENT PROFESSIONAL. DO NOT CHANGE RANGE OF MOTION HINGE SETTINGS WITHOUT SUPERVISION BY A MEDICAL PROFESSIONAL.
- WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM. CONSULT WITH YOUR MEDICAL TREATMENT PROFESSIONAL REGARDING SAFE AND APPROPRIATE ACTIVITY LEVEL WHILE WEARING THIS DEVICE.
- WARNING: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.
- WARNING: THE HINGE ON THIS BRACE IS DESIGNED TO LIMIT AND/OR CONTROL RANGE OF MOTION. IT IS NOT DESIGNED TO STABILIZE YOUR KNEE WHEN YOU ARE WEIGHT-BEARING OR TAKE THE PLACE OF A WALKING AID. FOLLOW YOUR PHYSICIAN'S ADVICE REGARDING WEIGHT-BEARING AND ALWAYS USE A PROPER ASSISTANCE DEVICE, SUCH AS CRUTCHES OR A WALKER.
- CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.
- CAUTION: FOR SINGLE PATIENT USE ONLY.

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INITIAL APPLICATION BY A MEDICAL PROFESSIONAL ONLY!

- UNLOCK STRAP LOCK CLIPS (A), UNCLIP BUCKLES (B), REMOVE THE THIGH AND CALF FOAM PADS .
- WRAP AND FASTEN THE THIGH AND CALF FOAM PADS AROUND THE LEG SO THAT THE PAD OPENINGS ARE POSITIONED ALONG THE FRONT OF THE LEG.
- SPREAD HINGE BARS APART, LAY BRACE OUT FLAT, POSITION DEVICE WITH KNEE CENTERED BETWEEN HINGES.
Orient the brace so the hinges are facing in the direction indicated and the small calf pads are towards the feet.
- LOOSEN FRICTION CLIPS. FOR PROPER FIT SLIDE UPPER AND LOWER TELESCOPING HINGE BARS TO ACCOMMODATE LEG LENGTH. LOCK FRICTION CLIPS.
Hinge bar length indicators assist in verifying consistent length selection on thigh and calf.
- POSITION HINGE BARS LATERALLY AND MEDIAL TO THE LEG, CENTER HINGE AT KNEE JOINT.
- LOOSELY FASTEN THE 2 STRAPS CLOSEST TO THE KNEE.
- LOOSELY FASTEN THE REMAINING 2 STRAPS.
- PULL STRAPS TIGHT TO REMOVE SLACK BEHIND THE LEG. BE CAREFUL TO MAINTAIN THE LATERAL AND MEDIAL POSITIONS OF THE HINGE BARS.
- LOCK STRAP LOCK CLIPS.
- PULL STRAPS TIGHT THROUGH THE BUCKLES. BE CAREFUL TO MAINTAIN THE LATERAL AND MEDIAL POSITIONS OF THE HINGE BARS.
- SECURE STRAP ENDS, USE HOOK AND LOOP Y-TABS AT STRAP ENDS TO AFFIX STRAPS. IT MAY BE NECESSARY TO SHORTEN STRAPS BY FOLDING THEM OVER BEFORE ATTACHING Y-TABS.

ROM HINGE ADJUSTMENTS:

- EXTENSION LIMIT SETTINGS MAY BE SELECTED BETWEEN 10° OF HYPEREXTENSION AND 30° BY PULLING THE TAB OUT AND SLIDING TO DESIRED POSITION.
- FLEXION LIMIT SETTINGS MAY BE SELECTED BETWEEN 10° OF HYPEREXTENSION AND 120°.
- THE HINGE MAY BE LOCKED BY SLIDING THE QUICK LOCK BUTTON INTO THE LOCKED POSITION AT ANY ONE OF 5 POSITIONS: 10° OF HYPEREXTENSION, 0° (NEUTRAL), 10°, 20°, 30°.

USE AND CARE OF YOUR T-SCOPE BRACE:

- After initial application, the T-Scope may be removed and reapplied by unclipping the buckles and unfastening the thigh and calf foam pads!
- Hand wash the foam pads and straps with mild soap, air dry. Do not place pads or straps into a mechanical dryer.
- Extra foam pads are available from Customer Care: (800) 321-0607

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