## ⚠️ WARNINGS ⚠️

**WARNING:** Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

**WARNING:** If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

**WARNING:** This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

**CAUTION:** Follow application and care process described in instructions.

**CAUTION:** For single patient use only.

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## Buttress Support Instructions for Use

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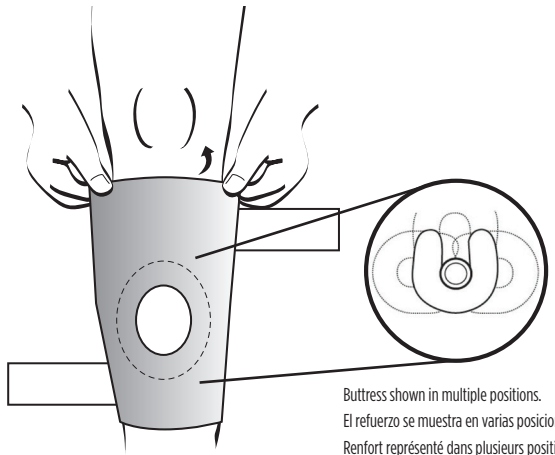
**1. Apply to Knee**

Grasp edges of the sleeve at the larger opening. Step through sleeve, pulling it up to the knee so that the opening at the front surrounds the knee cap.

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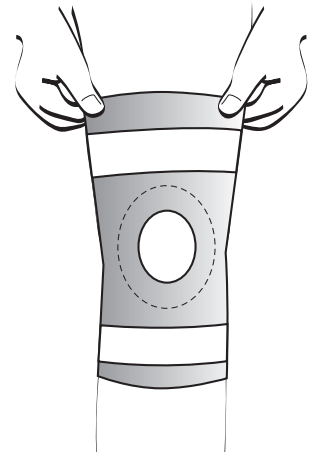
2

**2. Secure Straps**

Starting with the lower strap, feed straps through the D-rings and secure. Fit should be snug but not too tight so as to restrict circulation.

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**CARE & WASHING INSTRUCTIONS**

Hand wash in cold water with a mild detergent, rinse and lay flat to air dry.

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© 2017 Breg, Inc. All rights reserved.
MDSS GmbH