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ENGLISH
INTENDED USE/INDICATIONS:
 Indicated for immobilization of the knee, range of motion control following ACL, PCL, LCL and MCL surgeries, and meniscal repairs of the lower limb.
INTENDED USERS:
 The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.
CONTRAIKACIONES:
 Contraindicated for unstable fractures or for fractures of the proximal femur or the distal tibia or fibula.
WARNINGS AND PRECAUTIONS:
 • If any additional pain or symptoms occur while using this device, seek medical attention.
 • Do not leave this device in the trunk of a car on a hot day or in any other place where the temperature may exceed 150°F (65°C) for any extended period of time.
 • This device is offered for sale by or on the order of a physician or other qualified medical professional.
 • This device is not intended for reuse on a second patient or for extended use beyond 180 days.
 • This device is not intended to prevent injuries or reduce or eliminate the risk of reinjury.
INSTRUCTIONS FOR USE:
 1. **Release the over-center buckles.**
 a. Pinch the two buttons on the buckle releasing the buckle.
 b. Place brace beside the leg aligning the hinge with the joint line.
 2. **Set range of motion.**
 a. Move uprights to the desired position between the prescribed range of motion (ROM).
 b. Press the gray extension button and slide it to the desired setting. The button will spring out to set the extension range of motion.
 c. Press the gray flexion button and slide it to the desired setting. The button will spring out to set the flexion range of motion.
 d. To lock the brace at -10, 0, 10, 20, 30, or 40 degrees, move the uprights aligning the indicator mark with the prescribed setting in orange. Then slide the orange lock tab toward the hinge. This will lock the brace in place.
 3. **Set tibial and thigh length.**
 a. Slide the locking clip to free the length adjustment button.
 b. Depress length adjustment button and slide tibial section until ankle plates are just above the ankle bone.
 c. Slide the locking clip in place.
 d. Repeat length adjustment for the thigh.
 Note: If using a shoe insert, apply it now by following instructions in the Extender Plus Shoe Insert section of this application instructions.
 4. **Apply to leg.** With the leg extended. Slide the brace underneath the leg with closures at the front. Make sure the hinge is aligned with the knee.
 5. **Adjust the back of the straps.** Adjust the back of the strap by releasing the hook and pile attachment and pulling strap until slack is removed. Reattach the hook and pile. Make sure upright maintains the midline position.
 6. **Front strap adjustment.**
 a. Attach the over-center buckle.
 b. Detach the hook and loop attachment from the strap end and tighten to desired tension.
 c. Trim off excess strap and attach the hook and pile attachment.
 7. **Instruct patient.** Show patient how to get out of the brace by pinching the tabs on the over-center buckles, and sliding leg out of the brace.
SHOE INSERT INSTRUCTIONS
 A. **Insert the key.** Complete steps 1 and 2 of the Extender Plus Brace application instructions. Insert key (provided) behind spring by depressing the button and sliding the key between the spring and upright.
 B. **Remove the tibial extension.** Turn the key 90°. Slide the tibial extension out.
 C. **Attach the shoe insert to the brace.** Slide the shoe insert into the brace and stop at desired length. Turn the key 90° and remove. Repeat for the other side.
 D. **Apply sock.** Place the sock (included with Extender Plus Shoe Insert) on the foot. Proceed to step 3 of the Extender Plus Brace application instructions.

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