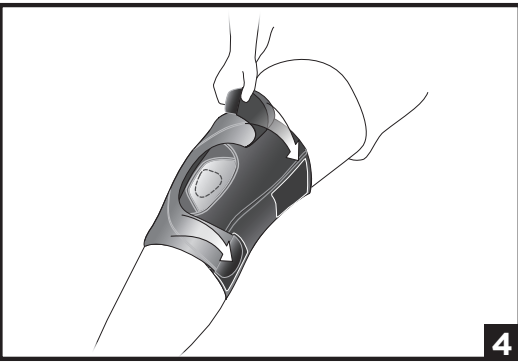
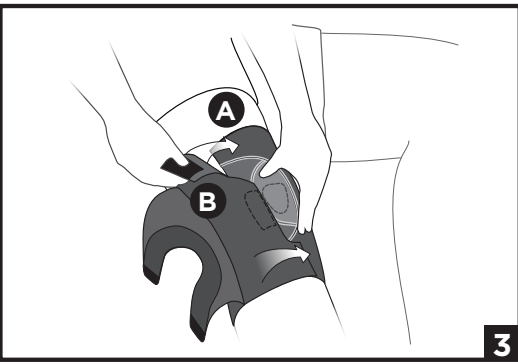
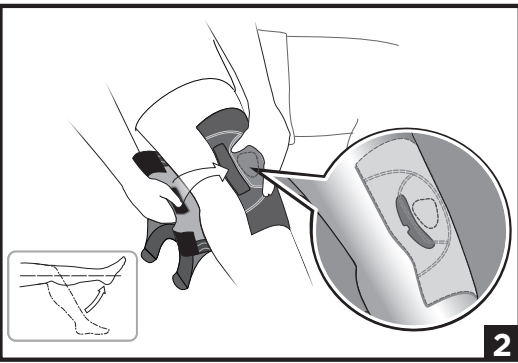
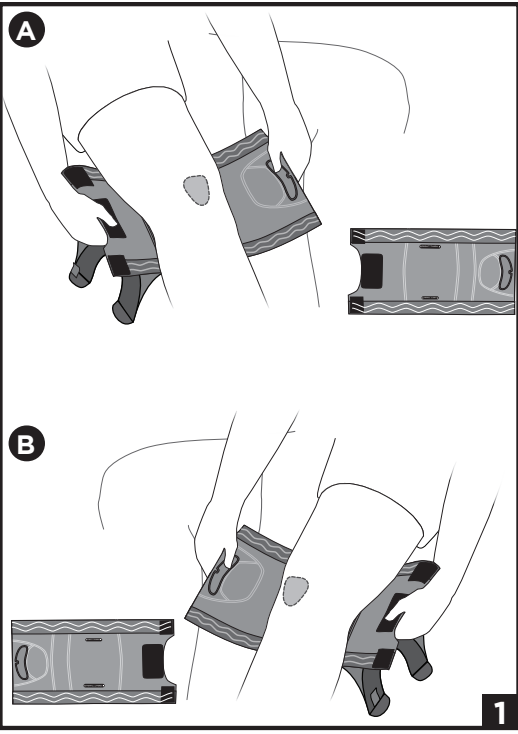
ENGLISH

INTENDED USE/INDICATIONS:

Patellofemoral knee bracing products are designed to provide control and support for patients who have patellofemoral discomfort resulting from maltracking, subluxation, or other patellofemoral injuries.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠ WARNINGS AND PRECAUTIONS:

- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.
- If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- Follow application and care process described in instructions.
- For single patient use only.

INSTRUCTIONS FOR USE:

Make sure you are seated with your leg straight.

- Open up the brace completely. Make sure the brace is positioned for the correct leg by using the labels "right thigh" 1A or "left thigh" 1B. on the fabric inside the brace.
- Line up the hinges of the brace with the top of the kneecap then wrap the inside of the brace over the kneecap making sure the buttress lies next to the outside of your kneecap.
- Attach the center hook first A, then the two outside straps B.
- Attach the outside strap to the opposite loop on the inside of the knee.

USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry. After exposure to salt water or dirt, thoroughly rinse the brace with fresh water and allow to air dry.

MATERIAL CONTENT:

High density polyethylene (HDPE), Lycra, Nylon, Polyester, Stainless Steel, thermoplastic elastomer (TPE).

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

NOTICE TO USERS: If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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