Intelli-Flo 3 X 5 Pad FITTING INSTRUCTIONS

for Patients and Medical Professionals

⚠ WARNING

The Intelli-Flo Pad can be cold enough to seriously injure the skin. The Intelli-Flo Pad is non-sterile and for single patient use only. Do not attempt to sterilize the Pad. Follow these instructions and the Operating Instructions and Product Inserts provided with the Polar Care Kodiak Motorized cold therapy units (available at www.breg.com or 1-800-321-0607).

1. **Discuss treatment with your licensed health care practitioner.** Provide a complete medical history including any reactions to cold. Certain medical conditions make cold-induced injury more likely. Ask your practitioner about potential adverse reactions and cold-induced injuries.
2. **Use only according to your practitioner's instructions** regarding the frequency and duration of cold application and breaks, how and when to inspect the skin, and total length of treatment. Federal law restricts this device to sale by or on the order of a licensed health care practitioner.
3. **Apply an insulation barrier to gray plastic channeled side** (such as the BREG Polar Dressing provided with each 3x5 pad, Webril, Kerlix, cast padding, elastic bandage) that provides complete coverage between the Intelli-Flo Pad and skin. **Do not let any part of the Intelli-Flo Pad touch skin.** If a sterile dressing has been applied to the treatment site that does not completely cover the skin under the pad, use an additional insulation barrier. The Intelli-Flo Pad alone is too cold to be applied directly to the skin.
4. **The Intelli-Flo 3x5 pad is only indicated for use on hand/wrist and foot/ankle applications.** Application to any body part other than what is indicated could lead to cold-induced injury.
5. **Apply the Intelli-Flo Pad** with the gray plastic channeled side towards the skin. Refer to the graphics below. Do not wrap pad so tightly as to restrict normal circulation, as this can cause injury.
6. **Connect the Intelli-Flo Pad** couplings to the Polar Care Kodiak unit couplings. The Intelli-Flo Pad hose couplings are only compatible with the Polar Care Kodiak unit couplings.
7. **Follow the Polar Care Kodiak Operating Instructions and Product Insert** inside the pouch; both are affixed to the unit.
8. **Inspect the skin** under the Intelli-Flo Pad (by lifting the edge) as prescribed, typically every 1 to 2 hours. Do not use the Polar Care Kodiak if dressing, wrapping, bracing, or casting over the Intelli-Flo Pad prevents skin inspections. Stop using and contact your practitioner immediately if you experience any adverse reactions, such as: increased pain, burning, increased swelling, itching, blisters, increased redness, discoloration, welts, other changes in skin appearance, or any other reaction identified by your practitioner.
9. **To end cold therapy,** disconnect electrical power and press the two metal tabs on the hose coupling and gently pull apart or remove the Intelli-Flo Pad from your body.

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
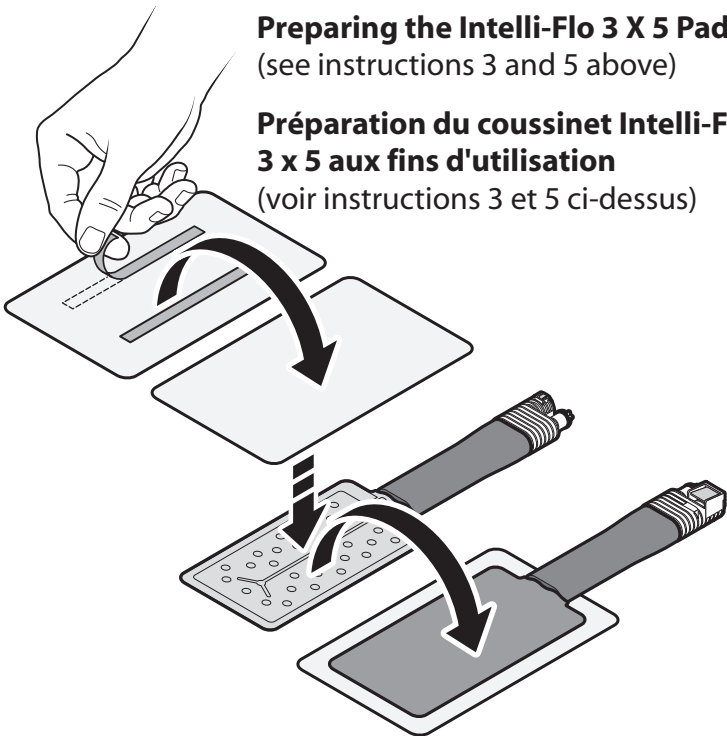
Preparing the Intelli-Flo 3 X 5 Pad for use
(see instructions 3 and 5 above)

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Connecting and disconnecting
the pad to the Kodiak
(see instructions 6 and 9 above)

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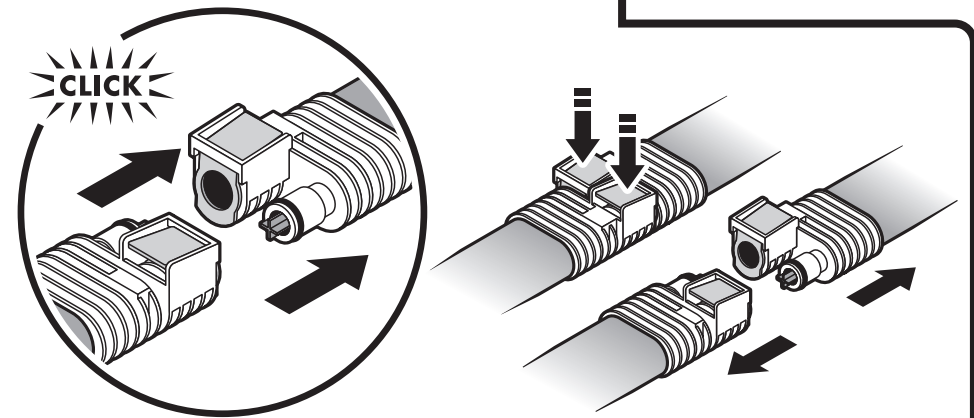Application Examples
Application types may vary

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