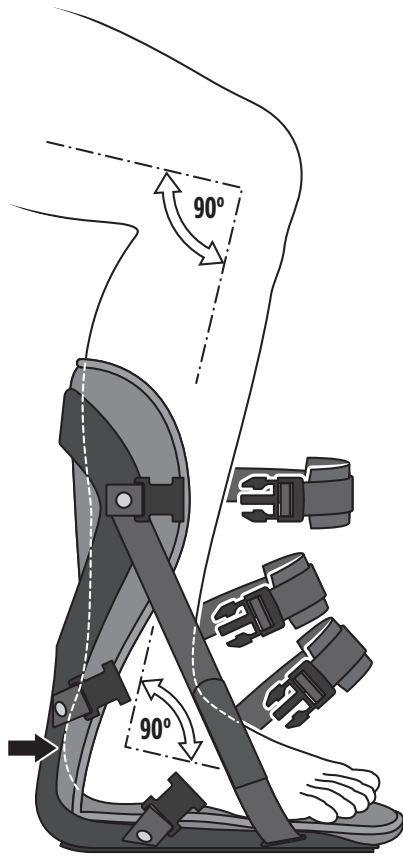
1

Open all 3 buckles. Place the affected foot in the splint with knee in flexed at 90°.

The heel should be at the very rear of the splint and the toes should not extend past the end of the splint. The ankle should be at 90° when the splint is properly positioned.

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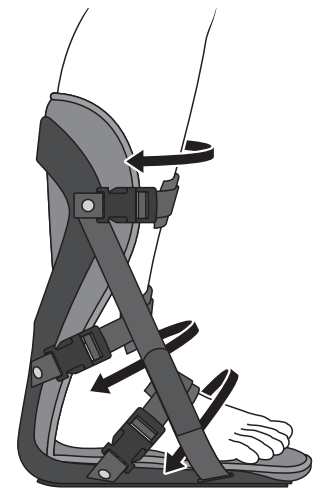
2


Fasten the buckles. If necessary, adjust the buckles to ensure a secure and comfortable fit.

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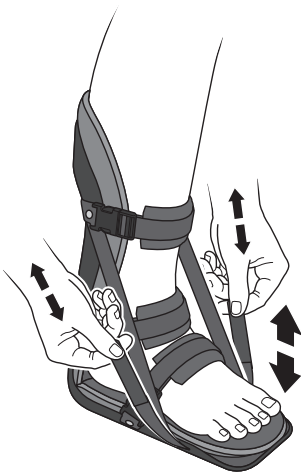3

Release the hook and loop straps on each side of the foot bed. Pull up, or lower the straps to adjust plantar fascia stretch.

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4

To achieve additional stretch, pull the front portion of the liner away from the footplate and insert the 5° foam wedge between the liner and the footplate.

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E/U authorized representative
MDS GmbH
Schiffgraben 41
D-30175 Hannover
Germany

WARNINGS

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

WARNING: If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

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© 2019 Breg, Inc. All rights reserved.