
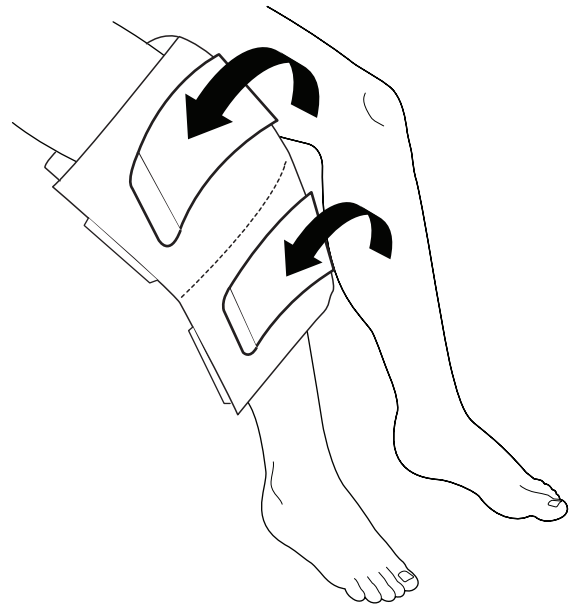


Polar Care Gel Wrap Knee

APPLICATION INSTRUCTIONS

1. Remove wrap and insert gel packs into pockets.
Note: Place gel packs into freezer until frozen
2. Place wrap over affected knee.
3. Secure the attached straps around the knee to hold wrap in place.
4. Secure to wrap with hook tab.
5. Adjust straps accordingly for comfort.
6. Never apply a frozen or heated pack directly onto the skin.
7. Always follow your physician's or healthcare provider's specific cold therapy treatment.

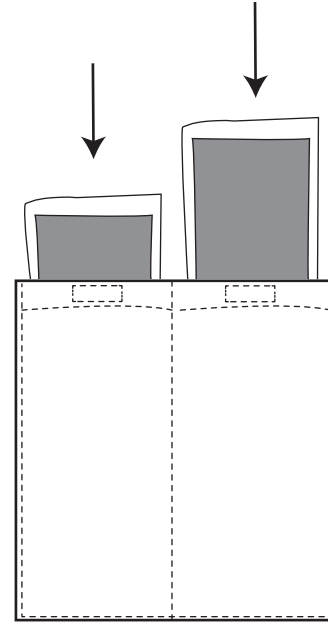

Breg, Inc.
2885 Loker Ave. E, Carlsbad, CA 92010 USA
Toll Free Tel: 1-800-321-0607
Local: +1-760-795-5440 Fax: +1-760-795-5295
www.Breg.com
© 2016 Breg, Inc. All rights reserved.

M.2616 Rev A 03/16