ENGLISH

INTENDED USE/INDICATIONS:

Indicated for post rotator cuff procedures, labral repair, lesions, dislocations, joint reconstruction.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠ WARNINGS AND PRECAUTIONS:

- FOR SINGLE PATIENT USE ONLY.
- IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.
- CAREFULLY READ FITTING INSTRUCTIONS AND WARNINGS PRIOR TO USE.
- THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM.

INSTRUCTIONS FOR USE:

- While supporting the injured arm, slide the forearm into the sling. Wrap the adjustable shoulder strap around the neck and shoulders and attach the strap end to the sling using the "alligator strap tab."
- Adjust the shoulder strap to the required length.
- Position the shoulder pad behind the neck. To secure the shoulder pad location (optional), pull the shoulder strap away from the shoulder pad, unfold the hook material on the top of the pad, then press the shoulder strap back onto the hook material to secure the pad in place.
- Attach the closure strap across the open top of the sling. Position the strap so that it holds the arm securely in the sling. Next, attach the thumb strap to the open end of the sling between the thumb and fingers.
- After sling has been fit, it may be easily removed and reapplied using the quick release buckle on shoulder strap.
- If a neutral pillow or 15° abduction pillow is included, place the pillow at the waistline. Attach the sling to the pillow, lining up hook material on the pillow with the loop on sling. Adjust the waist strap to fit.
- If waist strap is too long, cut it to fit. Use the small piece of double sided hook material (located on top of the pillow) to secure the strap end. Place the double sided hook material on the inside of the cut strap end and secure it to the waist strap.
- If no pillow is included, attach the waist strap to the sling and adjust to fit.

USE AND CARE:

Straps and sling must be removed and cleaned regularly depending on your activity level and perspiration. Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

MATERIAL CONTENT:

Cotton, Nylon, Polyester, Polyacetel, Polyurethane.

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

NOTICE TO USERS: If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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