INDICATIONS FOR USE WITH ARC 2.0:

For non-surgical or post-operative support and immobilization of the injured shoulder when varying degrees of internal rotation, external rotation and abduction are required. Positions include gunslinger, neutral plane, and statue of liberty.

INDICATIONS FOR USE WITH ARC 2.0 WITH PILLOW: For non-surgical or post-operative support and immobilization of the injured shoulder where glenohumeral positioning is key.

CAUTION: This device is not intended for reuse on a second patient or for extended use beyond 180 days.

CAUTION: Activity modification and proper rehabilitation are an essential part of a safe

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treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.

CAUTION: This device is not intended to prevent injury, or reduce or eliminate the risk of re-injury.

CAUTION: This device has been developed for the indications listed above, and may be damaged by unusual activities such as sports, falls, or other accidents. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

CAUTION: Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150F (65 C.) for any extended period of time.

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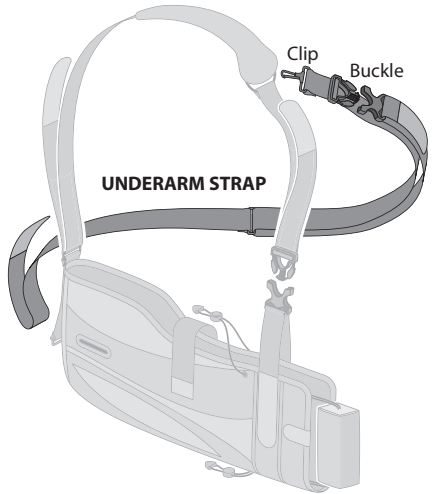
www.Breg.com
© 2015 Breg, Inc. All rights reserved.


EC	REP
----	-----

E/U authorized representative
MDSS GmbH
Schiffgraben 41
D-30175 Hannover
Germany

AW-CP020271 Rev B 10/15

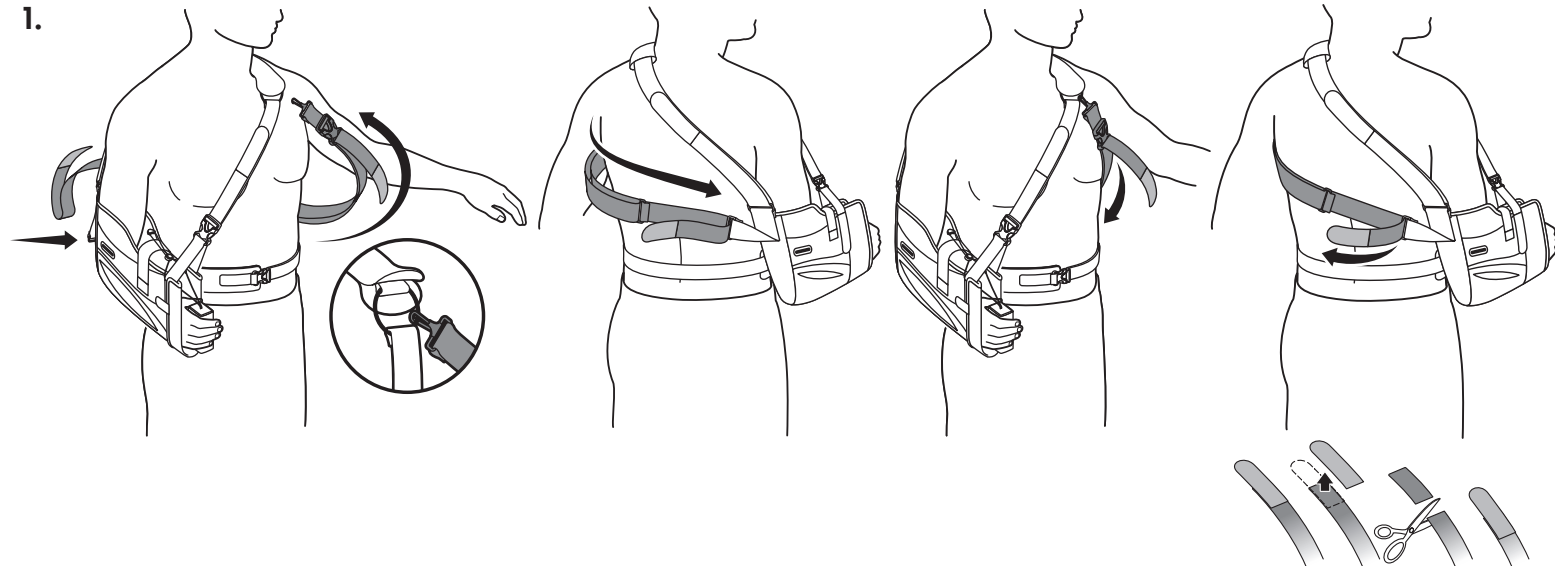1.

English

APPLYING THE UNDERARM STRAP TO THE ARC 2.0 (1.) AND TO THE ARC 2.0 WITH PILLOW (2.)

Example: Right Shoulder

Attach clip to the ring inside neck pad and pass underarm strap under the unaffected shoulder. Attach loose end of the underarm strap to the D-ring at the back end of the sling. Loosen the hook closures at either end of the strap and pull the strap ends, front and back, to a comfortable tension. Refasten the closures.

NOTE: Excess strap may be trimmed if necessary.

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