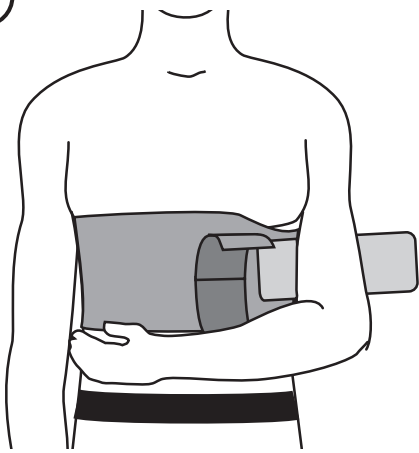
1

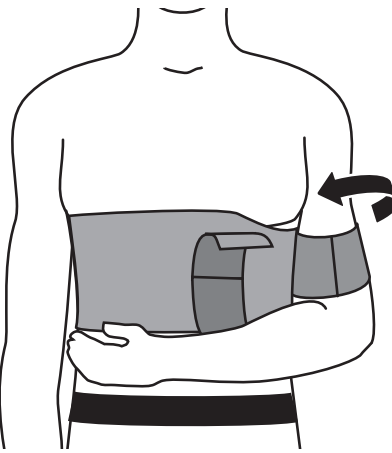
Secure immobilizer swathe around chest area with cutout portion under arm that is to be immobilized.

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2


Attach humeral cuff to immobilizing swathe with hook closure.

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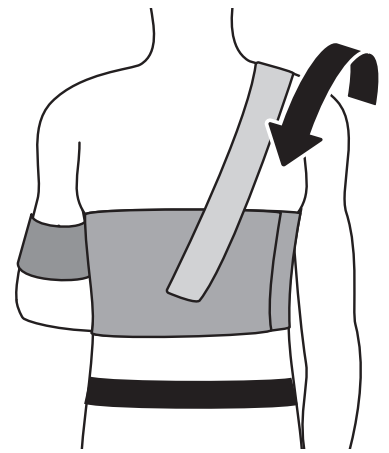3


For Deluxe version: Place adjustable shoulder strap over the unaffected shoulder, and secure to the back of the immobilizing swathe.

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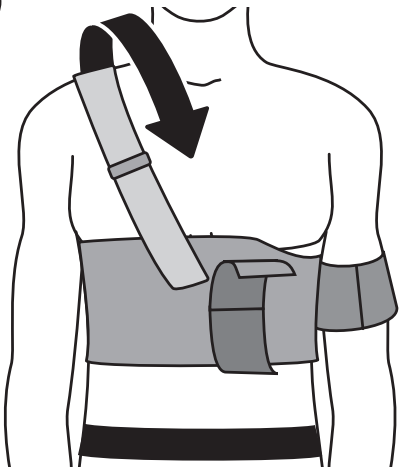4


For Deluxe version: Adjust shoulder strap for length and secure to front of the immobilizing swathe.

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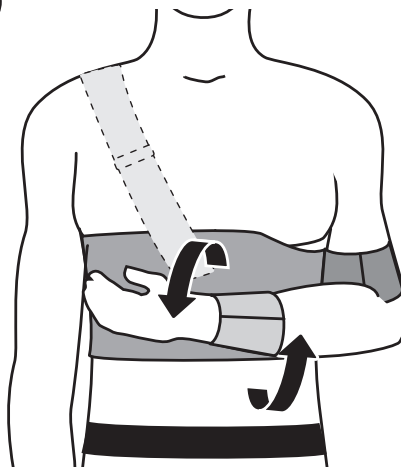5

Wrap and secure wrist/hand cuff to the immobilizing swathe.

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Deluxe Straight Shoulder Immobilizer

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WARNINGS

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

WARNING: If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

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Breg, Inc.