⚠️ WARNINGS ⚠️

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

WARNING: If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

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Deluxe Shoulder Slings and Immobilizers Instructions for Use

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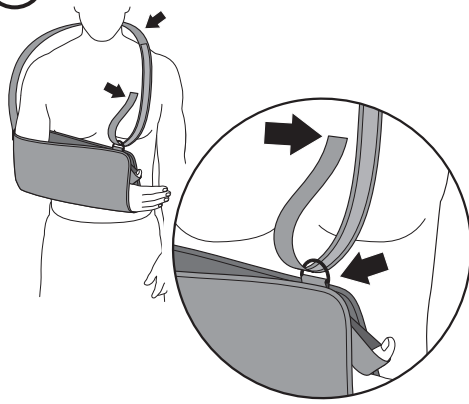
1

Wrap shoulder strap around neck and shoulders. Lace strap end through D-ring(s) and attach to shoulder strap.

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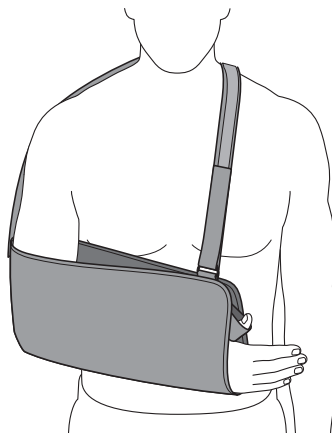
2

Properly applied sling with arm comfortably supported, and thumb in the thumb loop (if applicable).

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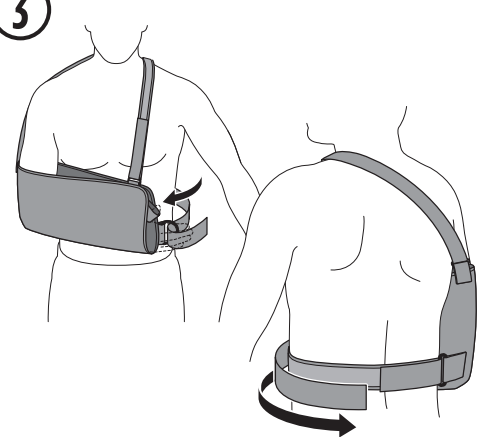
3

Shoulder Immobilizer: After fitting the sling as indicated above, wrap waist strap around waist and feed through plastic ring. Attach strap with hook and loop closure.

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CARE & WASHING INSTRUCTIONS

Hand wash in cold water with a mild detergent, rinse and lay flat to air dry.

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© 2019 Breg, Inc. All rights reserved.