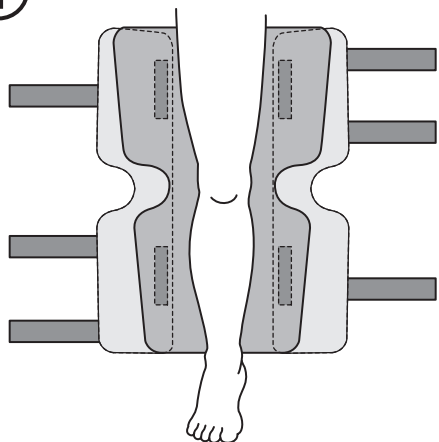
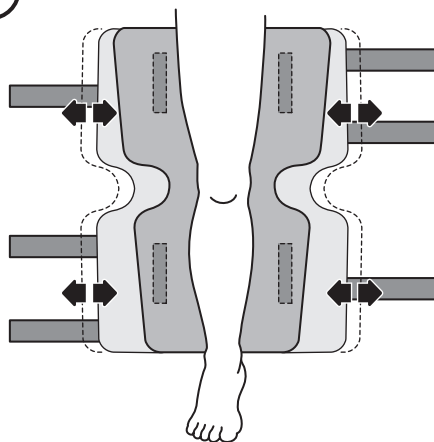
1

Place the immobilizer under or behind the leg.
Center the leg on the rear stay.

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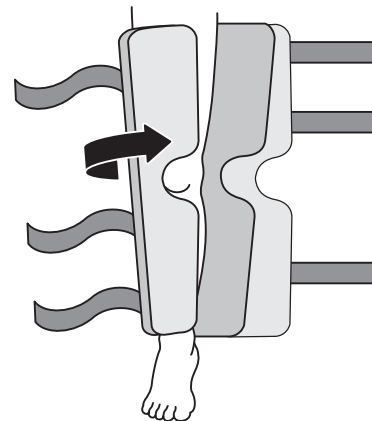
2

Adjust the removable panels laterally for correct sizing of thigh and calf circumference.

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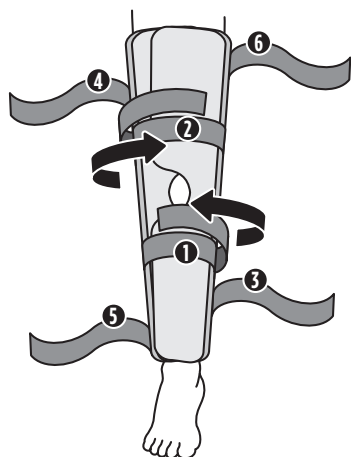
3

Once the immobilizer has been correctly adjusted, wrap the panels around the leg.

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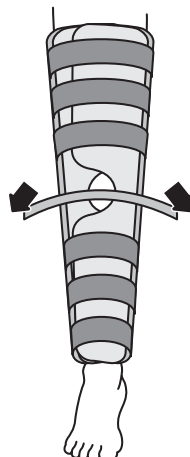
4

Starting closest to the knee, pull the straps through the D-rings and secure with the hook tabs provided. Then secure the remaining straps most distal and proximal on the immobilizer. Trim straps if necessary.

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5

If applicable, secure the patella strap over the center of the knee and attach to the hook tabs both sides.

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Deluxe Tri-Panel Knee Immobilizer Instructions for Use

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WARNINGS

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

WARNING: If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

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Breg, Inc.