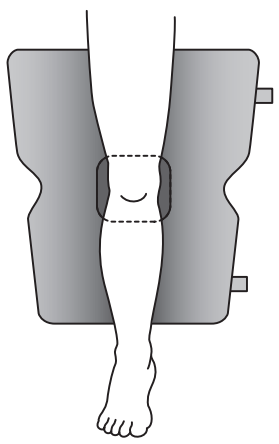
1

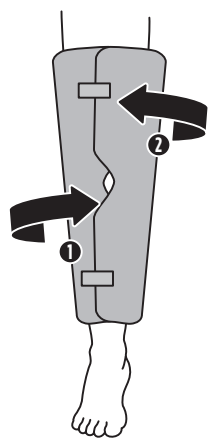
Place the immobilizer under or behind the leg.
Center the leg on the rear stay.

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2


Wrap the right half of the immobilizer around the knee, followed by the left.

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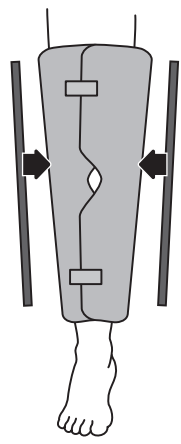3


Place the medial and lateral stays at the midline of the leg.

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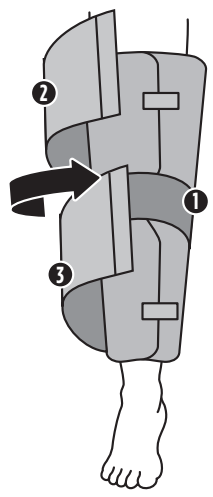
Starting with the center strap, secure the compression straps around the knee.

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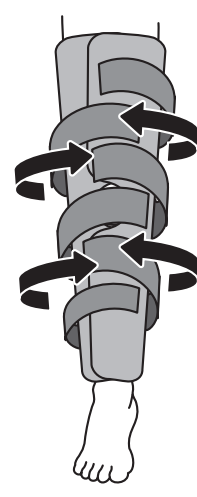Compression Knee Immobilizer Single Panel Knee Immobilizer

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WARNINGS

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

WARNING: If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

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Breg, Inc.