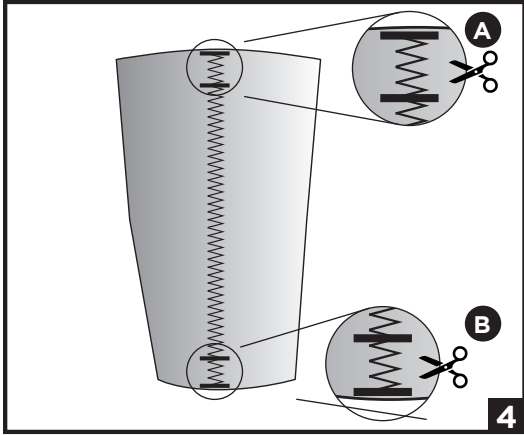
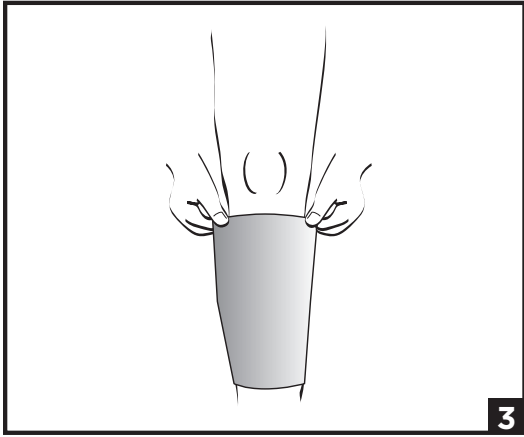
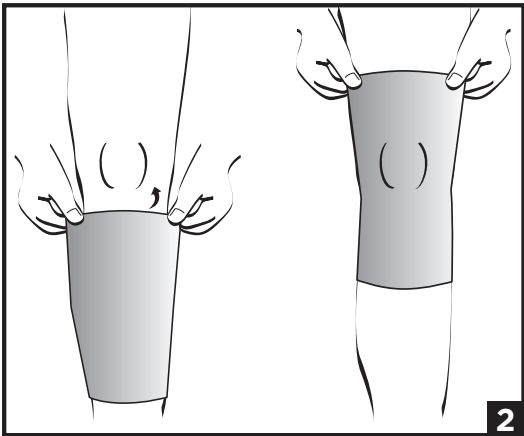
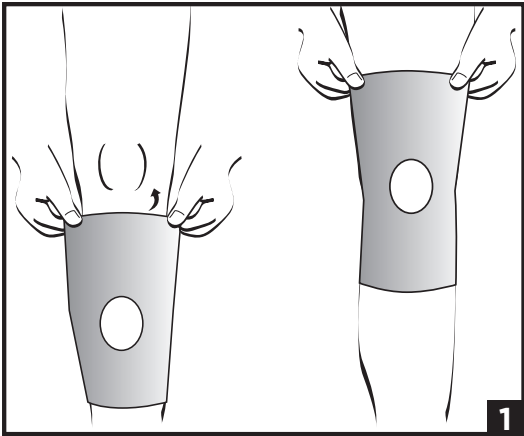
ENGLISH

INTENDED USE/INDICATIONS:

To provide compression; patella stabilization; support for hamstring, quadriceps injuries.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠ WARNINGS AND PRECAUTIONS:

- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.
- If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- For single patient use only.

INSTRUCTIONS FOR USE:

NEOPRENE KNEE SUPPORT

- Open Patella:** Grasp edges of the sleeve at the larger opening. Step into it and pull it up your leg. Position the sleeve so your kneecap is centered in the opening, with the top of the sleeve resting on your upper thigh.
- Closed Patella:** Grasp edges of the sleeve at the larger opening. Step into it and pull it up your leg. Center the sleeve over your kneecap, with the top of the sleeve resting on your upper thigh.

CALF SLEEVE

3. Apply Calf Sleeve

Grasp edges of the sleeve at the larger opening. Step through sleeve, pulling it up to the calf so it is centered below the knee and above the ankle.

4. Adjust length (optional)

To adjust length, cut between the additional hem line stitches at the top and/or bottom of the sleeve as shown.

- Cut between hem line stitch to shorten length (optional)
- Cut between hem line stitch to shorten length (optional)

THIGHT SUPPORT

- Grasp edges of the sleeve at the larger opening. Step through sleeve, pulling it up above the knee so it is centered above the knee and the top of the thigh. (See figure 3)

USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

MATERIAL CONTENT:

Neoprene, Nylon, Polyester

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

NOTICE TO USERS: If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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