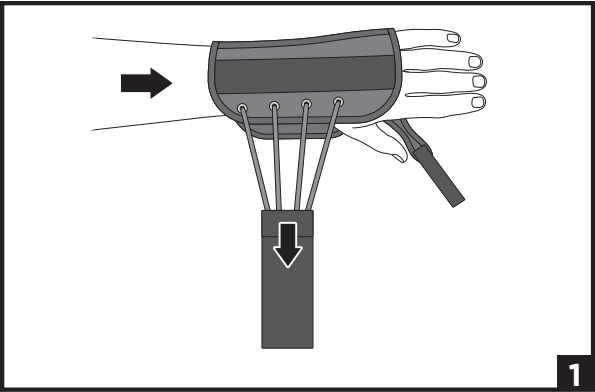
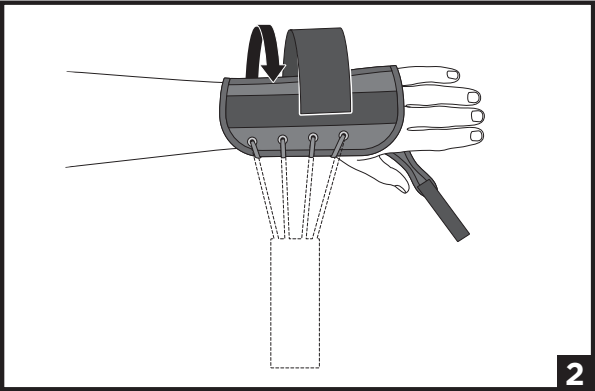
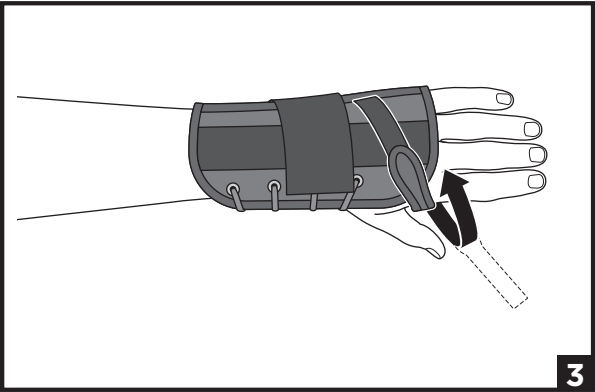
1

2

3

## ENGLISH

### INTENDED USE/INDICATIONS:

The intended use for wrist bracing products is to provide support and / or immobilization to the wrist for patients with wrist injury.

### INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

### CONTRAINDICATIONS:

Not Applicable.

### ⚠ WARNINGS AND PRECAUTIONS:

- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.
- If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- For single patient use only.

### INSTRUCTIONS FOR USE:

Example illustrates left hand brace application.

1. Slide hand into the brace and pull hook strap to tighten brace.
2. Wrap strap around the brace, and secure to the loop closure.
3. Line up the palmar stay below the middle crease of the hand and secure the thumb strap. Adjust for comfort.

### USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

### MATERIAL CONTENT:

Aluminum, Brass, Cotton, Nylon, Polyester, Polypropylene.

### WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see [www.breg.com](http://www.breg.com)

### DISPOSAL:

Dispose of product according to local regulations.

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