⚠️ WARNINGS ⚠️

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

WARNING: If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

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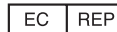
Patella Stabilizer Instructions for Use

Estabilizador de rótula Instrucciones de uso Stabilisateur de rotule Mode d'utilisation


2885 Loker Ave East
Carlsbad, CA 92010 USA
Toll Free Tel: 1-800-321-0607
Local: +1-760-795-5440
Fax: +1-760-795-5295
www.breg.com

© 2017 Breg, Inc. All rights reserved.

E/U authorized representative
MDSS GmbH
Schiffgraben 41
D-30175 Hannover
Germany

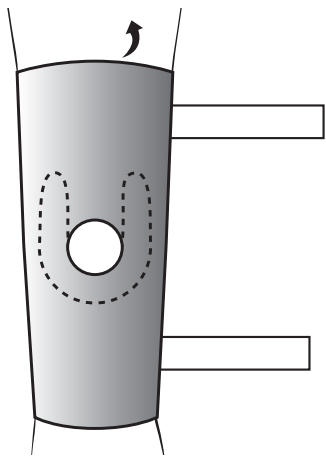1

1. Apply to Knee

Grasp edges of the sleeve at the larger opening. Step through sleeve, pulling it up to the knee so that the opening at the front surrounds the knee cap.

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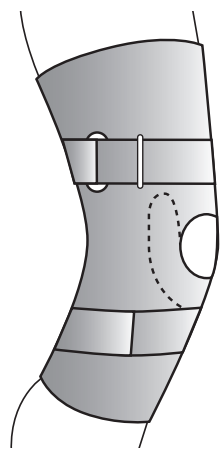
2

2. Secure Straps

Starting with the lower strap, feed straps through the D-rings and secure. Fit should be snug but not too tight so as to restrict circulation.

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CARE & WASHING INSTRUCTIONS

Hand wash in cold water with a mild detergent, rinse and lay flat to air dry.

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