1

1. Place wrap above Achilles tendon and around the smallest part of the ankle.

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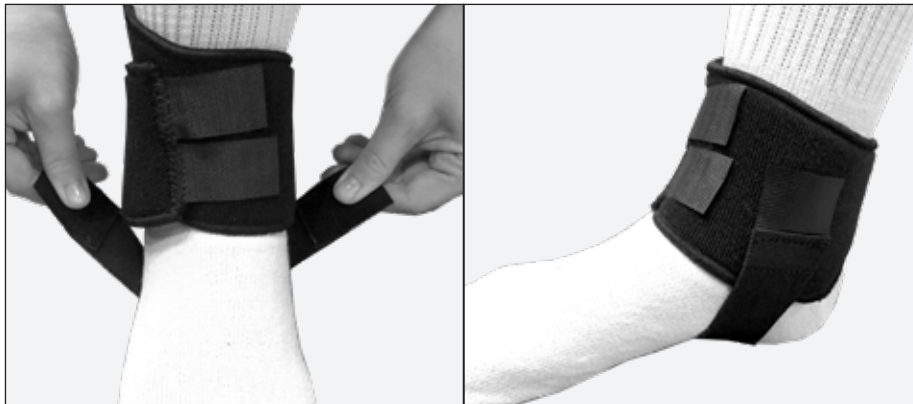
3

3. Remove elastic strap from wrap. Apply the elastic strap forward of the heel and back of arch. Wrap under the arch and pull up and back (approximately 15°). Attach hook to wrap. Adjust if necessary.

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© 2019 Breg, Inc. All rights reserved.

WARNINGS

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

WARNING: If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

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