## ENGLISH

### INTENDED USE/INDICATIONS:

The intended use for the wrist bracing products is to provide stability, support, and /or immobilization to the Wrist joint for patients recovering from a wrist injury or surgery.

### INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

### CONTRAINDICATIONS:

Not Applicable.

### WARNINGS AND PRECAUTIONS:

- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.
- If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- For single patient use only.

### INSTRUCTIONS FOR USE:

- (A) Wrist Lacer:** Slide the brace onto affected hand. Be sure thumb and thumb hole on the side of the brace are aligned.  
**(B) Wrist Lacer w/ Thumb Spica:** Slide the brace onto affected hand. Be sure thumb and elastic thumb loop are aligned.
- (A) Wrist Lacer:** Pull single (or double if 10" version) closure strap to tighten laces.  
**(B) Wrist Lacer w/ Thumb Spica:** Pull single closure strap to tighten laces. Slide thumb into elastic thumb loop.
- Wrap strap around brace and secure hook and loop closure.
- (A) Wrist Lacer:** Close thumb strap between thumb and fingers and secure to brace.  
**(B) Wrist Lacer w/ Thumb Spica:** Wrap thumb strap around the injured thumb and secure to brace.
- The curvature of the metal stay on the bottom of the brace should fit into the palm of the hand and extend down the forearm.
- If necessary, remove the malleable metal stay and bend it to the appropriate position.
- Trim hook and loop closure straps leaving approximately 1" – 2" of overlap.

### USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

### MATERIAL CONTENT:

Aluminum, Nylon, Polyester

### WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

### DISPOSAL:

Dispose of product according to local regulations.

NOTICE TO USERS: If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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