

**1**

Pull On  
Colocación  
Tirez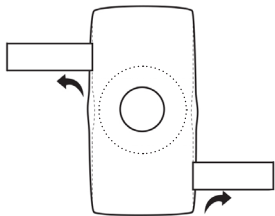

Wrap around  
Enrollar alrededor  
Enroulez autour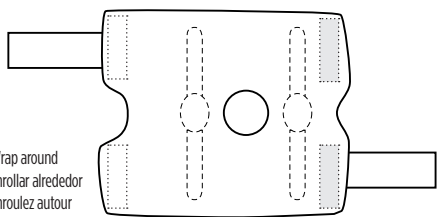


Unfasten straps.

Afije las correas.

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
**2**

Pull On  
Colocación  
Tirez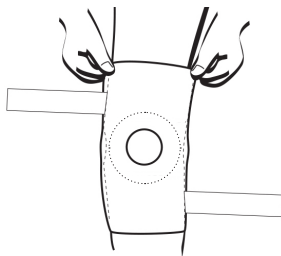

Wrap around  
Enrollar alrededor  
Enroulez autour

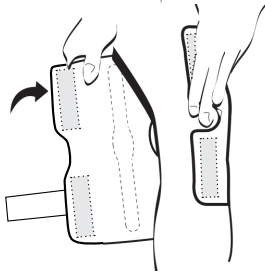
Place foot into the widest opening of the brace. Pull the brace up the leg until hinges align with the knee. For the wrap around version, align the hinges on the sides of the knee. Wrap the brace around the leg and secure the flaps. The opening should be directly over the knee cap.

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
Pull On  
Colocación  
Tirez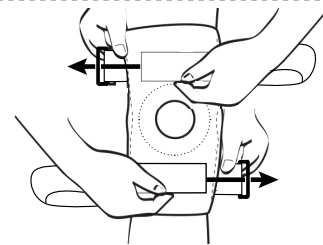

Wrap around  
Enrollar alrededor  
Enroulez autour

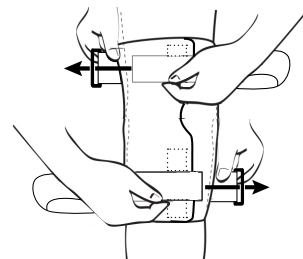Loop straps through the rings and pull back over the top of the lower thigh and upper calf, then fasten the hook and loop closure.

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**4**

Adjust straps for comfort and fit. The brace should be snug, but not uncomfortably tight. **NOTE:** Do not overtighten, adjust as necessary.

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## Hinged Knee Support Instructions for Use

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## WARNINGS

**WARNING:** Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

**WARNING:** If you experience or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

**WARNING:** This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

**CAUTION:** If swelling increases or pain, numbness or skin irritation occurs, discontinue use and contact your medical provider immediately. This product is only a support device, not to prevent injuries or prevent recurrence of injuries.

**CAUTION:** Follow application and care process described in instructions.

**CAUTION:** For single patient use only.

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© 2019 Breg, Inc. All rights reserved.