ENGLISH

INTENDED USE/INDICATIONS:

The intended use wrist bracing products is to provide stability, support, and/or immobilization to the wrist joint for patients recovering from a wrist injury or surgery.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠ WARNINGS AND PRECAUTIONS:

- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.
- If you experience or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- If swelling increases or pain, numbness or skin irritation occurs, discontinue use and contact your medical provider immediately. This product is only a support device, not to prevent injuries or prevent recurrence of injuries.
- Follow application and care process described in instructions.
- For single patient use only.

INSTRUCTIONS FOR USE:

UNIVERSAL WRIST BRACE

1. Slide affected hand into brace, gently guiding the thumb through the thumb hole.
2. Position the bottom front edge of the brace below the palmar crease.
3. Thread loop straps through the D-rings.
4. Gently pull on the straps until the brace is taut; the **iv** panel will automatically adjust to the user's hand size. The proximal strap (**i**) locks the panel into functional size. NOTE: Do not overtighten. For best results, adjust the strap a second time to ensure a comfortable fit or continue to adjust as necessary.
5. Final application.

UNIVERSAL WRIST BRACE W/ THUMB SPICA

- A. Slide affected hand into brace, gently guiding the thumb into the thumb support.
- B. Position the bottom front edge of the brace below the palmar crease.
- C. Thread loop straps through the D-rings.
- D. Gently pull on the straps until the brace is taut; the **iv** panel will automatically adjust to the user's hand size. The proximal strap (**i**) locks the panel into functional size. NOTE: Do not overtighten. For best results, adjust the strap a second time to ensure a comfortable fit or continue to adjust as necessary.
- E. Apply loop thumb strap around thumb attaching to hook. Final application.

USE AND CARE:

Remove metal stay. Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

MATERIAL CONTENT:

Aluminum, Nylon, Polyester, Polyoxymethylene (POM)

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

Versa Fit is a registered trademark of Ortho Systems, Inc.
AW-1.03606 Rev C 04/25
© 2025 Breg, Inc. All rights reserved.
www.breg.com
Fax: +1-760-795-5295
Local: +1-760-795-5440
Toll Free Tel: 1-800-321-0607
Carlsbad, CA 92008 USA
2382 Faraday Avenue, Suite 300

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