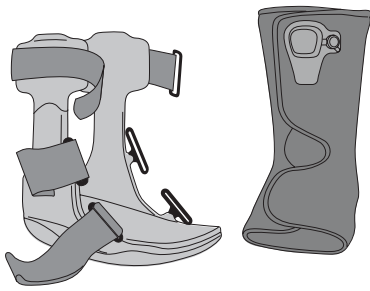
1

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

Unfasten hook and loop straps and remove inner soft liner from boot.

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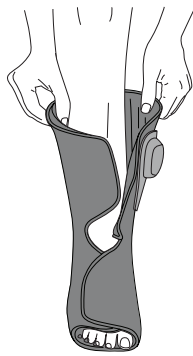
2


Open the soft liner and place foot inside with the heel fitting snugly into the back portion of the liner. Fasten and wrap foot flaps then the leg portion of the liner. Ensure liner is snug but not constrictive. Make sure pump is facing outward.

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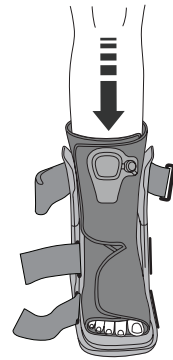3


Gently spread the side struts and place the lower leg into the brace.

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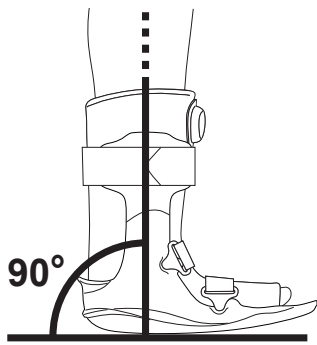4


The ankle should be centered at the struts with the foot and ankle at 90°.

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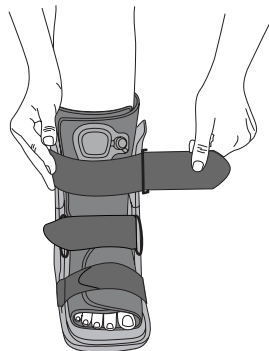5

Fasten the hook and loop straps beginning at the toe then moving up the leg. Do not overtighten. As swelling pattern changes throughout the day due to activities, rest, etc., it may be necessary to adjust the straps for comfort.

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Softgait Air Walker Short Instructions for Use

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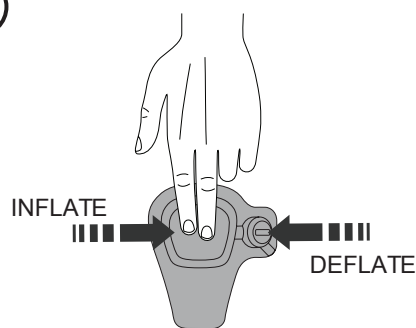
6

To inflate air liner, press and release center air bulb until liner provides a comfortable cushion and support. Do NOT overinflate.

To deflate or adjust compression, press down on smaller circular side button.

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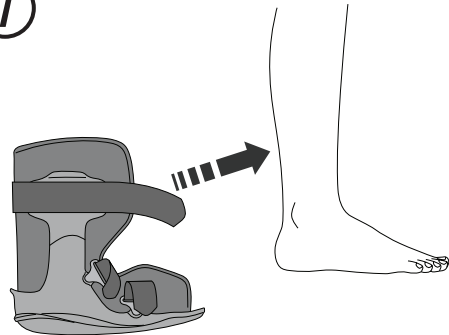
7

To remove, simply unfasten the straps without removing them completely from the assembled walker. Unfasten the soft liner and gently bring the foot and ankle forward and out of the walker.

To reapply, place the foot and ankle into the liner and reattach the hook and loop straps (As per Fig. 5).

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WARNINGS

INTENDED USE / INDICATIONS:

The Softgait Air Walker Short provides support for stable injuries to the foot and/or ankle, soft tissue injuries, and/or post-operative use.

WARNINGS AND PRECAUTIONS:

Do not overinflate aircells. Improper aircell inflation may cause significant skin irritation, especially in people with diabetic neuropathy (a diabetic nerve condition). Check your skin frequently for "hot spots" and skin irritation. If you see any skin changes or feel discomfort, reduce the aircell compression. Do not use this device if you cannot feel sensations whether due to post-op anesthesia, or nerve, skin or other conditions.

This product is not intended to be used directly on open wounds or broken skin. If pain and/or numbness occurs, contact your medical professional immediately. Ask prescribing medical professional which footwear to use.

Use caution when walking on wet or slippery surfaces to avoid injury. At high altitudes, the aircells will expand beyond optimal level. Adjust aircell pressure by tightening or loosening straps.

Always consult with your medical professional before making changes to the brace. This product is to be used under the supervision of a medical professional. This device is not intended for unsupervised public use.

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© 2019 Breg, Inc. All rights reserved.